GIMA

www.gimaitaly.com

GIMA Spa - Via Monza, 102 - 20060 Gessate (MI) - Italia

ITALIA: Tel. 199 400 401 (8 linee r.a.) - Fax 199 400 403

E-mail: gima@gimaitaly.com - www.gimaitaly.com

EXPORT: Tel. ++39 02 953854209 - Fax ++39 02 95380056

E-mail: export@gimaitaly.com - www.gimaitaly.com

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CRUTCHES

Carefully read this manual before using the crutches.

The crutches must be used properly, holding them by the handgrips and only to support the body (weight capacity 130 Kg). Use under a doctor's direction. The crutches must be adjusted correctly according to how they need to be used and to the body of the patient following the doctor's instructions.

Before use verify that the crutches are not damaged, that they are adjusted correctly and that the anti-skid rubber tips are not worn. If worn replace immediately.

Be very careful when the crutches are used on wet or inclined surfaces.

ADJUSTMENT

Forearm crutches Cod. 27793

The height can be adjusted between 95 and 117 cm by pressing the pin and making it click into the right slot.

The clamping screw avoids play between the two tubes and must be tightened only when the pin is inserted.

Standard crutches Cod. 27798

The height can be adjusted between 111 and 131,5 cm by pressing the pin and making it click into the right slot.

It is also possible to adjust the distance between the arm cuffs and the handgrip between 34 and 49 cm: remove the nut and slide out the screw from the handgrip, insert the screw into the desired hole and fix by tightening the wing nut.

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