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OPTION 9201C: REF. 1093 - REF. 1090 - REF. 1091 - REF. 1092

OPTION 9201C: REF. 1093 - REF. 1090 - REF. 1091 - REF. 1092

PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

Elastic orthosis with a thermoformable pad to stabilize the lumbar and lumbar-dorsal rachis and relieve pressure

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR)...

TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V. Sensitive® is a registered trademark of EUROJERSEY

SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or compression. It is advisable to not too tighten the device so as to avoid generating areas of excessive local pressure...

WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient.

SELECTION/SIZE

Table with columns: Item, Size, XS S M L XL XXL XXXL, Pelvic circum. cm, Front height cm, etc.

MAINTENANCE

- Do not bleach, No chemical cleaning, Do not iron, Do not tumble-dry, Washing instructions: Handwashing...

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PELOTT

- Handwash with mild detergent at a maximum of 30°C, leave to dry away from direct heat...

Dispose of this product and its parts responsibly

INDICATIONS REF. 1090/1091/1093

- Lumbago, Sciatic lumbalgia and lumbar cruralgia, Spondylarthrosis and lumbar discopathies, Slight traumas to the lumbosacral column...

INDICATIONS REF. 1092

- Treatment of osteoarthritis of the spine, Rheumatic diseases, Post-surgical protection of vertebral fractures...

CONTRAINDICATIONS

- Spinal stays fixed in an oblique position reinforce the abdominal section, 4 linear back pockets allow the insertion of optional rigid reinforced stays...

CHARACTERISTICS AND MATERIALS REF. 1090/1091/1093

- Structure in elastic Sensitive® fabric, breathable, laminated, rip proof, and antistatic, Adjustable pulley system with elastic straps for custom fit support...

CHARACTERISTICS AND MATERIALS REF. 1092

- Spinal stays fixed in an oblique position reinforce the abdominal section, 4 linear back pockets allow the insertion of optional rigid reinforced stays...

CHARACTERISTICS AND MATERIALS REF. 1093

- Structure in elastic Sensitive® fabric, breathable, laminated, rip proof, and antistatic, Adjustable pulley system with elastic straps for custom fit support...

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