MOPEDIA

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I Class Medical Device according to Directive 93/42/EEC
and further modifications

CODE

RP748 -Folding walker with seat - 4 swivel castors

INTRODUCTION

Thank you for purchasing the MOPEDIA walker by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

INTENDED USE

The walker is intended for people with movement difficulties and can be used as aid for deambulation.

CAUTION

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the walker category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.


- To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:
- The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive
- The devices in question ARE NOT MEASUREMENT DEVICES
- The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION
- The devices in question are sold in NON STERILE packing
- The devices in question should be considered as belonging to class I
- DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
- Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the walker
- Keep the packed walker away from heat sources
- SERVICE LIFE- the walker use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the walker

GENERAL DESCRIPTION

Parts view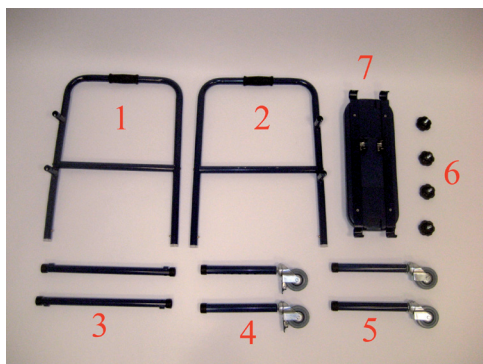


Description:

1. Frame, right side
2. Frame, left side
3. Connection bars (left and right)
4. Back legs complete with castors and brake (couple)
5. Front legs complete with castors (couple)
6. Knobs
7. Seat

(Fig.1)

ASSEMBLING

1. Open the packaging and verify the contents corresponds to the model you requested and no damages have been occurred during the transport. In contrary case contact your dealer
2. Assembly the right frame with the left frame by using the bars. Lock them with the equipped 4 knobs (Fig.2)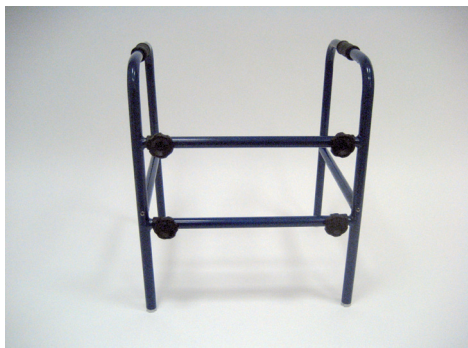


(Fig.2)

3. Insert the legs into the frame, paying attention the legs with castors and brakes are in back side (Fig.3)

NOTE Insert the legs pressing the locking clip on the frame. Ensure that the clip is completely out (Fig.4)

**THE LEGS MUST BE ASSEMBLED AT THE SAME HEIGHT.
VERIFY THE LEGS ARE ADJUSTED ON THE SAME HOLE**

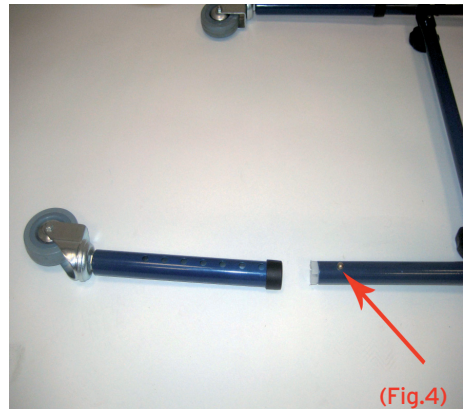4. Assembly the seat as shown in Fig.5

YOUR DEVICE IS CORRECTLY ASSEMBLED AND READY TO BE USED

BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly, paying particular attention to the correct knob assembling and to the correct legs engagement
- Always check the brakes and castors correctly work

USING WARNINGS

This walker guarantee a better stability, support and safety to a patient. However, it can't be used by patient with a low functional of arm or hand, and by patient with balance problems. The walker can be used both indoor or outdoor. We suggest to use on surfaces where the castors are able to completely touch the ground. It can't be used on rock surfaces or on inclined surfaces.

The device is not intended for people and other material transport

HOW TO USE

Use the walker

NOTE: take reference to general warnings chapter.

Use the walker as aid device for deambulation.

Use the handles and wheel the walker achieving a functional and reliable support to walk. It can be also used as seat.

NOTE TO USE THE SEAT PLEASE LOCK THE BRAKES FOLLOWING THE INSTRUCTION

Use the castors with brakes

The back castors are provided with brakes, to lock the walker.

To apply the brake, push the level on the castor.

To unlock the brake pull the level on the brake (Fig.6)

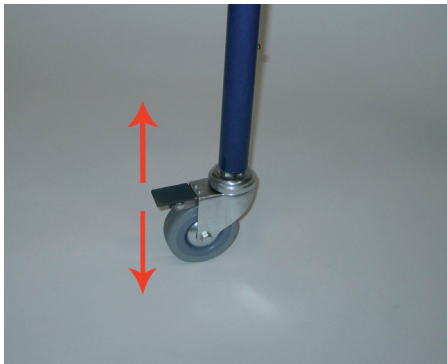
Fig.6

Use the seat

The seat is intended to give a short refreshment to the patient.
Unhook the seat from the walker front tubes and hook on the lateral tubes (Fig.7).
Before sit, be sure the brakes are duly locked.

Fig.7

MAINTENANCE

The MOPEDIA devices by Moretti are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized once a year.

We also recommend a periodic examination to the following parts:

- The walker frame
- Castors and brakes
- Welding points

In case of repair, use only original spare parts and accessories.

CLEANING AND DISINFECTION

Cleaning:

Use a damp cloth and mild soap. Then dry thoroughly before use

Never use water with temperature above 30°C

Never use jet water or steam washing machine

Disinfection

if you need to disinfect the device use a common disinfectant cleaner

NOTE Never use acids, alkalis or solvents such as acetone

CONDITIONS OF DISPOSAL

General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the walker in the appropriate disposal areas for recycling.

SPARE PARTS/ACCESSORIES

For spare parts and accessories refer to main catalogue.

TECHNICAL FEATURES

Dimension

Dimensions: cm 55x 49

Min. height : cm 76

Max. weight: cm 91

NOTE Never use acids, alkalis or solvents such as acetone

Technical specifications

Max weight	130 kg
Material	Aluminum frame, foam seta covered by PVC

WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

REPAIRING

-Warrantu repair-

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

-Repair not covered by warranty-

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

-Non-defective device-

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

WARRANTY CERTIFICATE

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Sold to _____

Street _____ Place _____

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com