GIMA

ASPIRATORE TOBI MANUALE

ASPIRATOR TOBI MANUALE

ASPIRATEUR TOBI MANUALE

ABSAUGER TOBI MANUALE

ASPIRADOR TOBI MANUALE

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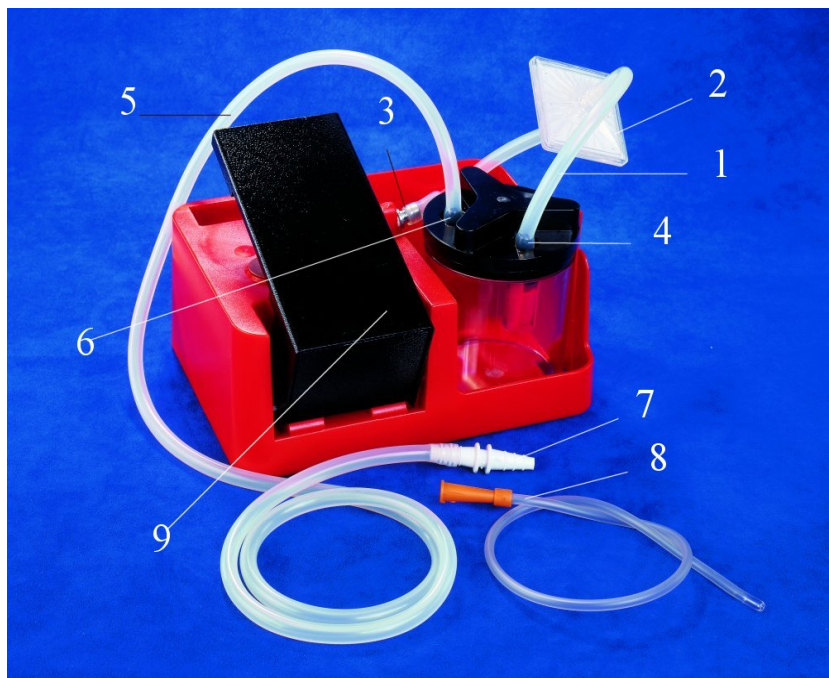
TOBI MANUALE it's a manual device to be used for the emergency field to be used for the aspiration of the body liquids (mucus or catarrh or blood). Easily movable and designed for a manual use. Thanks to this characteristics and to the rating that it has product is particularly suitable for a emergency use, on the thacheotomized patients, minor surgical applications and post - operative therapy at home. Made of highly heat resistant, in conformity with the latest European safety standard, the product is supplied with a polycarbonate 0.4 litre autoclavable jar with overflow valve.

GENERAL WARNINGS

READ INSTRUCTION MANUAL CAREFULLY BEFORE USE

ONLY HIGHLY QUALIFIED STAFF USE RESERVED

THE INSTRUMENT MUST NOT DISASSEMBLED
FOR TECHNICAL SERVICE ALWAYS CONTACT GIMA S.p.A.

IMPORTANT SAFETY RULES

1. When opening packing verifies the device state: always check the presence of damages on the plastic surface, that can make accessible inside parts of the device.
In such case don't never use the device. Check integrity before every use.
2. Keep special attention to:
 - Use only original accessories;
 - The device can be used only with the bacteriological filter;
 - Place instrument on stable and flat surfaces and to avoid the blockage of the air inlets on sides;
 - Keep off the reach of children or not capable people without supervision;
 - Use and keep the instrument in a safe environment, protected from bad weather condition and keep off excessive heating;
3. **The device must be used for the use it was designed.**
A different use must be considered improper and dangerous.
Manufacture cannot be considered responsible for damages due to improper use or connection to an electrical system not complying with current regulations.
Do not use for anything else but the use defined by manufacturer;
4. Instrument and accessories discharging must be done following current law regulations in every country of use.
5. For repairs, exclusively contact GIMA S.p.A. technical service and request the use of original spare parts. Failure to comply with the above can jeopardise the safety of the device;

TECHNICAL CHARACTERISTICS

Model	TOBI MANUALE
Typology (MDD 93/42/EEC)	Medical Device Class IIa
Classification UNI EN ISO 10079-2	MEDIUM VACUUM / 22l/min
Alimentation	Manual Force
Maximum suction aspiration (without jar)	- 40kPa (- 0.40 bar)
Weight	1.150 Kg
Dimension	225 x 165 x 85 mm
Working condition	Room temperature: 5 ÷ 35 °C Room humidity percentage: 30 ÷ 75 % RH
Conservation condition	Room temperature: - 40 ÷ 70 °C Room humidity percentage: 10 ÷ 100% RH

SYMBOLS

	CE marking in conformity with EC directive 93/42/CEE Manufactured by: CA.MI. di Attolini Mario & C. snc Via Ugo La Malfa 31 – 43010 Pilastro (PR) Italy
	Warning, consult the instruction manual
	To preserve in place coolness and dry land
	Conservation temperature: - 40 ÷ 70° C

ACCESSORIES SUPPLIES

DESCRIPTION
COMPLETE ASPIRATION JAR 400cc
CONICAL FITTING
TUBES SET 6mm x 10mm (TRANSPARENT SILICON)
ANTIBACTERIAL FILTER

The filter is produced with (PTFE) hydrophobic material which prevents fluids entering the pneumatic circuit. When the filter is wet, it's not possible to use the unit therefore the filter should be changed immediately. In case of possible contamination or discolouration, change the filter immediately. Don't use the suction unit without the protection filter fitted. If the suction unit is used in an emergency or in a patient where the risk of contamination is not known the filter must be changed after each use.

CLEANING OF ACCESSORIES

Wearing disposable latex gloves, proceed to clean the accessories as follows:

1. Grip the jar with your hand and turn the cap in a counter-clockwise direction.
2. Remove the red cage covering the float located in the jar cap (conical part with 3 openings)
3. Once the cage is removed, the float with the red seal comes out of the jar cap.
4. **To disinfect the accessories, use either denatured alcohol or hypochlorate-based solutions, easily purchasable from chemist shops.**
5. Once cleaning has been completed, insert the float in the cage, with the seal facing towards the cage opening.
6. Insert the opening of the cage into the seat in the jar cap.
7. To facilitate the cleaning of the jar, fill it 1/3 full of ordinary water.
The aspirated liquid, thus diluted, will then be easier to remove.

It is possible to autoclave the accessories cover and jar: insert the parts in autoclave and to effect a cycle of sterilization having care to position turned upside-down the graduated jar (with turned fund toward the tall one).

After the sterilization and the cooling to the temperature the components' environment, to verify that this last don't result damaged; disassembled therefore the container for inhaled liquids following the inverse operations to the dismantlement.

The aspiration tubes can be sterilized on autoclave using a sterilization cycle at 120°C.

The conical connector can be sterilized on autoclave using a sterilization cycle at 121°C.

The device is ready for a new employment now.

During the sterilization operations the operator must use disposable gloves in latex, shirts and opportune mask not to enter contact with possible substances contaminating

DO NOT WASH, STERILIZE OR PUT IN AUTOCLAVE THE ANTIBACTERIAL FILTER

PERIODICAL MAINTENANCE CHECKS

The **TOBI MANUALE** suction equipment does not need maintenance or lubrication.
 It is necessary to check functioning and instrument before every use.
 Unpack the instrument and **always check** integrity of plastic parts, they might have been damaged during previous use.
 Close the aspirator outlet with your finger.
 Press the foot and check vacuum functionally.

Type of fault	Cause	Remedy
1. No aspiration	Jar Cap badly screwed down	Unscrewed the cap, then rescrew it correctly
2. No aspiration	Lid seal not in its seat	Unscrew the cap and insert the seal properly in its seat
3. The float doesn't close	If the cap has been washed, ensure that the float is not partially detached	Insert the float into its place
4. The float doesn't close	The float it's covered by dirty material	Unscrewed the cap, leave the and put in on autoclave
5. Low suction	Foam inside the jar	Fill the jar to 1/3 full of ordinary water
6. No aspiration due to flow leakage of mucus	Filter blocked	Replace filter
Faults 1 - 2 - 3 - 4 - 5 - 6	None of the remedies has achieved the desired results	Contact the seller or GIMA After-sales Assistance Service

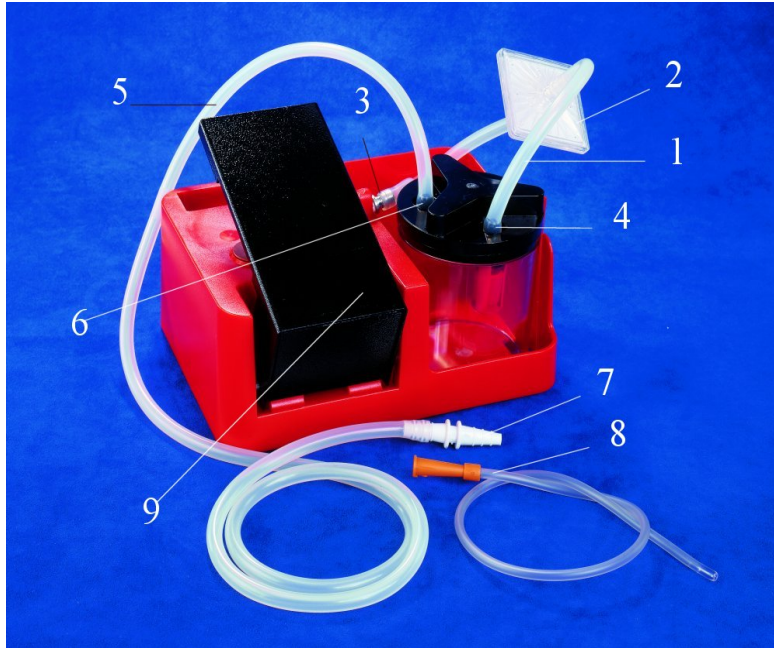
If the overfill security system it's activated, don't proceed with the liquid aspiration.
 If the overfill security system doesn't work there are two cases:
 1° case – If the overfill security system doesn't work the aspiration will be stopped by the bacteriological filter who avoid the liquid penetration inside the device.
 2° case – If both the security system doesn't work, there is the possibility that liquid comes inside the device, in this case return the device to GIMA S.p.A. technical service.

**BEFORE EVERY CHECKING OPERATION, IN CASE OF ANOMALIES OR BAD FUNCTIONING,
 PLEASE CONTACT GIMA S.p.A. TECHNICAL SERVICE.
 GIMA S.p.A. DOES NOT GIVE GUARANTEE IF INSTRUMENT, AFTER THE TECHNICAL
 SERVICE CHECKING, APPEARS TO BE TAMPERED.**

INSTRUCTION FOR USE

- Connect the short silicon tube **1**, with antibacterial filter **2**, to the suction connector **3**. The other tube, with one end connected to the filter must be connected with the other end to the jar's lid connector **4** where has been fixed the red float. When the 90% of the volume of the jar is reached there is the activation of the security float(the float close the aspiration connector on the jar) to avoid liquid penetration inside the device. The device must be used on a plan of horizontal operation.
- Connect the long silicon tube **5** to the other jar's lid connector **6**
- Connect the other end of the long silicon tube **5** to the probe plastic connector **7** then connect the suction probe **8** to it.
- Unscrew the jar's lid and fill the jar 1/3 full of ordinary water (this for an easy cleaning operations and an rapid reaching of the functionality vacuum) then rescrew the lid on the jar correctly.
- Push foot **9** to start suction.
- To extract the accessories and start with cleaning as write in the clean chapter

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