# GIMA

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## FEATURES

This stretcher was designed to make the transportation of patients inside or outside of regular surfaces or slightly inclined surfaces (max 5%) easier. In emergency cases the stretcher must be lifted and held by more than one operator.

A semi-automatic closing system on the supporting legs, helps load and unload the stretcher from properly equipped medical vehicles (ambulances).

For patient safety use the straps and the folding rails **F**.

The maximum load is 145 kg. calculated with the back lowered and the stretcher positioned on a flat surface.

The product, or parts of the product, must be used only for the purposes it has been designed for and specified in the instructions of use contained in this manual.

## WARNING

*The back and foldable side rails **F** and all the foldable parts are subject to risk of crushing; be very careful and make sure hands, feet and any other objects are not crushed by these parts. Non-compliance with these regulations could harm people or damage the equipment.*

*Do not use the product if damaged and contact the dealer.*

*Do not perform any temporary repairs. Use original spare parts only which must be installed according to their correct use.*

*Attention: check tubes, traverses and joints. If there are any burrs remove them with appropriate tools because they could hurt patients and operators during use.*

*Do not hit walls, steps or sidewalks because the stretcher could unexpectedly fold or turn over due to strong crashes.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

## UNPACKING

*Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.*

*They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).*

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

## OPERATION

*Pay attention to hinged or mobile elements, they may cause crushing.*

*Pay attention to the use of the unlocking handles on the supporting legs because they must be engaged when the stretcher is supported by the ambulance or another operator.*

Often when patients get off a stretcher they use it to support themselves during the operation. To avoid any unexpected movements of the stretcher, always engage the brakes **C** on the casters. Use the brakes when the patients is getting on or off of the stretcher and when it is left unattended.

Before moving the stretcher make sure the brakes are released. The brakes **C** must be tested periodically. If they do not work properly contact the dealer.

When the stretcher is being moved, always hold it tightly because it could move forward pushed by inertia or by the inclination of the surface, even if imperceptible. Do not make any sharp or quick turns or move down steep hills especially with the stretcher in horizontal position. In emergency cases, go up or down hills frontward with stretcher long-wise and hold onto it very tightly.

When the stretcher is left unattended, engage the brakes. To further protect the patient when left unattended, lift the side rails **F** and to make sure they are locked try folding them.

The side rails **F** must be used as a safety device and not to hold the patient onto the stretcher. Use the appropriate straps to fix the patient to the stretcher.

The back can be adjusted according to needs by using the lever **A** under the back and lifting it up to the desired height. Pay attention to risk of crushing when you bring the back in horizontal position.

The casters under the back help to load the stretcher on the ambulance. They must be adjusted according to the vehicle it has to be loaded onto. Perform this operation by fixing the pin in the hole **B** that matches the desired height.

The stretcher has a mechanical device to help load and unload it off of a vehicle.

Use the levers on the side of the operator (feet) to fold the legs and load: the left lever **D** unlocks the front legs. This is the first one to engage during loading operations, whereas the right lever **E** unlocks the back legs. When unloading the stretcher, make sure the legs are blocked when they are in upright position.

*Attention: use the right lever during loading/unloading operations. Do not engage them accidentally! Accidentally unlocking the legs could harm the patient or operator!*

## MAINTENANCE

Clean the stretcher without using abrasive products or solvents (we suggest using soap and water).

After washing the stretcher dry every part thoroughly.

To disinfect, use a combination of cleaning/disinfecting products.

Dilute the disinfectant following the manufacturer's instructions.

Periodically verify the condition of the mattress and the hinges or joints and plan repair/replacing interventions if necessary. Periodically control efficiency of brakes on the wheels. Extraordinary maintenance interventions must be carried out by authorized personnel only.

## GUARANTEE

Thank you for having acquired our product. This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered. Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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