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Instruction manual

Packaging and Components


Check all components after unpacking, please contact the supplier if they are missing or damaged.

Package list:

- Main part A: Toilet lift seat base 1 unit
- Handrail B (with control switch) 1 unit
- Handrail C 1 unit
- Power supply E (for charging the power supply) 1 unit
- Installation tool bag

PART	UNIT	USE
Fixing bolts	2	Repairing toilets and equipment
Screws	4	Repairing toilets and equipment
Screwdriver	1	Tool
Socket spanner	1	Tool
Quality certificate	1	Certificate and product information
Instructions for use	1	Instructions for use

Note: The installation positions (left or right) of handrail B and handrail C are optional, the right hand control is the default installation, the left hand control can be customized.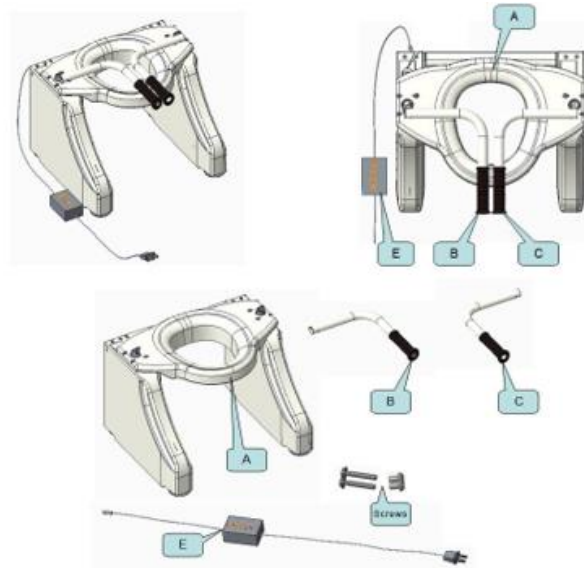

Using the AC adapter

1. The charger's indicator light will glow red to show that it is charging
2. The power display is located in the position shown in Figure 3.

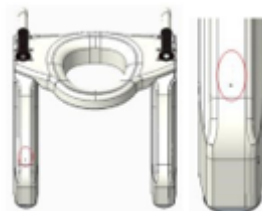
Figure 3 : Power display

Installing the handrail

See figure 4, screw the armrest base to the auxiliary lift base A.

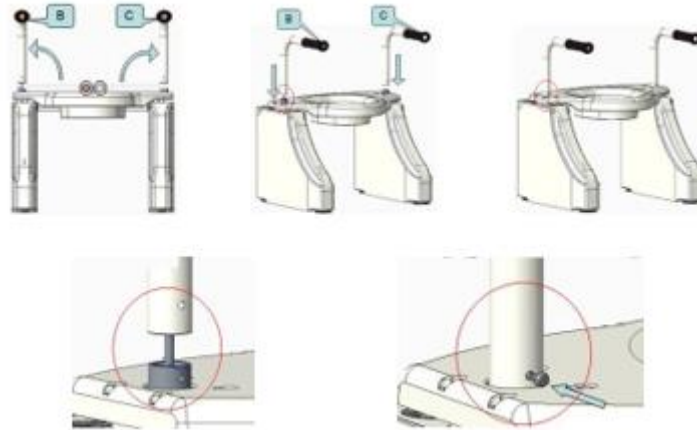
Figure 4 Installing the handrail

Note: Do not pull during the installation of the handrail B with the control switch circuit to avoid damage to the circuit.

Leg adjustment

4 legs (which can be replaced by 4 wheels with brakes) are set to the minimum height before delivery. Rotate the legs (figure 5) to change the height of the equipment, avoiding contact between the equipment and the toilet.

Figure 5 : Leg adjustment

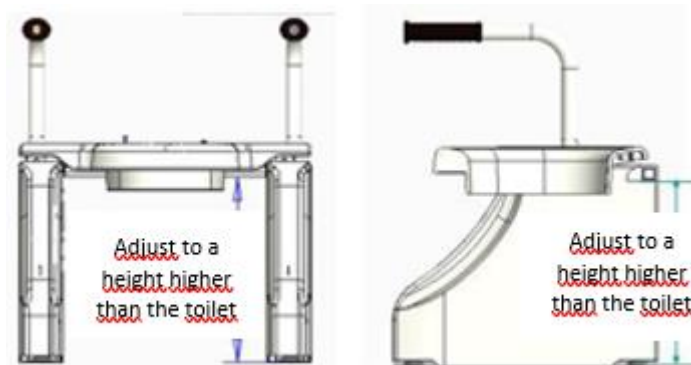
Figure 9 : Confirm the height of the frame

Caution: Adjust the 4 legs to make the equipment stable, and make sure that the lid surface is in a horizontal position. At the same time, make sure that the outermost lower edge of the seat of the equipment is higher than the upper edge of the ceramic base of the toilet. Otherwise, the equipment may function abnormally.

Fixing with the toilet

1. Remove the toilet cover and the fixing screws, only the ceramic base of the toilet remains.
2. Lift the unit (seat in its lowest state) and move it up the toilet base and lower it slowly to ensure that the unit's seat does not touch the toilet base.
3. Check the position of the machine on the toilet, to prevent the machine from colliding with the toilet during use.
4. For users with poor body balance, the crossbar of the main part A of the equipment should be fixed to the toilet with bolts (as shown in figure 7).

Fixing method 1: (directly fixed with bolts)

Figure 7.1 : The equipment is attached to the toilet with bolts

Fixing method 2: (For a special position of the toilet hole, first adjust the distance with an extension piece and then fix it).

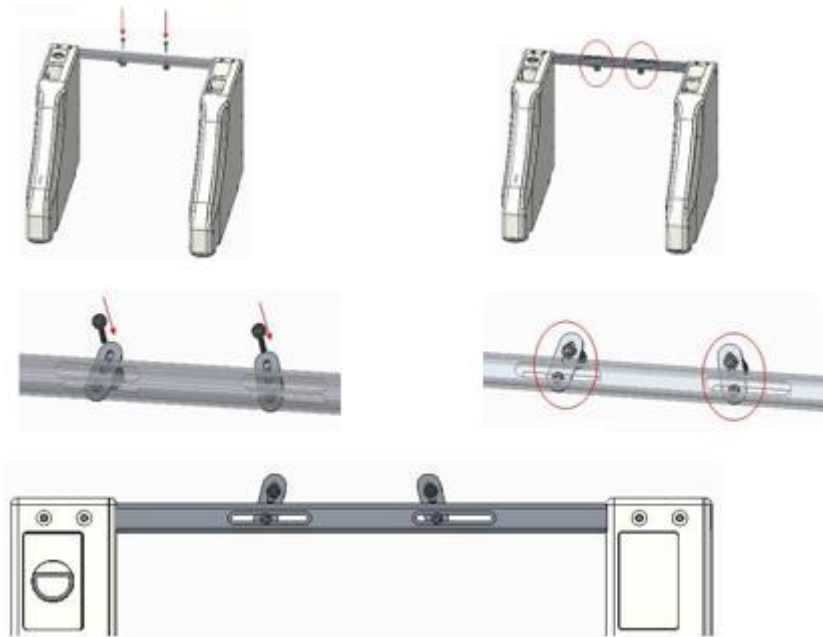
Figure 7.2 : Attach the equipment to the toilet after adjusting the distance with an extension piece.

Control Switch

The equipment switch is located on the front of the handrail B, users can continuously press the left and right buttons to control the raising and lowering of the equipment (as shown in Figure 8). The equipment stops operating when the switch is released.

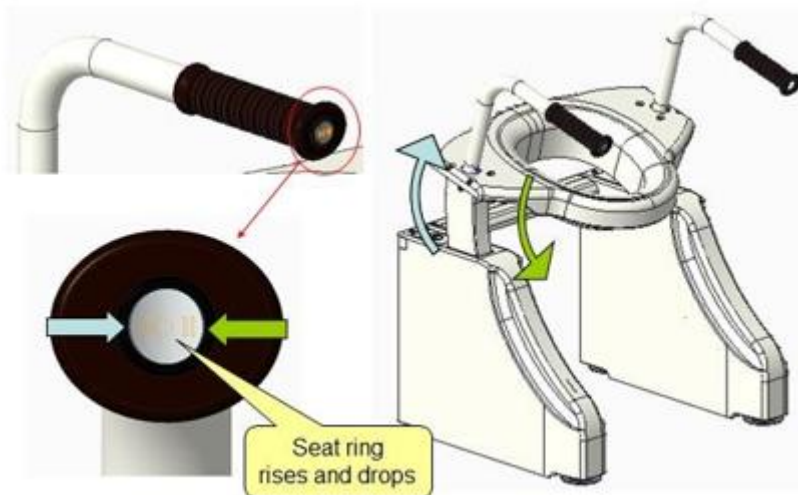
Figure 8 : Using the control switch

Product dimensions

The dimensions of the product are shown in figure 11

1. Width: 608 mm
2. Length: 525 mm
3. Height: 705mm
4. Distance from handrail: 505mm
5. Distance from the base of the machine: 425mm

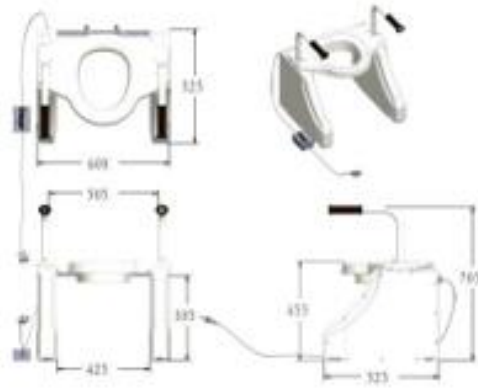
Figure 11 : Product dimensions

Technical specifications

Maximum weight: 200kg

Power supply: 85V~265V

Working power supply: 24V

Product power rating : 130w

Waterproof degree: IP44

Cleaning the equipment

Clean the equipment with mild, non-abrasive sanitary products, if the equipment is used by more than one person, the equipment should be disinfected each time. The equipment is not waterproof, do not clean it with a manual cleaning device, do not spray the equipment or immerse it in water.

Maintenance and service

4. This product is backed by a worldwide warranty, 1 year whole unit warranty, lifetime maintenance services
5. This device (excluding the battery) can be used more than 10,000 times in case the user uses and takes good care of the equipment according to the user's manual;
6. All the components of the device are not allowed to be repaired by the user himself, contact the authorized dealer for the revision.