**BRAS D'APPUI FIDJI SUR COLONNE
FIDJI FLOOR FIXED FOLDING SUPPORT RAIL
REF.815120**

NOTICE D'UTILISATION- USER MANUAL

AVERTISSEMENT DE SECURITE GENERALE – GENERAL SAFETY WARNING

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This product should only be fitted by a suitably qualified person. Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

UTILISATION PREVUE – INTENDED USE

The BRAS D'APPUI FIDJI SUR COLONNE is a high quality wall mounted steel rail which provides a stable support to assist the user raise and lower themselves for toileting. The rail can be raised back to the wall when not in use or when transferring from the side. It is not intended for use in a shower.

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Read these instructions before installation and use.

- This product must be correctly assembled, installed, and maintained.
- This product must be installed by a suitably qualified, competent, person.
- Never drill through pre-stressed concrete to install a folding support rail
- Always check for concealed electrical cables and water pipes when installing; also check your legal requirements relating to earth bonding of extraneous metal parts.
- Do not exceed the maximum load - Note this is based on the product being fitted to a suitable surface using appropriate fixings.

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1. Position the Wall Plate against the wall at the height required and mark the position of the holes.
2. Drill the holes, as appropriate for the fixings/ screws being used.
Note: As the substrate can vary considerably, we do not recommend any particular fixings; if in any doubt; consult a reputable fixing supplies company for advice.
3. Place Wall plate against wall and secure using the chosen fixings. When fitted, check that it is securely fixed to the wall and that the arm can be raised and lowered without interference. Also check that the friction lock is effective and that the support rail will hold itself at any angle during movement (i.e. it does not drop down on its own).
4. Important; After installing the product, perform a final tighten and inspection of all fasteners
5. Test the installation, ensure the rail is rigid and stable.

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Clean regularly, wiping down using a soft cloth and a mild detergent /disinfectant solution. Do not use abrasive cleaners. Do not autoclave. Always ensure that you wipe dry all equipment and treat joints with a water dispersing agent . Periodically check the stability and rigidity of the product and, where applicable, that screws are tightened and in good condition. If you see any damage or suspect a fault, do not use and contact your supplier.

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