Yes, you can.®

Invacare® **Octave**

User's Manual (GB)

CE

Product Certificate

Congratulations with your new bed **Invacare® Octave™** from Invacare® EC-Høng A/S.

Your new bed is **CE**-marked in accordance with directive 93/42/EEC of 27th of May 2003 concerning medical devices.

Furthermore, **Invacare® Octave™** is a class I product in accordance with directive no. 409 of 27th of May 2003 concerning medical devices.

Invacare® Octave™ has been developed and constructed with consideration for the user and others handling or assisting at the bed. **Invacare® Octave™** has been developed in accordance with the European Standard EN 1970.

Invacare® Octave™ has throughout the entire production process been supervised and quality controlled, and the finished bed has been inspected by our finished goods control. Identification label and QA-mark are located on the bed confirming that the finished goods control has approved the bed.

Please read the entire user's manual before using the bed.

Invacare® EC-Høng A/S is certified according to ISO 9001 and ISO 13485.

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Table of contents

User part	4
1. General information	4
2. Operation of Invacare® Octave	5
Technical information	8
3. Information	8
4. General Information	8
5. Receiving Invacare® Octave	9
6. Disassembly parts	13
7. Wiring	14
8. Part numbers for accessories	15
9. Service and maintenance	16
10. Maintenance chart	17
11. Trouble-shooting with the electrical system	18
12. Lubrication plan	18
13. Cleaning	19
14. Technical specifications	19
15. Electrical data	19
16. Disposal	20
17. Weights	20

User part

Congratulations on your choice of Invacare® EC-Høng's Octave nursing bed.

- This bed has been developed for users over 12 years of age requiring care in a nursing home or at home.
- The bed combines excellent stability and ergonomic design.
- A specially designed 4-sectioned mattress support provides the user with excellent comfort and provides optimal working conditions for the carer.

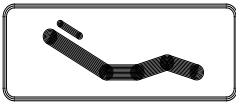
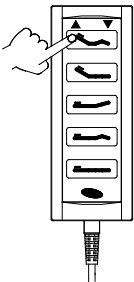
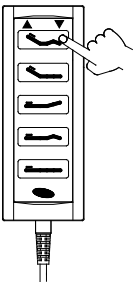
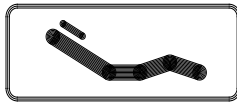
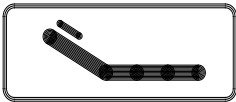
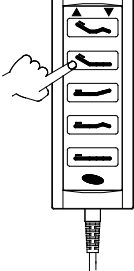
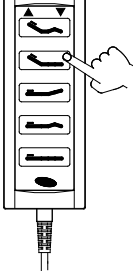
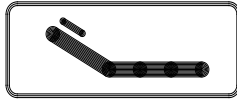
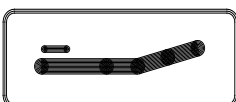
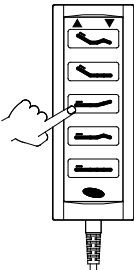
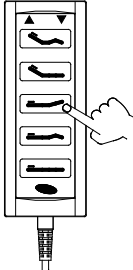
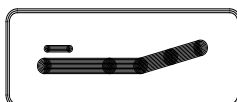
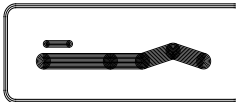
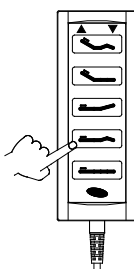
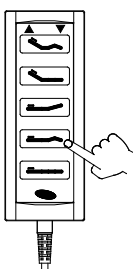
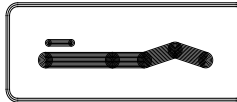
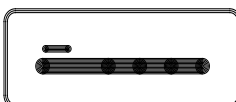
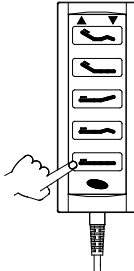
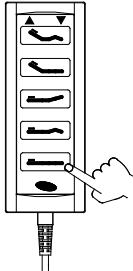
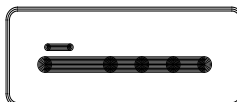
In order to optimise the patient's comfort Invacare® EC-Høng A/S recommends using a 15-cm mattress.
(see order numbers sec. 8).

I. General information

Invacare® EC-Høng A/S accepts no liability for any use, change or assembly of the product other than as stated in this User's Manual.

2. Operation of Invacare® Octave

Sitting position (green button) Up  Button with the symbol shown above. Push the left-hand side of the button.	 	Sitting position (green button) Down  Button with the symbol shown above. Push the right-hand side of the button.
Backrest up  Button with the symbol shown above. Push the left-hand side of the button.	 	Backrest down  Button with the symbol shown above. Push the right-hand side of the button.
Leg section up Electric  Button with the symbol shown above. Push the left-hand side of the button.	 	Leg section down Electric  Button with the symbol shown above. Push the right-hand side of the button.
Thigh section up  Button with the symbol shown above. Push the left-hand side of the button.	 	Thigh section down  Button with the symbol shown above. Push the left-hand side of the button.
Height up  Button with the symbol shown above. Push the left-hand side of the button.	 	Height down  Button with the symbol shown above. Push the left-hand side of the button.

There is a risk of entrapment whilst operating the bed.

Always leave the bed in the lowest position. Otherwise there is a risk of entrapment due to accidental lowering of the mattress support. A person can be seriously injured under the bed during height adjustment.

Operating castors with central braking system

Braking the bed: If the brake is in neutral, step on the red pedal.

Releasing the brake: Step on the green pedal, until the brake is in the neutral position.

The bed's castors must be locked whilst nursing a patient in bed or when using any positioning function. The bed must be locked when the patient moving in and out of the bed. The bed must not be transported when occupied.

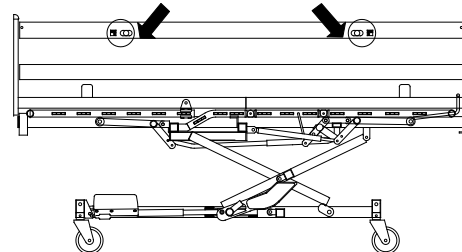
The castors might under special conditions be difficult to manoeuvre due to the floor covering being untreated or in poor condition. In matters of doubt, Invacare recommends the placement of a suitable type of protection between the castors and the floor.

Operation of the BRITT IV wooden siderail

UP: Pull up the top wooden bar, until the locking pin locks with an audible click.

DOWN: Lift the top wooden bar and press the two locking pins together. Lower the siderail.

Locking pins

Lift

Release

Lower

There is a risk of entrapment on assembly and operation of the siderails.

Britt IV wooden siderails is conform to European standards. For those with a small bodysize, there will however be a risk of entrapment between the lower wooden bar and the frame of the bed..

Adjusting the height of the lifting pole

Loosen the cord as shown in fig. A. The lifting handle can now be adjusted to the desired height. Press the cord together as shown in fig. B and check that the cord is locked in the cord lock by pulling the handle.

A

B

Emergency release of the mattress support parts

An emergency release of the mattress support is necessary in the case of e.g. a power or motor failure. An emergency release of the height adjustment is NOT possible!

- Remove the plug from the mains before emergency release of the mattress support.
- In an emergency case the mattress sections are released by pulling out the cotter pin from the motor in question.

A minimum of 2 people are required to release a mattress section. Both people should hold the mattress section in locked position. One person pulls out the cotter pin. Both slowly lower the mattress section until it is completely down.

Technical information

Invacare® EC-Høng A/S is certified according to DS/EN ISO 9001/ISO 13485 which ensures that our customers are always supplied with Invacare® EC-Høng A/S products of uniform quality.

Throughout the entire production process our materials/products are quality controlled by the operators. Furthermore there is a final test when the product is fully assembled.

The operator carries out the final test, which comprises a check of all moveable parts, motors/gas springs and castors, issues a QA number on the product thereby confirming its quality.

QA XXX

If the product does not correspond to the quality demands of Invacare® EC-Høng A/S, it will be discarded.

If, contrary to our expectations, a problem should arise in connection with the delivered product, please contact your Invacare® supplier.

3. Information

Approvals

- Invacare® Octave is -marked in accordance with Directive 93/42/EEC of 27 February 2002 concerning medical devices.
- Invacare® Octave has undergone a risk analysis according to NF EN 14971.
- Invacare® Octave is approved to carry a max. load of: 385 kg incl. equipment.

Max. patient weight: 350 kg (Only if the equipment - bedend. siderail, support handle, lifting pole and mattress - weight max. 35 kg).

Control units and motors are protected according to IP 66.

If an electronic part has IP 66 protection, the lock cam of the control unit **must** be used, otherwise Invacare® EC-Høng A/S provides no guarantee for the protection.

4. General Information

Invacare® EC-Høng A/S accepts no liability for any use, change or assembly of the product other than as stated in this User's Manual.

If the functions of the bed change, please send the bed immediately for a check-up according to the maintenance chart, chapter 10.

The Invacare® Octave must be stored in a room with a humidity equivalent to normal indoor conditions. The bed must be stored in a frost-free environment.

5. Receiving Invacare® Octave

Due to condensation, may the bed may not be used until it has gained room temperature.

Check whether the bed shows any signs of damage. If the bed is damaged, see terms of delivery.

The bed - mattress support and base frame - will be delivered assembled on a trolley.

By ordering a standard bed following parts will be delivered.

- Assembled bed on trolley.
- Fitting (2 pcs.) incl. mounting screws (8 pcs.).
- Wooden-bedends (2 pcs.).

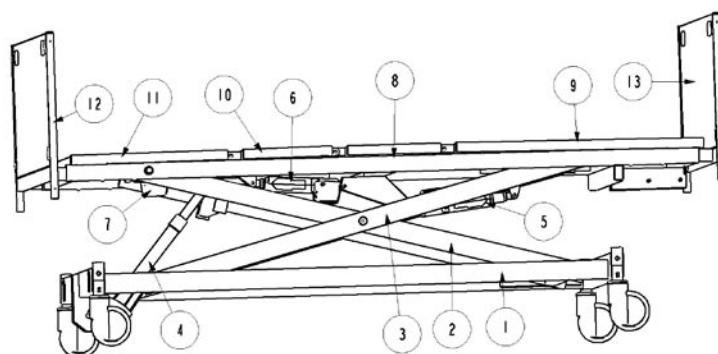
The following can be delivered as option: (see chapter 8 for order no.).

- BRITT IV wooden siderail (pair), only for L=200 cm (standard in the UK).
- Wooden bar (4 pcs.) incl. mounting screws (8 pcs.).
- Support handle.
- Lifting pole incl. mounting screws (2 pcs.).
- Mattress support extension 10 cm.
Can be used in head- and bedend.

Parts of a standard bed

1. Base with central braking system.
2. Inner shear arm.
3. Outer shear arm.
4. Motor, hi/low.
5. Motor for backrest.
6. Motor - thigh section (with control unit).
7. Motor - leg section
8. Top frame incl. seat section.
9. Backrest.
10. Thigh section
11. Leg section
12. Fitting
13. Wood - bed ends.

Mounting the Invacare® Octave.

Receiving the bed on the trolley: (2 persons required)

1. Apply all 4 braked castors on the trolley.
2. Apply all 4 braked castors on the bed.
3. Remove the delivered equipment from the trolley.
4. Lower the bed and the trolley, into horizontal position -standing on the bed castors.
5. Dismount the trolley from the bed.

When lowering the bed into horizontal position 2 persons are required.

Connect Invacare® Octave to the mains.

For ergonomic working position:

Raise the bed (hi/low) to approx. 60 cm above floor level. Ref. chapter 2 “Operation of the Invacare® Octave”.

1. Assembling the fittings.

- a) Push bed end through the pipes on the top frame.
(Visible insert pipe is about 4.5 cm).
- b) Tighten the bed end with enclosed screws.

2. Mounting the wooden bed end

- a) Slide the wooden bed end into the U-profile.

Mounting wooden adapters and wooden siderail (accessories).

1. Mounting wooden trim.

- a) Remove the exposure on the bed end.
- b) Mount the Wooden trim with the enclosed mounting screws.
- c) Make sure that all 4 Wooden trims have been correctly mounted/tightened.

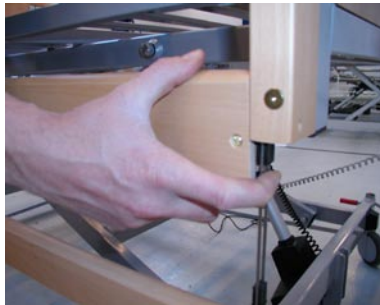
2. Mounting the Britt IV wooden siderail (accessories).

1. Raise the bed about one third.
2. Remove the protective tape from the locking pins at the ends of the siderail.
3. Mount one end at a time. Lift the lower wooden bar and guide the metal bow to the bed end rail (A).
4. Press the locking pin of the siderail with a finger (B).
5. Guide the siderail to the upper position. There will be an audible click when it's locks.
6. Mount the end stop brackets (C).

A

B

C

3. Mounting support handle (accessories)

- a) Mount the support handles on the tube of either the back- or leg sections.

Note! Do not mount the support handle on the seat- /thigh section.

- b) Tighten the thumbscrews.

Note! The distance between 2 support handle or support handle and bed end must be more than 25 cm.

Placing support handles in wrong positions can cause serious damage or suffocation, if body parts are trapped between support handles or between support handle and bed end.

4. Mounting the lifting pole (accessories).

- a) Push the lifting pole bottom piece in the fitting under the backrest.
- b) Mount the enclosed screws and nuts and tighten them.

5. Mounting the mattress support extension (10 cm).

- It is not possible to use the wooden siderail in combination with mattress support extension. Therefore dismount the wooden side rails first.

The mattress support extension can be mounted in either the head- or foot end.

- a) Loosen screws from the bed end.
- b) Remove the bed end.
- c) Remove the ferrules in the tube ends for the mattress support and mount the extension.
- d) Tighten up with the enclosed screws.
- e) Mount the bed end in the outer position (4 screws per fitting).
(Visible insertpipe is about 14.5 cm).

6. Disassembly parts

Dismounting mattress support - base

The mattress support can be disassembled from the base. To disassemble the mattress support from the base relevant tools and 2 persons.

- a) Dismount all accessories incl. bed end.
- b) Lower the bed into lowest position.
- c) Dismount the cable from the motor for height from the control unit.
- d) Raise the back section to the highest position.
- e) Loosen and remove the screws/nuts between mattress support and shear arms.
- f) Lift the mattress support free from the guideway.
- g) Lower the back section to the lowest position.
- h) Remove the power plug from the bed and lift the mattress support off the shear arm.

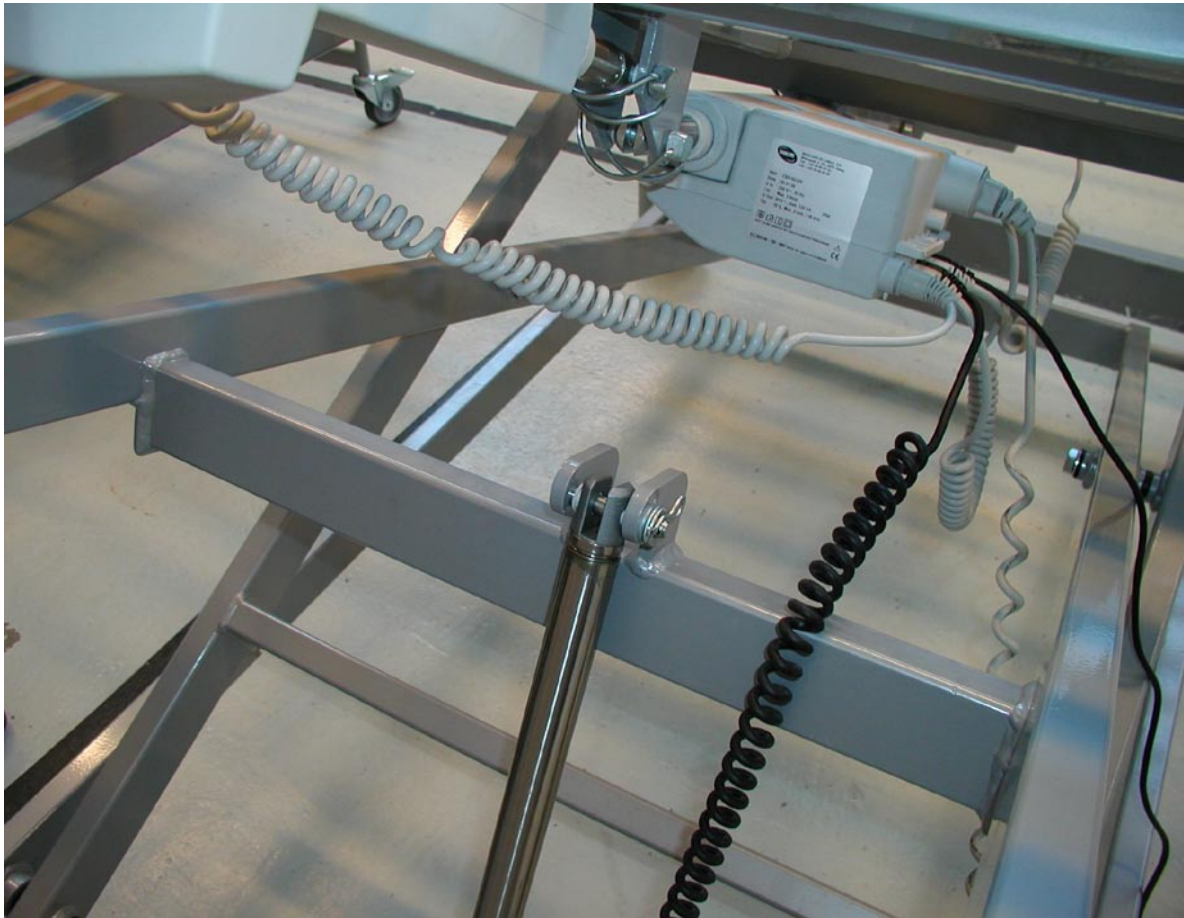
Dismounting the BRITT IV siderail

1. Raise the bed about one third.
2. Place the siderail in the highest position.
3. Dismount the end stop brackets. (4 pcs.).
4. Lower the siderail. (see "Operation of the BRITT IV wooden siderail" on page 6). Lower the siderail, until it is free of the bed end rail.

7. Wiring

Note! Mains cable is placed below the frame of the base.

The bed is not intended for use together with MEDICAL ELECTRICAL EQUIPMENT.
It can be dangerous to roll over the mains cable.
Do not bring mains cable into contact with moving parts.

Disconnect the plug from the mains supply before moving the bed.
The cables must be mounted in such a way that they are kept clear of the floor and do not block the castors.

8. Part numbers for accessories

Article	Dimensions	Order no.
Britt IV wooden siderail (Beech, complete pair)		I427875-0101
Britt IV wooden siderail (Cherry, complete pair)		I427875-0102
Wooden trim for siderail (Beech, set of 4)		I446500-0101
Wooden trim for siderail (Cherry, set of 4)		I446500-0102
Support handle (1 psc.)		I446353-0152
Lifting pole.		50.57700.M0
Mattress support extension 10 cm		I446498-0152
Mattress max. load 200 kg	I 16x205x15	I484429
Mattress max. load 350 kg	I 16x205x15	I451259
Mattress max. load 200 kg (extension)	I 16x10x15	I484640
Mattress max. load 350 kg (extension)	I 16x10x15	I451260

9. Service and maintenance

Service and maintenance of the Invacare® Octave must only be performed by personnel who have received the necessary instruction or training.

With normal daily use, service must be carried out according to the service schedule after 2 years use and thereafter every second year.

When moving the bed, service must be carried out according to the service schedule.

Note!

The top frame must be supported during service inspections to prevent accidental lowering.

For beds with electrical equipment a safety test comprising control of the motors' performance and mechanical state is recommended once a year.

Motors, hand control and control unit

These parts are not serviceable, they must be re-placed

10. Maintenance chart

Service and maintenance of the Invacare® Octave must only be performed by personnel who have received the necessary instruction or training. Bed-serial/Id. no.: _____					
Date:	Initials.:				
Circlips and cotter pins checked.					
Screws tightened.					
Weldings checked.					
Siderail locking and moving system tested.					
Castor fittings tightened.					
Castor brakes checked.					
Height adjustment motor checked.					
Backrest motor checked.					
Thigh section motor checked.					
Leg section motor checked.					
Wiring correct and undamaged. Plugs undamaged.					
Damaged paintwork repaired.					
Lubrication according to the lubrication plan (chapter 12).					
Accessories checked.					

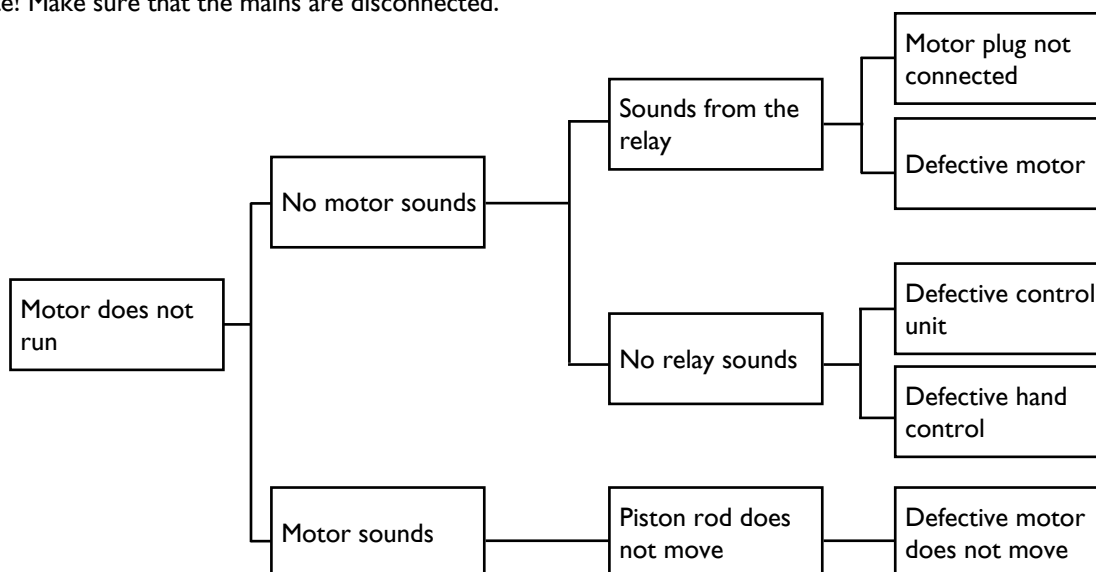
A service contract can be made in the countries, where Invacare® has its own sales company.

In certain countries Invacare® offers courses in service and maintenance of the Invacare® Octave.


Spare parts lists and additional user manuals are available from Invacare®.

I 1. Trouble-shooting with the electrical system

Note! Make sure that the mains are disconnected.

I 2. Lubrication plan

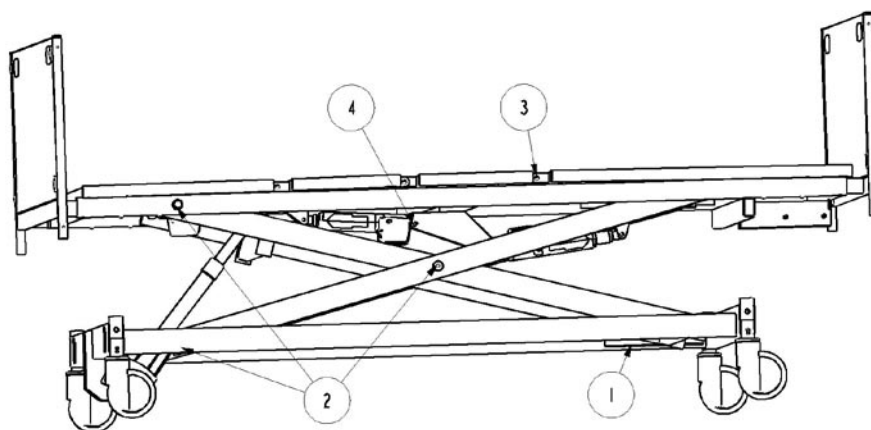We recommend to lubricate the Invacare® Octave according to the following instructions:

- | | |
|--|-------------------------|
| 1. Guideways | - lubricate with grease |
| 2. Swivel on shear arms | - lubricate with oil |
| 3. Swivel on mattress support -accessories | - lubricate with oil |
| 4. Swivel on motor | - lubricate with oil |

Lubricate with medically clean oil, e.g. KEW-WO 50, order no.: 813239


13. Cleaning

The Invacare® Octave cannot withstand cleaning in washing machines nor flushing with a water jet. Use ordinary household detergents. Dry the bed after cleaning. Never use acids, alkalines or solvents. Please make sure the power plug is removed from the socket.

Important details for cleaning hand control, control unit and motors

IP 66: Always clean with water and brush. Water may be pressurised, but never use high-pressure cleaning or steam cleaning directly on the units.

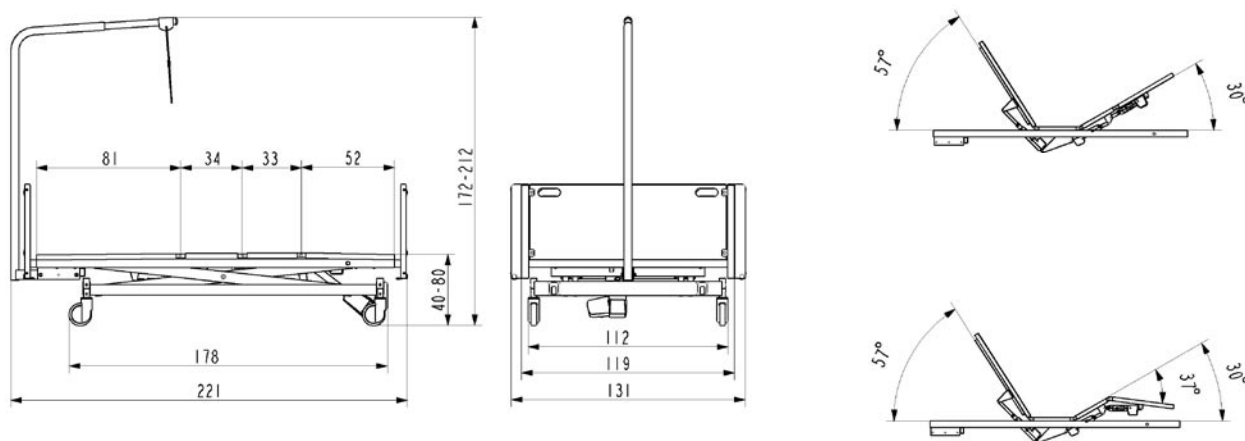14. Technical specifications

All measurements are stated in cm. All angles are stated in degrees. All measurements and angles are stated without tolerances.

Invacare® reserves the right to change the stated measurements and angles without warning.

Max. patient weight: 350 kg (provided that bed ends cot sides and accessories do not exceed 35kg)

Room temperature: 10° - 40°

15. Electrical data

Voltage supply: 230 V AC / 50 Hz.

Max. current input: 1 A.

Intermittent (Duty cycle of motors): 10% max. 6 min/h.

Protection class: IP 66.

Insulation class: II, type B.

Output: 70 VA

The patient is not separated from the ground and the chassis.

Double insulated.

The control unit has no isolator, disconnecting the plug is the only means of isolating the power supply.

Sound level: 48 dB (A).

16. Disposal

This product has been supplied from an environmentally aware manufacturer that complies with the Waste Electrical and Electronic Equipment (WEEE) Directive 2002/96/CE.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

The 'crossed out wheelie bin' symbol is placed on this product to encourage you to recycle wherever possible.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

17. Weights

Mattress support	72	kg
Base incl. shear arm	71	kg
Bed end fitting	5	kg/pcs.
Wooden bed end	8.5	kg/pcs.
Complete bed, excluding accessories	170	kg
Wooden trims 2 pcs.	2	kg
Wooden siderail (BRITT IV)	7.5	kg/pcs.
Lifting pole.	10	kg
Support handle	2	kg
Mattress	approx. 8	kg

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INVACARE Australia Pty Ltd
INVACARE EC-Høng A/S