GARANZIA

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WARRANTY

This aluminium walking stick has **2-YEAR** warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In the case you need to get your aluminium walking stick repaired and you fulfil the conditions of warranty, send the it to the nearest retailer together with the purchase receipt.

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MOPEDIA

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Acquistato in data / Purchasing date _____

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FOLDING WALKING STICKS INSTRUCTION MANUAL

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I Class Medical Device according to
EEC directive 93/42

Code RP145BC-NC-NL

Thank you for purchasing **MOPEDIA** by Moretti folding walking stick, manufactured to support people with movement difficulty. It consists of bronze or black coloured anodise-plated aluminium structure with possible folding in four parts, anatomic wooden handle, and non-slip rubber tip. The maximum endurable weight 130kg. Adjustable height each 2,5 cm:

- **RP145BC** from 77,5 to 87,5 cm
- **RP145NC** from 77,5 to 87,5 cm
- **RP145NL** from 82,5 to 92,5 cm

To adjust height:

Push the button to adjust the height. Draw out the stick paying attention to the corresponding holes up to the desired height which should reflect user's needs. Then leave the push button to fix the stick again.

ATTENTION: USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES.

Avoid to leave the walking stick in the contact with water for long periods and control its sturdiness periodically. Keep it away from children to avoid any possible risks. Replace rubber tip every time it is worn out.

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Moretti S.p.A.