ARDEA

CS930

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I Class Medical Device  
according to Directive 93/42/EEC and further modifications

## CS930 ELECTRIC WHEELCHAIR 6 WHEELS "VIRGO"

### 1. Introduction

Thank you for purchasing an ARDEA electric wheelchair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

**NOTE: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.**

### 2. Intended use

The electric wheelchairs are intended for people with movement difficulties

#### CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Do not drive if you are under the influence of alcohol or drugs
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

### 3. Ec declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the wheelchair category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

### 4. General warnings

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed wheelchair away from heat sources
- **SERVICE LIFE**- the wheelchair use limit is defined by the wear of the parts
- **ALWAYS** keep hands and fingers clear of moving parts to avoid injury
- **DO NOT** allow children to play on or operate the wheelchair

## 5. Electromagnetic compatibility warnings

The wheelchair operating can be influenced by electromagnetic interferences.

The interferences come from:

- Radio station
- Tv station
- Radio amateur transmitters
- Shop's alarm
- Mobile phone or cordless

The electromagnetic interference (from radio wave sources) can cause the motorized wheelchair to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the motorized wheelchair control system. There are a number of sources of relatively intense electromagnetic fields in the everyday environment. Some of these sources are obvious and easy to avoid. Others are not apparent and exposure is unavoidable. However, we believe that by following the warnings listed below, your risk to EMI will be minimized.

- A) Do not operate hand-held transceivers, such as citizens band radios, or turn ON personal communication devices, such as cellular phones, while the power chair is turned ON
- B) Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them
- C) If unintended movement or brake release occurs, turn the power chair OFF as soon as it is safe
- D) Be aware that adding accessories or components, or modifying the power chair, may make it more susceptible to EMI
- E) Report all incidents of unintended movement or brake release to Moretti Spa.  
Note whether there is a source of EMI nearby

## 6. General description


### 6.1 - Front view and description

- A) Padded back seat with adjustable headrest
- B) Width and height adjustable armrests
- C) Joystick
- D) Joystick adjustment knob
- E) Extended footplate
- F) Belt
- G) Engine unlocking levels


### 6.2 - Rear view and description

- A) Rear bag
- B) Armrest height adjustment knobs
- C) Rear reflectors

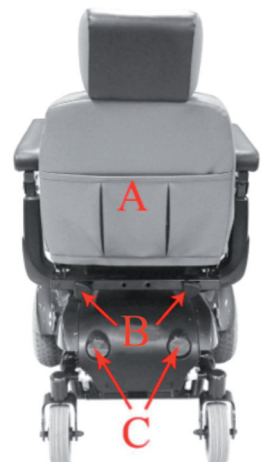### 6.3 - Joystick view and description

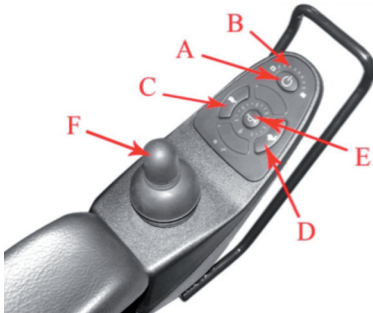- A) On/Off button
- B) Self diagnostic, warning lights and battery gauge
- C) Speed control button (accelerate)
- D) Speed control button (decelerate)
- E) Horn
- F) Joystick

### 7. Activation and connection

Inside the carton box you find the following parts:

- 1. Electric wheelchair
- 2. Back seat
- 3. Armrest (right and left)
- 4. Joystick and support

Follow these assembling instructions:

1. Assemble the seat with the wheelchair
2. Pull up the seat lock lever to let the seat being inserted correctly
3. Release the lock lever

4. Insert the left and right armrest in the supports and screw the knobs

5. Assemble the joystick and screw the knobs situated under the armrest. Adjust the length follow your necessity

6. Connect the cable to the joystick and lock it using the metallic hooks


**NOTE** - The cable is long enough to be put under the seat if exceed


7. Press the rivets on the battery cover with an acute object

8. Raise the cover up and remove it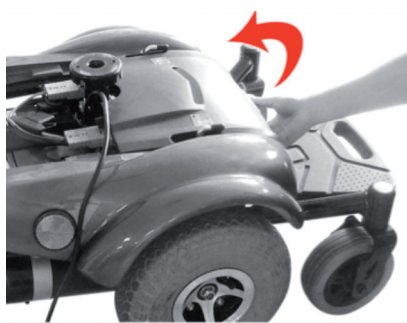

9. Plug in the batteries connections

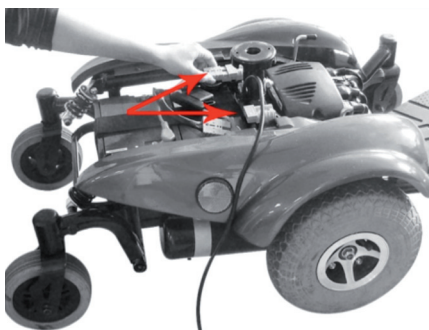10. Close the batteries with their covers

### 8. Before use

- Check the battery correctly works, otherwise recharge it
- Check the wheelchair parts are correctly assembled

- Always check the state of the parts to guarantee a safe use of the device.  
In particular pay attention and check the wheels

## 9. Using warnings

- Only a person a time can use the electric wheelchair
- The max weight cannot be higher than 136kg
- Always fasten the seatbelt
- Before get in or get off of the wheelchair turn off the device
- Always drive carefully, with feet on the footrest, and paying attention to the presence of people on your way
- If possible, use pedestrian streets. Pay attention in crossing roads
- Even though the wheelchair can go on gradient of 10° (17,5%), do not drive on gradient higher than 6° (12%) and pay attention to the bend
- Do not use max speed in bend
- Before drive the wheelchair check the parts are correctly assembled
- Do not drive in case of the rear wheels are damaged or have been disassembled
- Reduce the speed in reverse motion, downhill, dirt patch and in bend
- During downhill never set the engines in NEUTRAL (N) mode
- Always follow the rules of the road
- Do not use the wheelchair in wet grounds. The warranty does not cover damages caused by water

## 10. How to use

### 10.1 - Starting

#### - Adjust the joystick

1. Unscrew the knob as shown in figure
2. Adjust the joystick following your necessity
3. Lock the knob

**NOTE** - Pay attention to the cable during the adjustment

### - Adjust armrest width

1. Unscrew the knob shown in the picture
2. Adjust armrest support width
3. Screw the knob

### - Adjust armrest inclination

1. Lift the armrest
2. Unscrew the nut shown in the picture and adjust the inclination
3. Screw the nut
4. Pull the armrest down

### - Adjust headrest height

1. Push the button shown in the picture and adjust the height
2. Release the button at the desired height

## - Adjust the backrest inclination

1. Fold down the seat back
2. Unscrew the nut shown in the picture
3. Adjust the screw length and lock it with nut (operate on both side)

## - Rotate the seat

1. Pull the lever shown in the picture
2. Rotate the seat clockwise and anticlockwise as well
3. Release the lever at the desired position

**NOTE** - Do not rotate the seat more than 90° avoiding the cable crushing or deformation

## - Footrest height

1. Unscrew and remove the screws shown in figure
2. Adjust the footrest height
3. Insert the screws and tighten them

**NOTE** - If necessary the footrest can be closed upright

## 10.2 - Use the command

1. Push ON/OFF to turn on the joystick and the Led will light
2. Adjust the speed using the apposite button (Turtle- decrease the speed; Hare- increase the speed)
3. Honk if necessary

**NOTE** - Do not abuse, respect the rules of the road

4. Use the joystick to let the wheelchair move to the desired direction

## 10.3 Disassembling

1. Disconnect the cable from the command

2. Disconnect the cable from the armrest hooks

3. Fold down the seat back
4. Pull the lever which unlock the seat and extract it


5. Press the rivets on the battery cover with an acute object

6. Raise the cover up and remove it

7. Disconnect the batteries connections

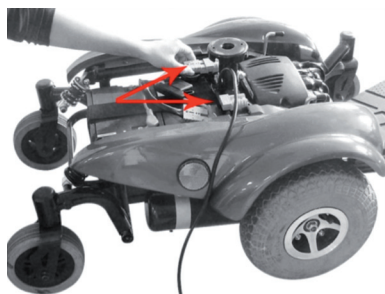8. Open the Velcro stripe

## 9. Remove the batteries

### 10.4 - Drive the wheelchair

Before seat on the wheelchair verify:

- The wheelchair is turned off
- The armrest and the footrest are correctly positioned
- The battery charger is unplugged from the wheelchair

When you seat on the wheelchair ensure the adjustments are suitable for your driving. In particular be sure the joystick do not strike your hand during the driving.

Start up:

- Set to LOW speed
- Gently push ON button
- Wait 2 seconds (this is automatically set up in order to prevent accidental start)
- Push the joystick ahead. The more you push the more the wheelchair accelerate.  
To stop release the joystick
- To turn left or right gently push the joystick to the desired direction
- Pull backward for the reverse motion

**WARNING!**

**IN CASE OF EMERGENCY RELEASE THE JOYSTICK TO STOP THE WHEELCHAIR**

### 10.5 - Drive uphill

The international law fixes the safety gradient to drive an electric wheelchair without difficulties. The slope is 6°/12%. The Moretti electric wheelchair can do up to the safety gradient. In case you are driving on the maximum slope you need the maximum speed, so use the proper button to increase the speed. Be careful in case of high slopes.

### 10.6 - Drive downhill

To proceed with caution in dangerous slopes use the proper button to decrease the speed, in order to have the best control on the wheelchair. Anyway the electric wheelchair is provided with an automatic braking system which operate whether you try to increase the speed. During the downhill we recommend to drive carefully, and never stop the wheelchair in dangerous slopes.

**10.7 - Cross obstacles**

Always put yourself in front of a hurdle, never sideways. You can clear the hurdle up till 4 cm. Do not try to clear hurdle higher than 4cm. The wheelchair is equipped with anti-tip system which can interfere with pavements climb. Be careful during the obstacles climbing.

**10.8 - Self-diagnosis**

If the led start to flash when you turn on the electric wheelchair, check the troubleshooting list on the manual, and follow the possible solutions.

**10.9 - Move the wheelchair manually**

If you need to move manually the wheelchair, turn off the controller and position the proper levers on N (NEUTRAL), move the wheelchair, and position the lever on D (DRIVE) again.

**10.10 - Recharge the battery**

The electric wheelchair uses two rechargeable batteries (12V/36Ah) and a 25v/5Ampere charger. The batteries must be recharged before using the wheelchair and after extended use. Ensure the wheelchair is turned off and the engine levers are positioned on D (DRIVE). Always check the battery indicator. We suggest to recharge the battery when the red led flash.

1. Connect the charger to the power point situated under the controller
2. Ensure the power selector is set up to 220V (or the voltage you may have)  
Connect the charger to a power point on the wall and turn on the switch

*NOTE* - Keep the battery charger away from heat sources

3. The battery charger is provided with two control lights:

- Red light shows the battery charger is operating
- The yellow light become green when the charge is complete

4. Before disconnecting the charger let the light become green

5. Turn off the battery charger, disconnect the cable from the wheelchair and from the power pack

6. If the green light flashes for more than 40 minutes, this shows a malfunction on the wheelchair

**Check as follow**

- The battery charger cables are correctly connected
- Check the fuse correctly works
- The wheelchair is turned off
- If the previous operations are correctly done, and the green light still flashes, contact your dealer

*NOTE* - The necessary time to charge the battery can vary from the beginning battery state.

Charge the battery more than the necessary does not damage it, and it can't be overcharged

Retain these rules:

- Charge the battery once a month or more if you use the electric wheelchair regularly  
In particular charge the battery after a 3km trip
- If you won't use the electric wheelchair for one month or more, ensure the battery is totally discharged. Before using again charge the battery completely
- The battery and the wheelchair will be at their best state when the battery will be charged almost 10 times
- The maximum time of charge is 10 hours. If the yellow light is not yet become green after this period, contact your dealer
- The battery charger can overheat during the working. The device is provided with an internal fan

Do not block up the fan

For safety reasons follow the following suggestions:

1. Do not use the battery charger if the cable is damaged
2. Do not use extension cable for the power pack to avoid flame or black out

In case you need to replace the battery contact your dealer. Follow the correct replacement process. Do not throw out in the environment after use.

**Do not use the inserting container for municipal waste. We recommend to dispose the battery in the appropriate disposal areas for recycling.**

## 11. Maintenance

The ARDEA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every year. In case of repair, use only original spare parts and accessories.

### 11.1 - Annual maintenance

We recommend to make a regular maintenance service, in order to prevent malfunction and future complications.

The annual maintenance service includes:

- Wheels and pneumatic check
- Battery, cables and connections check
- Controller check, following the user needs
- Frame, screws and bolts check
- Cover and upholstery check

### 11.2 - The ordinary maintenance includes

- Clean the controller and the joystick
- Clean the frame and the upholstery to avoid grime
- Clean the wheels
- Grease the wheels axles and the springs every 3 months, if necessary
- Check the wheels. The tread wear and tear should not be more than 1,5mm
- Recharge regularly the battery
- Check the correct functioning of the joystick, in particular ensure it comes back to the central position when released. For any doubts contact your dealer
- Check the cables covers for any damage or tear and wear. For any doubt contact you dealer

## 12. Cleaning and disinfection

- DO NOT immerse in water the wheelchair
- Use only a damp cloth. Pay attention to the controller

**NOTE** - For a better disinfection use a small amount of mild soap then rinse carefully. Avoid any liquids soak in the device

- Never use acids, alkalis or solvents such as acetone

## 13. Conditions of disposal

### 13.1 - General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the electric wheelchair in the appropriate disposal areas for recycling.

### TREATMENT OF BATTERIES (Directive 2002/96/EC)

At the end of its life, the product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment or at retailers that provide this service. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. To remark the need to dispose of electrical equipment separately, the products is marked with crossed mobile waste bin.

### DISPOSAL OF WASTE BATTERIES (Directive 2006/66/EC)

This symbol on the battery or on the packaging indicates that the battery provided with this product shall not be treated as household waste. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. The recycling of the materials will help to conserve natural resources. At the end of their life hand the batteries over to the applicable collection points for the recycling of waste batteries. For more detailed information about recycling of this product or battery, please contact your local Civic Office, your household waste disposal service or the shop where you purchased the product.

## 14. Spare parts/accessories

For spare parts and accessories refer to main catalogue

## 15. Troubleshooting

You can find here some suggestions to solve possible problems you may face with the electric wheelchair. On the controller there are 10 led. Activate the auto-diagnosis pushing ON button and the led lights depending on the malfunction

N. Of led Enlightened	Malfunction	Solution
1	Battery fault	Check the battery status and the connections. Recharge the batteries at least 8 hours
2	Left motor fault	Check the left motor, connections and cables
3	Short circuit on left motor	Contact your dealer
4	Faulty right motor	Check the right motor, connections and cables
5	Short circuit on right motor	Contact your dealer
6	External interference	Check whether the battery charger is connected or if there are electromagnetic interferences
7	Joystick fault or the position is not neutral	Release the joystick to neutral. Turn off then turn on the wheelchair
8	Controller fault	Check the connections and cables
9	Brake fault	Check the connections and cables
10	The controller voltage is greater than usual	Check the battery status and connections

### 15.1 - Other problems

- 1) The wheelchair doesn't work even though it is turned on
  - Check the battery status. The led should be turned on
  - Check the auto-diagnosis led, they should be always turned on
  - Check the cables are duly connected
  - Check the battery are correctly connected
  - If you don't find the solution in the previous suggestions, contact your dealer
- 2) If after a 14 hours recharge, the led light is not yet green, contact your dealer

The wheelchair is equipped with a safety drive system. In case of error the led starts flashing, specifying the kind of malfunction

### 16. Technical features

Technical features			
Total length	1080 mm	Max slope	6°/12%
Width	610 mm	Obstacle	400 mm
Height	Min. (1080 mm) Max. (1150) mm	Minimum radius of curve	620 mm
Front wheels	Ø 150 mm	Brake	Electromechanic
Central wheels	Ø 260 mm	Seat	500 mm
Rear wheels	Ø 150 mm	Motor	2 x420W, it goes up to 4900 per minute
Weight with battery	82 kg	Battery	2x 12V-36 Ah
Battery weight	24 kg	Battery charger	External 5A
Max speed	7 km/h	Battery life	40km (depending on the user weight)
Max loading weight	136 kg		
Distance from ground up	85 mm	System	Dynamic 50 A

## 17. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 17.1 Rechargeable batteries warranty

The original batteries and the spare batteries are covered by a 90 days warranty during a period of 6 months, with regards to manufacturing defeat or to what concern the law regulation. If charge batteries are unused for more than 3 consecutive months, the warranty is void. If exhausted batteries are unused for more than 3 consecutive days, the warranty is void.

### 17.2 Repairing

**Warranty repair:** If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

**Repair not covered by warranty:** A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

**Non-Defective device:** The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

### 17.3 Spare parts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

### 17.4 Exempt clauses

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**MORETTI****ARDEA****WARRANTY CERTIFICATE**

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

**MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)**MADE IN TAIWAN**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)