**MOPEDIA®**

MA RP700x\_RP701x\_RP703\_RP710\_RP713 02 A\_05-2016

**STAMPELLE  
MANUALE D'ISTRUZIONE**

---

**CRUTCHES  
INSTRUCTION MANUAL**

## INDICE

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I Class Medical Device according  
to Directive 93/42/EEC and further modifications

### 1. Codes

RP710	Shoulder crutches - Adult
RP713	Shoulder crutches - Teenagers
RP700B	Forearm crutches - Child
RP700G	Forearm crutches - Child
RP700R	Forearm crutches - Child
RP701A	Forearm crutches - Adult
RP701B	Forearm crutches - Adult
RP701R	Forearm crutches - Adult
RP701V	Forearm crutches - Adult
RP703	Forearm crutches - Folding
RP705A	Forearm crutches - With shock-absorber
RP706N	Forearm crutches - With soft handgrip

### 2. Introduction

Thank you for purchasing a **MOPEDIA** walking canes by Moretti S.p.A. This user manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

### 3. Intended use

The **MOPEDIA** by Moretti walking sticks are aids manipulated by one arm adapted to support and / or help the mobility of people with mobility problems.

#### CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

### 4. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the cane aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this aim Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. DO NOT use/install the devices in question for any other purpose than that one expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying the conformity with Directive 93/42/EEC to the proper authorities and further modifications

## 5. General warnings

- READ carefully the entire manual before using the device .
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- DO NOT allow children to play with or operate the device

## 6. General description

### 6.1 Shoulder crutches

Made of lightweight anodized aluminum. The axillary and the handgrip have an anatomically shaped anti-slip grip and are constructed of durable material to offer comfort and safety. Fitted with non-slip rubber tips, height adjustable every 2.5cm.

**RP710** From 112.5 to 140cm

**RP713** From 93 to 114 cm

### 6.2 Forearm crutches

Made of anodized aluminum with anatomic injection molded handgrip. The handgrip is in one piece for comfort and safety. Equipped with noise-ring on the sliding tube, red reflector on the handle, non-slip, rubber tips, the armcrutch is adjustable every 2.5cm.

**RP703** Folding model for an easy transport

**RP705A** Model with shock absorber

**RP706N** Model with soft handgrip for more comfort

**RP700x** From 65 to 81 cm

**RP70x** From 74 to 97 cm

**RP703** From 76 to 97 cm

### 6.3 To adjust armcrutch height

You need to adjust the crutches according to your height.

#### SHOULDER MODEL

Standing position with your palms resting on the handles. Once you have found the correct height, the crutch should arrive between 1 and 2 cm below the armpits. Handles must be in line with the upper part of the hips. To adjust the height of the handle, unscrew and remove the screw. Insert the screw into the desired hole and tighten securely.

#### FOREARM CRUTCH

Press the button and slide the tube to match the holes, bringing it to the desired height and maneuverability of the patient. Handgrips must be in line with the upper part of the hips. Replace the stopper on the hole to lock the tube and make sure that the button is fully into the hole

**Note: Once properly adjusted crutches, arms must be bent comfortably while you are in an upright position.**

## 7. Before use

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the device is assembled correctly

## 8. Warnings for use

- Use extra caution on wet and slippery surfaces
- DO NOT use with patients who exceed the weight indicated in the product label and in the technical features chapter of the present manual
- Avoid leaving the device in contact with water for long periods of time
- Check periodically its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks

## 9. How to use

Hold on crutches, carry them out and move the injured limb by shifting the weight on the handles of the crutches, swing your body forward and keep the foot healthy well-stacked and finally bring forward the healthy foot. The crutches should never be behind the patient.

### CLIMBING STAIRS

- Place the good leg on the step
- Place the crutches and finally the affected leg

### LAY DOWN THE STAIRS

- Place the crutches on the step below
- Get off the injured leg
- Finally, to support the healthy leg

## 10. Maintenance

The **MOPEDIA** devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

## 11. Cleaning and disinfection

### 11.1 Cleaning

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove more difficult dirt. Dry thoroughly before use. Never use water with temperature above 30°C. Never use jet water or steam washing machine.

### 11.2 Disinfection

if you need to disinfect the device use a common disinfectant cleaner.

**Note:** Never use acids, alkalis or solvents such as acetone

## 12. Conditions of disposal

### 12.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend disposing the device in the appropriate disposal areas for recycling.

**13. Technical features**

Codes	Color	Material	Height adjustment	Max weight
RP700B	BLUE	Anodized aluminum	65-81 cm	80 kg
RP700G	YELLOW	Anodized aluminum	65-81 cm	80 kg
RP700R	RED	Anodized aluminum	65-81 cm	80 kg
RP710		Anodized aluminum	112.5 - 140 cm	130 kg
RP713		Anodized aluminum	93 - 114 cm	130 kg
RP701A	SILVER	Anodized aluminum	74 - 97 cm	100 kg
RP701B	BLUE	Anodized aluminum	74 - 97 cm	100 kg
RP701N	BLACK	Anodized aluminum	74 - 97 cm	100 kg
RP701R	RED	Anodized aluminum	74 - 97 cm	100 kg
RP701V	GREEN	Anodized aluminum	74 - 97 cm	100 kg
RP703		Anodized aluminum	76 - 97 cm	100 kg
RP705A	BLACK	Anodized aluminum	74 - 97 cm	100 kg
RP706N	BLACK	Anodized aluminum	74 - 97 cm	100 kg

**13.1 Spare parts**

- RV7010** Rubber tips for armcrutch - grey color - 2pcs  
**RV7011** Rubber tips for armcrutch - black color - 2pcs  
**RV7020** Rubber tips for shoulder crutches - 2pcs  
**RV7050** Sponge shoulder rest for RP71x - 2pcs  
**RV7040** Sponge handle for RP71x - 2pcs

**14. Warranty**

Moretti products are guaranteed from material or manufacturing faults for a period of ( two ) 2

years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 14.1 Repairs

**Warranty repairs** In the event of a product Moretti being defective in material of manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

**Repair not covered by warranty** A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully covered by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

**Non-defective device** The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

### 14.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

### 14.3 Exempt clauses

Except as expressly specified in this warranty and in the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, in the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

WARRANTY CERTIFICATE

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

MADE IN P.R.C

Via Bruxelles, 3 - Melegnano  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)