MOPEDIA®

MA RP75x 03 A_07-2016

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CE - I Class medical device,
according Directive 93/42/EEC and subsequent amendments

1. CODES

- RP755S** - Painted steel demountable shoulder walkers
- RP755F** - Painted steel demountable shoulder walkers with brakes
- RP756S** - Painted steel foldable shoulder walkers
- RP754S** - Painted steel foldable shoulder walkers, legs with adjustable height
- RP756A** - Painted steel foldable walkers with antibrachial support
- RP754A** - Painted steel foldable walkers with antibrachial support, legs with adjustable height
- RP755T** - Painted steel demountable walkers with antibrachial table
- RP756T** - Painted steel foldable walkers with antibrachial table
- RP758S** - Painted steel foldable shoulder walkers, legs with adjustable height

2. INTRODUCTION

Thank you for choosing a walker by MOPEDIA range, manufactured by MORETTI S.p.A. design and quality are a guarantee of comfort, safety and reliability. MOPEDIA its walkers, designed and built to meet all your demands for a practical, correct use. This user manual provides you some suggestions as to how correctly use the walkers you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the walkers. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The MOPEDIA walkers by MORETTI S.p.A. are devices to be used with two arms and are intended for people with movement difficulties.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserve the right to change the information contained in this document without previous notice.

4. DECLARATION OF EC CONFORMITY

MORETTI S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI S.p.A. and belonging to the WALKERS category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC. To this aim MORETTI S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the mentioned directive.
2. The complete list of the devices in question is available in annex A to the declaration.
3. The devices in question are **NOT MEASUREMENT** tools.
4. The devices in question are **NOT INTENDED** for clinical investigation.
5. The devices in question are sold in **NON STERILE** packing.
6. The devices in question should be considered as belonging to class I.
7. Do not use/install the devices in question for any other purpose than that one expressly indicated by MORETTI S.p.A.
8. Moretti S.p.A. maintains and submits the technical documentation certifying the conformity of the products in question with the directive 93/42/EEC to the proper authorities.

5. GENERAL WARNINGS

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual, checking with a particular attention at the warnings before use;
- For proper use of the device consult your doctor or therapist;
- Keep the packaged product away from any source of heat because the packaging is made of cardboard;
- The useful life of the accessory is determined by wear of parts which cannot be repaired and/or replaced;
- Be careful when children are nearby;
- Do not exceed the maximum payload.

6. GENERAL DESCRIPTION

The folding or demountable, shoulder or antibrachial walkers MOPEDIA, are made to support people with movement difficulties. Constructed with chromed and painted steel, with 4 castors 100mm, 2 with brakes, adjustable height, adjustable support in width and depth *(See model). The adjustments for height and width allow the use of the product by persons of small, medium or big size. The walker can be used with a padded seat *(RP757 Optional). Available with underarm crutches or antibrachial support.

These devices facilitate the movement of people with limited static equilibrium and/or dynamic, increasing the amplitude to the supporting base. They provide support in situations of weakness and/or paralysis of the lower limbs. They help to download on the arms part of the person weight in case it is required a limitation of the load on one or both lower limbs.

6.1 To adjust height (RP754S-RP754A- RP756T)

To adjust the height, press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished. Repeat described procedure for each leg, at the end, check all four legs have the same height. Maximum endurable weight 130kg.

6.2 Shoulder walkers Demountable frame

RP755S

RP755F

Foldable frame

RP756S

Height adjustable
RP754S

Height adjustable
RP758S

6.3 Antibrachial walkers

RP756S

Height adjustable

RP754S

Height adjustable
RP758S

6.4 Antibrachial table walkers

Demountable frame

RP755T

Foldable frame

RP756T

7. ASSEMBLY INSTRUCTIONS

7.1 Shoulder and antibrachial walkers

1. Start assembling your walker with the pole of the front frame which has to be inserted in the tubes of the lateral frame and fixed with two brass buttons.
2. *Foldable - Simply , open the frame. **RP756S - RP754S - RP754A - RP756A**

*Demountable - Use the two horizontal poles to install in the back frame to strengthen the whole walker. Fix the poles with the four screw-buttons. **RP755S - RP755F**

3. Adjust the rests to the patient's desired height and fix them with the two lock stems.
4. It is easy to mount and dismount the seat which is hung by means of hooks on the poles of the lateral frame
***(RP757 optional)**

7.2 Walkers - RP758S

1. Open the frame as in pic. (1.) Pay attention that safety buttons correctly block the sides of the walker, as in pic. (2 and 3.) Then, insert safety clips in the holes in the rear part of the walker as in pic. (4.)

2. Connect legs with wheels to the frame (legs with brakes must be on the back). Safety buttons have to click when they are blocked.

NOTE Legs have to be all at the same height. Pic. (5)

3. Place the handles in the tubes and block them with lever. Pic. (6)


NOTE If levers can not tighten enough the tube, tighten the screw nut.

4. Insert shoulder support tubes from the longer side, in the holes pic. (7) and block them with levers.
5. Insert the shoulder supports in the tubes. Pic. (8)
6. Adjust the height of the walker following the indication of the patient. The seat can easily be installed in the walker thanks to 4 hooks.
***(RP757 optional)**



How to fold the walker:

1. Remove the seat.
2. Remove safety clips
3. Press the safety buttons in the front and in the rear part as in pic. (9-10) and fold the four parts inward. Pic. (11-12)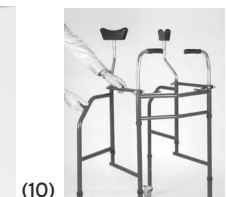

7.3 Antibrachial table walkers RP755T - RP756T

1. Start assembling your walker with the pole of the back frame which has to be inserted in the tubes of the lateral frame and fixed with two brass buttons.

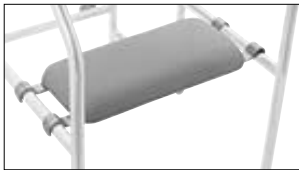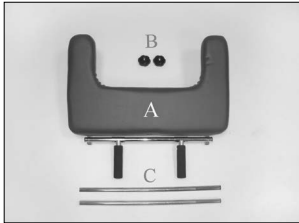2. *Foldable - Simply , open the frame. **RP756T**

*Demountable - Use the two horizontal poles to install in the back frame to strengthen the whole walker. Fix the poles with the four screw-buttons. **RP755T**

3. To assembly the padded armrest, insert the poles (ref.C) into the support tubes, once they are at the same height block them with the levers on the walker.

4. Place the padded table (ref.A) and adjust the height, in order the user can rest his arms forming a 90° angle.

5. Adjust the handles at the desired opening and block them with the screw-buttons supplied with the padded table.

6. Assure that the screws are duly tighten.

7. The padded table is now correctly assembled and ready to use.

8. The seat can be easily assembled or removed thanks to the hooks on the poles of the lateral frame. *(RP757 optional)

8. BEFORE USE

WARNING!

- Always check the good condition of the product
- Pay attention on the presence of electrical cables in way, wet surfaces, rugs, toys or debris, furniture, little steps and gaps in floors. Do not use the walker on stairs or escalators
- USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES
- Avoid to leave the walker in contact with water for long periods and control its sturdiness periodically.
- Keep it away from children to avoid any possible risks.

9. WARNINGS FOR USE

This device provides greater stability, support and safety while moving. In any case, it cannot be used by people with a functionality of the hand and / or arm visibly weak and even by people with balance problems. The walker can be used both outside and inside. We recommend using only on surfaces on which all four legs touch the ground. It should not be used on rocky terrain or on inclined surfaces because the walker can become unstable. The device is not designed for the transport of persons or other material.

- Use extra caution on wet and slippery surfaces.
- Do not use with patient who exceed the weight indicated in the product label and in the technical features chapter of the present manual.
- Avoid to leave the device in contact with water for long periods of time.
- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out.
- In case of doubts about use, adjustment or maintenance, contact your dealer.
- Keep it away from children to avoid any possible risks.

10. HOW TO USE

The walker is a personal product, so you may want to adjust the height of the walker by specialized personnel.

NOTE: Refer to the safety instructions in this manual. Use the walker as aid device for walking. To do this use the two handles and push forward the aid thus obtaining a support convenient and reliable for walking.

**WARNING!**

Pay attention on the presence of electrical cables in way, wet surfaces, rugs, toys or debris, furniture, little steps and gaps in floors. Do not use the walker on stairs or escalators.

11. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

- It is extremely important to check the walkers before each use. More specifically, please check that all screws and bolts are well tightened.
- Please, check the walkers can move freely with the wheels unlocked by the brakes.
- Check the rear wheel brakes function correctly.

12. CLEANING AND DISINFECTION**12.1 Cleaning**

To clean the aid and the structure use a damp cloth with solution of alcohol or neutral soap (in equal parts) mixed with water.

**WARNING!**

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacturer cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

12.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

13. CONDITIONS OF DISPOSAL 

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

14. SPARE PARTS ACCESSORIES

For spare parts and accessories, please always refer to the MORETTI master catalogue.

**WARNING!**

Only use original moretti accessories. Moretti S.p.A. Company shall not be liable for any incident caused by improper use.

14.1 Accessories



- **RP757** Padded seat
- **RV7115** Barre d'appesantimento - paio

15. TECHNICAL FEATURES**15.1 Technical specifications**



- Chrome plated and powder coated steel frame
- Castors with brake and block rotational
- Ergonomic PU armrest
- Maximum loading of 130Kg

15.2 Dimension


Shoulder walkers - Demountable frame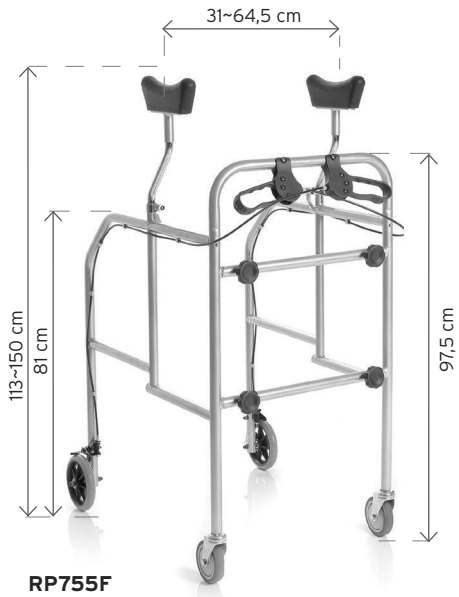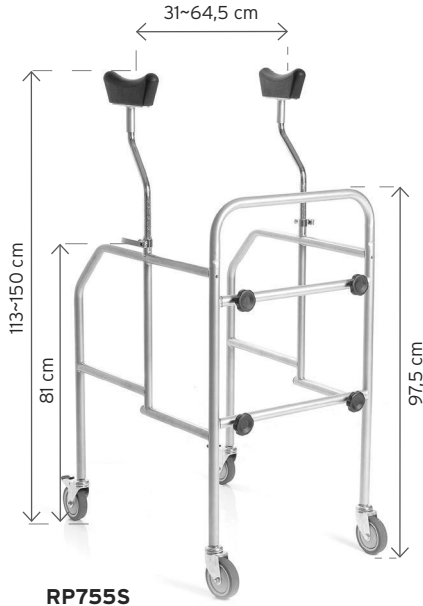


Shoulder walkers - Foldable frame

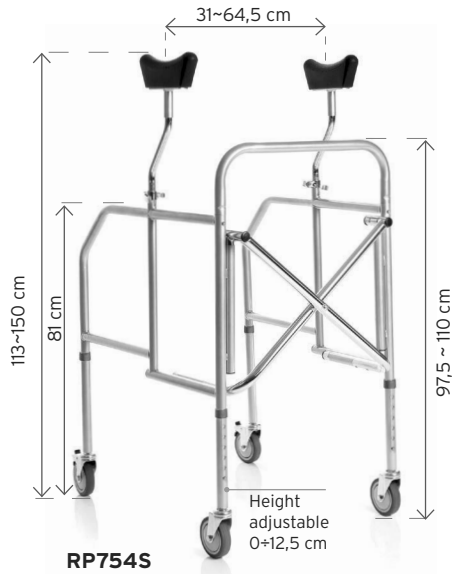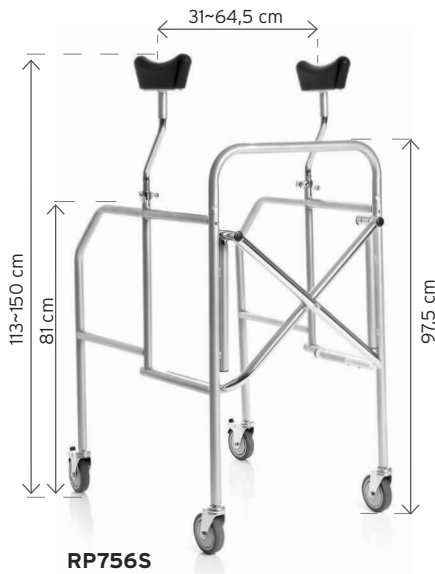
Shoulder walkers - Foldable frame

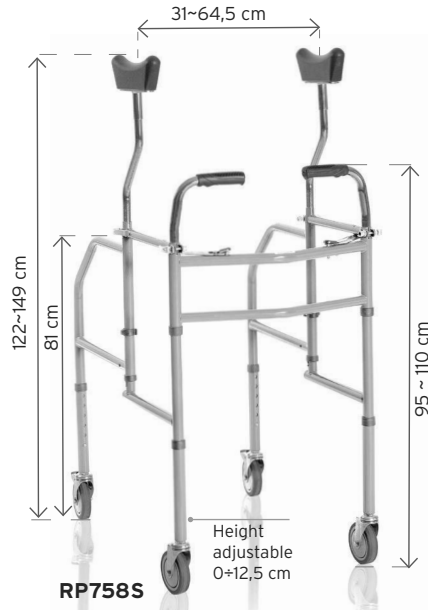Antibrachial walkers Foldable frame

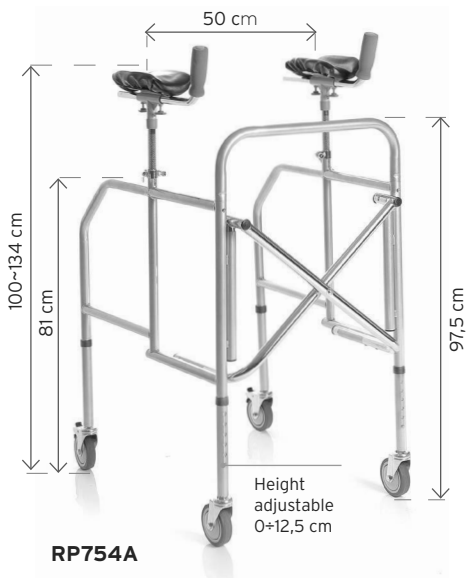
Antibrachial table walkers Demountable frame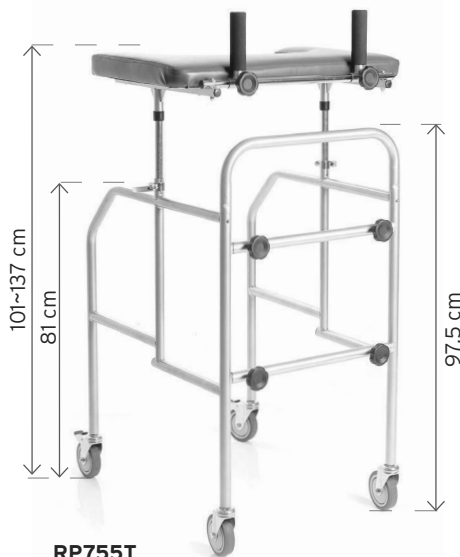

RP755T

Foldable frame

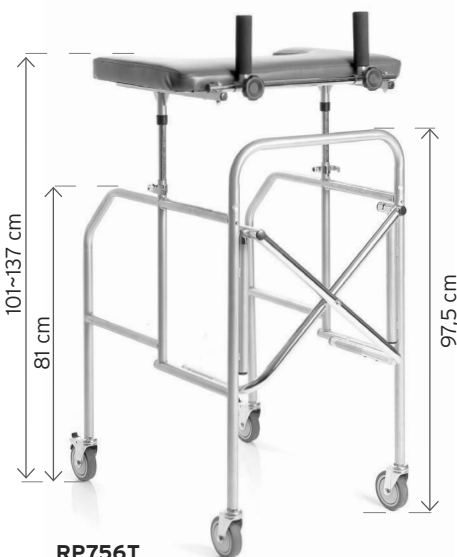RP756T

16. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

16.1 Warranty for rechargeable battery (if provided)

The original batteries and the spare batteries are covered by a (ninety) 90-days warranty as far as their performance is concerned and by a (six) 6-months warranty as far as manufacturing faults are concerned or as required by law. Warranty on fully charged batteries left in an unused state for more than (three) 3 consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than (three) 3 consecutive days is automatically void.

17. REPAIRING

17.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A

reparation or replacement does not renew nor extend the warranty.

17.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

17.3 Non defective device

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

18. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

19. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com