GARANZIA

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WARRANTY

The product has 2-YEAR warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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MOPEDIA

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ACQUISTATO IN DATA / PURCHASING DATE _____

RIVENDITORE / AUTHORIZED DEALER _____

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VENDUTO A / SOLD TO _____

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MORETTI S.P.A.

INTERNET: www.morettispa.com E-MAIL: info@morettispa.com
Made in Taiwan

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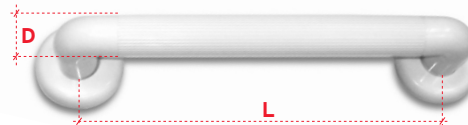
RS972/45

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RS972/60

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Class Medical Device according to EEC directive 93/42 & subsequent amendments

Codes

RS970/30	Ø 32 mm length 30 cm
RS970/45	Ø 32 mm length 45 cm
RS970/60	Ø 32 mm length 60 cm
RS972/30	Ø 36 mm length 30 cm
RS972/45	Ø 36 mm length 45 cm
RS972/60	Ø 36 mm length 60 cm

Introduction

Thank you for choosing a MOPEDIA plastic safety bath handles from the MOPEDIA by MORETTI S.p.A. product range. The design and quality of these products is a guarantee of comfort, safety and reliability. MOPEDIA by MORETTI S.p.A. plastic safety bath handles are designed and built to meet all your demands for practical, correct use. This user's manual provides some suggestions as to how to correctly use the bath handles you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the plastic safety bath handles. Should you have any queries, please contact your retailer for advice and assistance.

Intended use

The MOPEDIA safety bath handle by Moretti is a bathroom aid for people with motor difficulty.

EC Declaration of conformity

MORETTI S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI S.p.A. and belonging to the PLASTIC SAFETY BATH HANDLES category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC. To this end MORETTI S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of

directive 93/42/EEC as prescribed by Annex VII of the said directive.

2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I.
6. DO NOT use/install the devices in question for any other purpose than that expressly indicated by MORETTI S.p.A.
7. MORETTI S.p.A maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities.

Assembling

- Choose a solid and safe wall to install the handle.
- Make sure that it is possible to drill the wall safely for people and things.
- Make sure to position the handle bar at the most suitable height for users.
- Lift the flange cover to show the hole screw.
- Offer the handle up to the wall in the desired position and mark the screw hole position.
- Drill the wall and push correct sized plugs for the screw you are using into the wall.
- Offer the handle up to the wall, and drive the screws (we recommend using stainless steel countersunk screws 5x50mm).
- Tighten well the screws.
- Finally, clip the screw cover over the flanges. This will give to your handle an elegant appearance and hide the screw head.

NOTE STAINLESS STEEL SCREWS ARE RECOMMENDED TO REDUCE THE RISK OF RUST

**Warning**

For a correct use please make reference to the following manual.
Pay attention to the presence of children.

Directions for use

The safety bath handle if horizontally placed becomes an efficient foothold for movement such as sit or stand up from wc.

Caution! DON'T BEAR DOWN ON THE HANDLE WITH WHOLE BODY WEIGHT, JUST USE THE HANDLE AS SUPPORT FOR MOVEMENT.

Maintenance

The MOPEDIA by Moretti devices are carefully checked before the marketing and provided of CE mark. Before every use make sure the bath handle is correctly fixed to the wall.

Cleaning

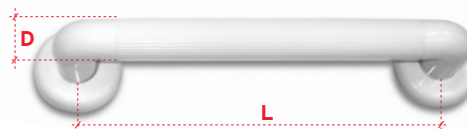
We recommend to use only warm water with mild soap.

General warnings

Do not use the product for a purpose not indicated in this manual. Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer can modify the device or the instruction manual without prior notification.

Technical features

MATERIAL: the grab is composed by extruded PVC and strong ABS injection moulded plastic for the end parts.

Dimension

D = diameter

L = length

RS970/30

diameter 32 mm, length 30cm

RS970/45

diameter 32 mm, length 45cm

RS970/60

diameter 32 mm, length 60cm

RS972/30

diameter 36 mm, length 30cm

RS972/45

diameter 36 mm, length 45cm

RS972/60

diameter 36 mm, length 60cm