GARANZIA

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WARRANTY

This rigid adjustable cervical collar has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your rigid adjustable cervical collar repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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**RIGID ADJUSTABLE CERVICAL COLLAR
WITH & WITHOUT CHIN
INSTRUCTION MANUAL**

Moretti S.p.A.
Via Bruxelles 3 - Melegnano
52022 Cavriglia (Arezzo)
Telefono +39 055 96 21 11

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I Class Medical Device
according to EEC directive 93/42

Codes RP180* (without chin)
Codes RP181* (with chin)

Thank you for purchasing the rigid adjustable cervical collar **MOPEDIA** by MORETTI, recommended for osteoarthritis, whiplash injuries, neck strains and strains by restricting movement of the cervical vertebrae.

Realised in polyethylene with foam pad on top bottom edge and covered with vinyl for patient comfort.

Soft, completely height-adjustable to provide flexion or extension positioning. Velcro fastening for a better patient fitting. Perforated for air circulation and cooling.

Composition

78% POLYETHYLENE - 22% EXPANDED POLYURETHANE, vinyl leather-covered foam.

Available in 4 different sizes:

S (7,5 H X 51 L)
M (9,5 H X 55 L)
L (10 H X 58,5 L)
XL (10,5 H X 62,5 L)

Use

Put the collar between chin and chest breast. Close the collar adjusting the velcro ensuring the neck is completely stopped.

Washing

Hand Wash: Garment may be laundered through the use of water.

Detergent or soap and gentle hand manipulation. Max. temperature 40°C.

Do Not bleach: No bleach product may be used.

Do Not iron.

Dry Flat: Lay out horizontally for drying.