GARANZIA

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WARRANTY

This arm sling with immobilizer has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your cotton arm sling repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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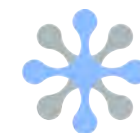
**ARM SLING WITH IMMOBILIZER
INSTRUCTION MANUAL**

Moretti S.p.A.
Via Bruxelles 3 - Melegnano
52022 Cavriglia (Arezzo)
Telefono +39 055 96 21 11

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I class medical device
according to ECC directive 93/42

Code RP175

Thank you for purchasing the arm sling **MOPEDIA** by Moretti, suitable to be used with plaster or after sprain. Realised in cotton with reinforced border, adjustable regulation system through metal buckle.

Available in one size.

Use

Put your arm inside the cotton sling, adjust the regulation to immobilize the arm.

Washing

 Hand Wash: Garment may be laundered through the use of water.

 Detergent or soap and gentle hand manipulation. Max temperature 40°C.

 Do not bleach: no bleach product may be used.

 Do not iron.

 Dry flat: lay out horizontally for drying.