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www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

I Class Medical Device according
to directive 93/42/EEC and subsequent amendments

CODES

ST850/1 1-section air cells cushion 36x36 cm h 6 cm
ST850/2 2-section air cells cushion 36x36 cm h 6 cm

ST851/1 1-section air cells cushion 40x40 cm h 6 cm
ST851/2 2-section air cells cushion 40x40 cm h 6 cm

ST852/1 1-section air cells cushion 46x40 cm h 6 cm
ST852/2 2-section air cells cushion 46x40 cm h 6 cm

ST853/1 1-section air cells cushion 50x46 cm h 6 cm
ST853/2 2-section air cells cushion 50x46 cm h 6 cm

ST855/1 1-section air cells cushion 36x36 cm h 10 cm
ST855/2 2-section air cells cushion 36x36 cm h 10 cm

ST856/1 1-section air cells cushion 40x40 cm h 10 cm
ST856/2 2-section air cells cushion 40x40 cm h 10 cm

ST857/1 1-section air cells cushion 46x40 cm h 10 cm
ST857/2 2-section air cells cushion 46x40 cm h 10 cm

ST858/1 1-section air cells cushion 50x46 cm h 10 cm
ST858/2 2-section air cells cushion 50x46 cm h 10 cm

INTRODUCTION

Thank you for purchasing the air cells cushion from the **LEVITAS** line by MORETTI S.p.A. Its design and quality guarantees comfort, safety and reliability to the user. Air cells cushions from the **LEVITAS** line by MORETTI S.p.A. have been designed to satisfy all your demands. This instruction manual contains advices and for your safety and for a correct use of the aid. In case of doubts, contact your retailer.

INTENDED USE

The Air cells cushion from the **LEVITAS** line by MORETTI S.p.A. is designed to ensure the balance and to protect the patient from the risk of forming pressure sores and tissue breaks. It guarantees a perfect weigh distribution, exploiting to the utmost pressure relief in every possible position. In addition, the air cushion adapts itself to the individual shapes of the patient and allows the full immersion of the point sciatic.

• Who are the air cells cushions designed for?

- For anyone who is uncomfortable in the sitting position.
- For wheelchair users.
- For anyone who has to stay in the sitting position for an extended period of time

- (e.g. for the aftercare of an incident or an illness, elderly people, etc).
 - For anyone risking to develop pressure sores.
 - For anyone has to adjust pelvic obliquity.
- **What do we use the air cells cushions for?**
 - For the correct positioning of the pelvis and the lower extremities.
The pelvis and the lower extremities provide a stable base for the trunk, head and neck and the upper extremities.
 - For pressure relief.
 - For prevention of skin breakdown.

WE REMIND YOU THAT IN ANY CASE THE BEST WAY TO PREVENT THE FORMATION OF THE PRESSURE SORES IS TO CONSTANTLY CHANGE THE PRESSURE POINTS BY MAKING THE PATIENT STAND UP OR LIFTING HIM UP REGULARLY.

EC DECLARATION OF CONFORMITY

MORETTI S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI S.p.A. and belonging to the AIR CELLS CUSHION range category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC. To this end MORETTI S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the said directive.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I.
6. DO NOT use/install the devices in question for any other purpose than that expressly indicated by MORETTI S.p.A.
7. MORETTI S.p.A maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities.

TECHNICAL FEATURES

Material

CUSHION COVER PVC
Base in PU - cover and base side cover in NYLON

Dimensions

ST850/1 - 1-section air cells cushion **36x36 cm h 6 cm**
ST850/2 - 2-section air cells cushion **36x36 cm h 6 cm**

ST851/1 - 1-section air cells cushion **40x40 cm h 6 cm**

ST851/2 - 2-section air cells cushion **40x40 cm h 6 cm**

ST852/1 - 1-section air cells cushion **46x40 cm h 6 cm**

ST852/2 - 2-section air cells cushion **46x40 cm h 6 cm**

ST853/1 - 1-section air cells cushion **50x46 cm h 6 cm**

ST853/2 - 2-section air cells cushion **50x46 cm h 6 cm**

ST855/1 - 1-section air cells cushion **36x36 cm h 10 cm**

ST855/2 - 2-section air cells cushion **36x36 cm h 10 cm**

ST856/1 - 1-section air cells cushion **40x40 cm h 10 cm**

ST856/2 - 2-section air cells cushion **40x40 cm h 10 cm**

ST857/1 - 1-section air cells cushion **46x40 cm h 10 cm**

ST857/2 - 2-section air cells cushion **46x40 cm h 10 cm**

ST858/1 - 1-section air cells cushion **50x46 cm h 10 cm**

ST858/2 - 2-section air cells cushion **50x46 cm h 10 cm**

Maximum user's weight

130 Kg

WARNING!

Read carefully this instruction manual for a correct use. Pay attention to the presence of children.

INSTRUCTION FOR USE

CUSHION

Inflation pump is included in each air cells cushion kit.

1. Place air cushion on a chair
2. Twist cushion valve counter-clockwise to open (twisting valve is the milling one)
3. Attach white hose of pump to valve.
4. Inflate cushion.
5. Twist cushion valve clock wise to close and pull out the inflation pump of the cushion valve.
6. Sit down on the cushion
7. Insert the hand under the buttocks at the boniest point.
8. Open the valve slowly and begin to release air. Close the valve when there is at least fingertips distance between buttocks and the cushion.
9. The cushion will be then properly inflated for that person.

In case of more 2-section cushion, you can inflate same method as above

COVER

Each air cells cushion is included with a cover that helps to protect the cushion surface. The bottom of the cover is made of slip-resistant material that fastens the cushion to the chair and helps to keep the patient in balance.

To cover your cushion, follow simple instructions below:

1. Unfold the cover, black slip-resistant base facing downward and the zip upwards.
2. Place the cushion to the cover. Cell part is upwards at fist.
3. Keep air cushion valve outside the cover through holes on the cover.
4. Push the cushion gently to the cover. It may be necessary to shake the cushion side by side in order to make slip in.
5. Close the zip carefully. Air cells cushion is now ready for use.

CAUTION

1. Regulate the cushion pressure with the inflation pump or the manometer every 5-10days.
2. Operate under following temperature and humidity ranges:
 - Temperature range: 0~40°C
 - Humidity range: 10~90%RH

CARE AND MAINTANCE

All the Moretti devices are carefully checked and supplied with CE mark, once launched on the market. Use only original spare parts and accessories.

HOW TO REPAIR AIR CUSHION

1. Deflate air cells cushion.
2. Use abrasive paper to gently sand damaged area until smooth surface.
3. Smear both the damaged surface and patch with few glue.
4. Wait 5 minutes before parching up (apply shaggy fabric on glue).
Press the patch down.
5. Inflate air cushion once repaired area is dry.

CLEANING AND WASHING INSTRUCCION

CUSHION

1. Use lukewarm water and neutral soap to clean the product
2. Wipe the cushion using wet cloth and rinse with cold water.
3. Air dry your air cushion.

Caution

- DO NOT use a non-neutral detergent.
- DO NOT wash air cushion in a washing machine.
- DO NOT use a heat or steam autoclave.
- Away air cushion from direct sunlight.

COVER

1. Wash in the washing machine with cold water and washing powder

or liquid detergent; rinse with cold water, and air dry.

Caution

- DO NOT use drier, heater or bleacher.
- DO NOT iron.

STORAGE

- Temperature range: 0~40°C.
- Humidity range: 10~90%RH .
- There may be up to a 10 % tolerance of all specifications.

GENERAL WARNINGS

Do not use the product for a purpose not indicated in this manual. Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer can modify the device or the instruction manual without prior notification.

WARNING

1. *Keep cushion save from sharp items.*
2. *Avoid direct sunshine and fire.*
3. *Keep cushion save from hot cigarette butts.*
4. *Air cells cushion max weight is 130 kg. DO NOT overload.*
5. *DO NOT use air cushion once it is destroyed by external force.*
6. *Before sitting down, make sure no unusual objects left on the cushion.*
7. *The cushion should be used individually, DO NOT use with other heating or freezing equipments.*

SPARE PARTS/ACCESSORY

For spare parts and accessories please see Moretti S.p.A. master catalogue.

WARRANTY

This product is guaranteed for **2 YEARS** from the purchase date. The distributor reserves the right to repair or replace any parts affected by production defects or defects in the materials used, at no expense for the customer. **The warranty shall not apply in the event of damage caused by improper use, abuse, alterations or dismantling by unauthorized persons.** For effecting repairs in compliance with the above-mentioned conditions for validating the warranty, send the product together with the purchase receipt to the nearest dealer.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Dealer _____

Street _____ Town _____

Sold to _____

Street _____ Town _____

Via Bruxelles 3 - Meleto 52022 Cavriglia (Arezzo) - Italy - Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN KOREA