LOGIKO

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INSTRUCTION MANUAL

MORETTI

Moretti S.p.A.
Via Bruxelles 3 - Melegnano
52022 Cavriglia (Arezzo)
Telefono +39 055 96 21 11

Fax +39 055 96 21 200
www.morettispa.com
info@morettispa.com
Made in P.R.C.

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Via

Località

Congratulations on the purchase of the **MORETTI. LOGIKO ECHO** stethoscope. The line **LOGIKO ECHO** By **MORETTI** is a synonym for stethoscopes designed and made to assure great performance and acoustical superiority. We suggest you to read this instruction manual carefully before the use of the stethoscope.

This manual can give you useful advice for a correct and safe use of the product. In case of any doubts, please contact your retailer for help.

LOGIKO ECHO stethoscopes line by **MORETTI** is a complete line of models that can meet different requirements.

- **DM130**

Single head stethoscope with flat **Head in Anodised Aluminium**. It is suitable especially for the pressure measurement. The flat head can be easily inserted under the cuff of the sphygmomanometer.

- **DM500**

Double head stethoscope, **Head in Anodised Aluminium**, suitable for high and low frequency sounds.

- **DM505**

Double head stethoscope, **Head in Anodised Aluminium**, suitable for high and low frequency sounds for paediatric use.

- **DM530**

Double head stethoscope, **Head in Stainless Steel**, suitable for high and low frequency sounds.

- **DM535**

Double head stethoscope, **Head in Stainless Steel**, suitable for high and low frequency sounds, in particular

for cardiology auscultation. **Adult model.**

- **DM540**

Double head stethoscope, **Head in Stainless Steel**, suitable for high and low frequency sounds. **Paediatric model.**

- **DM545**

Single head stethoscope with flat **Head in mat finishing anodised Zinc** and anatomic head.

- **DM561**

Rappaport stethoscope. It is suitable for a precise auscultation without inconvenience or loss of signal. It offers 5 different auscultation modes thanks to two diaphragm types and three bell types.

USE

For a correct use of the **LOGIKO ECHO** stethoscope, adjust the spring tension of the headset to your needs.

To reduce spring tension in the headset, hold each eartube at the bend near the eartip and gradually pull apart until fully extended. To increase spring tension, grasp the headset with one hand where the eartubes enter the tubing and squeeze until the tubing on the eartube touches the other. Repeat as necessary. To turn the double head, it is sufficient to hold the Y tube in correspondence with the head and turn it by 180° until you hear CLICK.

CAUTION

Before any single use, control the state of the eartips. Once worn-out, the eartips should be replaced to avoid damages to the ear canal.

Avoid extreme heat and cold.

Avoid hot sterilization

Do not immerse the stethoscope in any liquid.

CLEANING

The entire stethoscope can be wiped clean with alcohol or soapy water. If disinfection is required, the stethoscope may be wiped with a 70% isopropyl alcohol solution.

TECHNICAL FEATURES

Product	Material	Diameter of the head	Height of the head
DM130	Aluminium	45 mm	10 mm
DM500	Aluminium	45/31 mm	20 mm
DM505	Aluminium	40/26 mm	20 mm
DM530	Stainless STEEL	46/36 mm	23 mm
DM535	Stainless STEEL	47/36 mm	33 mm
DM540	Stainless STEEL	40/26 mm	20 mm
DM545	Anodized Zinc	46 mm	23 mm
DM561	Aluminium	45/31 mm	36 mm

EEC DECLARATION OF CONFORMITY

MORETTI S.p.A. declares under its sole responsibility that all the items of the stethoscopes product family produced and commercialized by **MORETTI** S.p.A. itself satisfy all the applicable requirements set by the directive 93/42/CEE for Medical Device.

GUARANTEE

This device has 2-year warranty from the purchase date which is certified by the receipt or invoice.

The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge.

The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In the case you need to get your device repaired and you fulfil the conditions of warranty, send the device to the nearest retailer together with the purchase receipt.

WARRANTY CERTIFICATE

Device model _____

Purchasing date _____

Authorized dealer _____

Street

Place

Purchased by

Street

Place