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I Class Medical Device  
according to Directive 93/42/EEC and further modifications

## CODES

<b>CP900-40</b>	RELIEF wheelchair- small wheels - 40 cm seat
<b>CP900-45</b>	RELIEF wheelchair- small wheels - 45 cm seat
<b>CP910-40</b>	RELIEF wheelchair- big wheels - 40 cm seat
<b>CP910-45</b>	RELIEF wheelchair- big wheels - 45 cm seat

## INTRODUCTION

Thank you for purchasing an **ARDEA ONE RELIEF** wheelchair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

**NOTE:** CHECK IF ANY PART HAS SHIPPING DAMAGES AND TEST BEFORE USING. IN CASE OF DAMAGE, DO NOT USE THE DEVICE. CONTACT YOUR DEALER FOR FURTHER INSTRUCTIONS.

## INTENDED USE

The RELIEF wheelchairs are intended for people with movement difficulties and can be for self-propulsion ( when big wheels are assembled) or can be wheeled.

### CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

## DECLARATION OF CONFORMITY


- Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the wheelchair category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.
  - To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility
1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive
  2. The devices in question **ARE NOT MEASUREMENT DEVICES**
  3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
  4. The devices in question are sold in **NON STERILE** packing
  5. The devices in question should be considered as belonging to class I
  6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
  7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

**GENERAL WARNINGS**


- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the wheelchair
- Keep the packed wheelchair away from heat sources
- SERVICE LIFE- the wheelchair use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the wheelchair
- Close the wheelchair before lifting it. DO NOT attempt to lift the wheelchair by any removable parts

**GENERAL DESCRIPTION****- Parts view**

Depending on the model chosen, the composing parts are as follow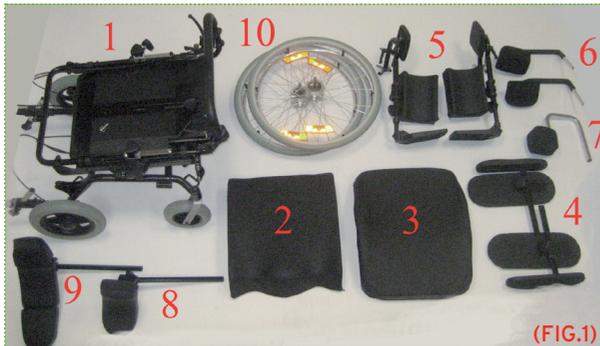

1. Frame
2. Padded seat
3. Padded backrest
4. Armrest and armrest sideboard
5. Elevating legrest
6. Shoulder stopper
7. Hip stopper
8. Headrest
9. Three-dimensional headrest (optional)
10. 60cm back wheels

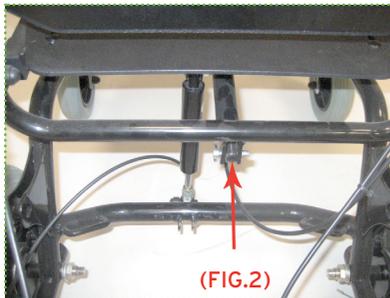**ASSEMBLING****- Open the wheelchair**

Lift the backrest and connect the backrest to the piston by the hinge (FIG.2)

## - Backrest and seat assembly

(FIG.3)

Lean the padded seat and backrest on the frame, in correspondence to the Velcro strips. Fix them with a bit pressure (FIG.3)

## - Armrest assembly

(FIG.4)

Insert the armrest as shown in (FIG.4) adjust the height as your needs and lock with "A" and "B" knobs.

## -Footrest assembly

(FIG.5)

Insert the legrest on the hinge on the frame, then rotate inwards until the level are locked (FIG.5)

### -Hip stopper assembly

Insert the hip stopper in the tube under the seat. Adjust the position and lock with knobs (FIG.6)

### - Shoulder stopper assembly

Insert the shoulder stopper in the supports behind the backrest. Adjust the position and lock with knobs (FIG.7)

### - Headrest assembly

Insert the headrest in the support behind the backrest. Adjust the position and lock with knobs (FIG.8)

### **BEFORE USE**

- Check the automotive parts usage, in order to guarantee a safe use of the device for people and things
- Assure all the parts are correctly assembled. Checking carefully the correct assembling of the rear wheels
- Assure the brakes on the back wheel correctly works

### **USING WARNINGS**

- Do not use the wheelchair as seat for vehicle
- Assure the brake handles is duly adjusted and correctly works
- Avoid entrapment of finger into wheels or brakes during the wheelchair use
- The wheelchair temperature increase if exposed to the sun for long period

### **HOW TO USE**

#### **WARNING!**

- Getting in and out of the wheelchair may be performed independently, provided you have an adequate mobility and upper body strength
- Do not stand on removable parts

**DO NOT STAND ON THE FOOTREST WHEN GETTING IN OR OUT OF THE WHEELCHAIR, OVERTURNING RISK!!!**

#### **- Get in the wheelchair**

1. Position the wheelchair as close as possible to you
2. Engage wheel locks . We suggest to remove the armrest to ease the patient getting in the wheelchair
3. Lean on the frame with arms and slowly go down towards the seat

#### **- Get out of the wheelchair**

1. Position the wheelchair as close as possible alongside the seat to which you are transferring
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting out of the wheelchair
3. Shift body weight into seat with transfer

#### **- Stairs and height difference (how to face them)**

#### **WARNING!**

The following method must be used only from experienced patients. If you are not experienced, let you help from an assistant.

**GET ON**

Position the wheelchair at the curb to be crossed. Balance the wheelchair on the rear wheels, in order to lift the front wheels, enough to lean them on the obstacle. Bend your trunk forward and vigorously push the handrail. Continue pushing until the rear wheels are climbing over the obstacle.

**GET OFF**

To get off the stair follow the reverse procedure, positioning the rear side of the wheelchair at the curb, keep the handrails vigorously and get off the obstacle slowly; then balance the wheelchair on the rear wheels in order to complete the descent.

**WARNING!**

WE RECOMMEND TO USE THIS METHOD ONLY IN PRESENCE OF OBSTACLE LOWER THAN 10CM !!

**- Use the armrest**

The armrest can be adjusted in height and width:

**Adjust the height** - unscrew the knob for armrest lock and adjust the height

**Adjust the width** - unscrew the knob under the seat and adjust the width

**-Use the legrest**

The legrests can be adjusted in length, inclination, width and depth:

**Adjust the length** - Unscrew the knob on the legrests frame. Adjust the length of the legrests according to the patient's needs. Lock the knob again.

**Adjust the inclination** - Unscrew the side knob placed at the knees height and change the inclination. Lock the knob again

**Adjust the width** - You need a 6 mm allen wrench. Unscrew the screw under the seat and adjust the width according to the patient's needs. Lock the screw again

**Adjust the depth** - Unscrew both the side knobs placed under the seat, adjust the depth according to the patient's needs and lock the knobs again.

**-Use the reclining backrest**

Adjust the backrest inclination by pressing the left lever on the handle bar and press the backrest down. Release the backrest to take it back to the usual inclination

**-Use the seat**

Adjust the inclination of the seat by pressing the right lever on the handle bar and press the seat down. Release the seat to take it back to the usual inclination

**WARNING!**

**These techniques are intended to be used only by experienced users. In contrary case it is required a technical assistance**

**-Use the sliding seat**

The top of the seat can be adjusted in depth to give the patient more comfort. Unscrew the 10mm nuts under the seat and adjust the position onwards or backwards.

**-Use the hip stopper**

Use the hip stopper to avoid the continuous contact between lower limbs. It can be adjusted by using the knob under the seat.

**- Use the rear big wheels**

The rear 60cm wheels are used when self-propelled. Grasp the handrail and wheel through the desired direction

**- Use the rear small wheels**

The rear 30cm casters are used when wheel by an assistant through the handles

**- Use the quick release wheels****WARNING!**

- 1. Before removing a wheel make sure that the wheelchair does not incur accidental falls**
- 2. Do not remove a wheel with the patient seated on the wheelchair**

The Relief wheelchair is equipped with quick release wheels. To remove the wheel press the button in the centre and extract the wheel outwards. Follow the procedure reversed to position the wheel.

**-Use the shoulder stopper**

The shoulder stopper can be adjusted in height, length and in orientation:

**Adjust the height/width** - unloose the knob on the backrest to adjust height and length

**Adjust the orientation** - in the back of each stopper there are spherical pins which allow to adjust the shoulder stopper by unloosing the knob in the back.

**-Use the headrest**

The headrest can be adjusted in height, depth and orientation:

**Adjust the height** - unloose the knob in the back of the headrest

**Adjust the depth** - unloose the knob in the back of the headrest. The support allow the headrest to be adjusted in depth and inclination.

**Adjust the orientation** - the back of the headrest is equipped with a spherical pin which allow the headrest to be oriented

**-Use the handle bar**

The handle bar is intended to wheel the wheelchair or to adjust the backrest and the seat inclination. The handle bar can be adjusted in height by unloosing the knobs on the bar, then lift or lower it.

**-Use the anti-tipper castors**

Adjust the height by pulling the metallic ring, release it when the correct adjustment is found

**-Use the brakes**

The back wheels are provided with drum brakes. Press the correspondent lever on the handle bar. Release to wheel the wheelchair.

**NOTE THE BRAKES CAN AUTOMATICALLY REMAIN LOCKED. TO USE THIS FEATURE PUSH THE LOCKING ELEMENT UNDER THE BRAKE LEVER. TO UNLOCK THE BRAKE PRESS THE LEVER UNTIL THE LOCKING ELEMENT IS COMPLETELY OUT FROM THE GEAR.**

**- Carrying**

The wheelchair has been designed to be easily carried. Follow the reversed procedure described in "Assembly" chapter. The closed wheelchair can be carry in the car boot.

## **MAINTENANCE**

The **ARDEA ONE** devices by Moretti S.p.A are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to:

- Check the frame screws at the first use and during the first month
- Every 3 months check all the components status, in particular verify the screws and the nuts are duly tight

In case of repair, use only original spare parts and accessories.  
Follow the suggestions in chapter "BEFORE USE" .

## **CLEANING AND DISINFECTION**

### **- Cleaning Cover**

The cover is in artificial leather and they all can be removed. Clean with warm water and mild soap

### **- Cleaning pad**

Clean with warm water and mild soap

### **-Cleaning frame and metallic parts**

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting.

### **- Cleaning brakes and wheels**

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous use the wheelchair with wet wheels.

### **- Disinfection**

If you need to disinfect the device use a common disinfectant cleaner

**NOTE NEVER USE ACIDS, ALKALIS OR SOLVENTS SUCH AS ACETONE**

**CONDITIONS OF DISPOSAL****- General conditions of disposal**

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycling.

**SPARE PARTS/ACCESSORIES**

For spare parts and accessories refer to main catalogue

**TECHNICAL FEATURES****- Dimension**

Dimensions and weight																
Measures in cm																
<b>CP900-40</b>	113	81	45	108	64	57	40/60	40	49	57	-	41	30	20	5	100
<b>CP900-45</b>								45		62		43				
<b>CP910-40</b>								40		60		45				
<b>CP910-45</b>	113	81	45	108	64	57	40/60	45	49	65	-	46	60	20	5	100

**- Technical specifics**

• <b>Frame</b>	Powder coated steel
• <b>External cover</b>	Synthetic leather
• <b>Internal cover</b>	Cotton
• <b>Backrest/seat/headrest pad</b>	PU
• <b>Armrest/ hip and shoulder stoppers /legrest pad</b>	Foam
• <b>Front wheels</b>	20 cm PU
• <b>Small rear wheels</b>	30 cm PU
• <b>Big rear wheels</b>	60 cm PU
• <b>Rear brakes</b>	Drum brakes
• <b>Seat angle</b>	6° front
	16° rear
• <b>Backrest inclination</b>	0-50°
• <b>Weight</b>	24 kg
• <b>Max weight</b>	100 kg
• <b>Resistance to ignition</b>	the materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2
• <b>Norme di riferimento</b>	EN12183
• <b>Environmental temperature</b>	-20°/+40°C
• <b>Storage</b>	Store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)

## WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## REPAIRING

### - Warranty repairing

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

### - Repair not covered by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

### - Not-defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

## SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

**EXEMPT CLAUSES**

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, , including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**WARRANTY CERTIFICATE**

**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

MADE IN P.R.C.

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)