DIMED

CE

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Moretti S.p.a.INTERNET: www.morettispa.com E-MAIL: info@morettispa.com

MADE in P.R.C.

I Class Medical Device
according to ECC directive 93/42

CODES

DMV520 FIBER OPTIC OTOSCOPE

DMV525 OPHTALMOSCOPE

DMV530 OTOSCOPE AND OPHTALMOSCOPE KIT

INTRODUCTION

Thank you for purchasing an otoscope/ophtalmoscope Dimed by Moretti S.p.A. Their design and quality guarantee comfort, safety and reliability to the user. This instruction manual contains advices for your safety and for a correct use of the device. We suggest you to read this instruction manual before the use of the product. In case of doubts, contact your retailer.

INTENDED USE

OTOSCOPE

A medical device that allows the analysis of the auditory canal.

OPHTALMOSCOPE

A medical device that allows the analysis of the eye, in particular, the retina of the eyeball.

These devices can be used by professional and trained personnel only.

EC DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the products made and traded by Moretti S.p.A. and belonging to otoscopes/ophtalmoscope range are in accordance with the European Directive on Medical devices (93/42/CEE).

Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

- 1) The above mentioned products satisfy all the requirements as per annex I of the directive 93/42/CEE as per annex VII^o of the before mentioned directive;
- 2) The above mentioned products are NOT MEASURING DEVICES;
- 3) The above mentioned products are NOT INTENDED FOR USE IN CLINICAL VALIDATION;
- 4) The above mentioned products are NON STERILE;
- 5) The above mentioned products are to be considered "Class I^o Medical Devices;
- 6) IT IS NOT ALLOWED to use the above mentioned products except for the use indicated by the Moretti S.p.A.;
- 7) The manufacturer makes all the technical documentations available to the relevant authorities which prove the compliance with the 93/42/EEC Directive;

ASSEMBLING INSTRUCTION **OTOSCOPE/OPHTHALMOSCOPE**

Unscrew the base and insert inside two alkaline batteries (AA) 1,5 V Place them with the positive pole orientated to the head of the device. Fasten the base again.

WARNING!

Read carefully this instruction manual for a correct use. Pay attention to the presence of children.

INSTRUCTIONS

These devices can be used by professional and trained personnel only.

OTOSCOPE

Adjust the adaptor on the base, considering the auditory canal of the patient (usually the 4 mm model is for adult, while 2,4 mm is for children). Switch on the optical fiber light with the switch on the handle. Insert the speculum in the patient ear to do the physical examination. If necessary, you can also use the magnifying glass in the head of the device (possible magnification 3x).

OPHTALMOSCOPE

Switch on the light with the switch on the handle. Adjust the quality of the image with the lateral cog to choose the appropriate dioptré. In front of the device there is the cog to adjust the beam of light to analyze the eye.

MAINTENANCE

Dimed by Moretti S.p.A. devices are CE marked and checked before the launch on the market. We suggest you to remove batteries if you don't use the device. Use only original accessories and spare parts.

REPLACEMENT

HEAD

Switch off the light, unscrew the head anticlockwise and replace it with a new head. Screw clockwise until the end.

HANDLE

Unscrew the handle and remove the batteries. Screw the handle again clockwise until the end.

BATTERIES

Unfasten the final part of the handle, place there 2 AA 1,5V batteries with positive pole turned to the head of the

device, then fasten again.

ATTENTION

Wait at least 5 minutes after turning off the light of the device because it could heat.

CLEANING

Use only a damp cloth.

N.B. the otoscope speculum can be sterilized for 20 minutes at 134° C.

GENERAL WARNINGS

Do not use the products for purposes not indicated in this manual. Moretti S.p.A. declines all responsibility for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer can modify the device or the instruction manual without prior notification.

TREATMENT OF FLAT BATTERIES

DISPOSAL OF THE PRODUCT ACCORDING TO EC DIRECTIVE 2002/96/EC

Batteries are hazardous waste. Do not dispose them together with house-hold garbage. At the end of their life hand the batteries over to the applicable collection points for the recycling of waste batteries. For more detailed information about recycling of this product or battery, please contact your local Civic Office, or the shop where you purchased the product.

TECHNICAL FEATURES

OTOSCOPE

- **LIGHTING SYSTEM** Optical fiber
- **BRIGHTNESS** 18,5 Lumen
- **MAGNIFICATION** 3X
- **LIGHTING BULB** 2,5V 0,7° diameter 3,8 mm
- **BATTERIES** 2 AA 1,5V alkaline
- **WORKING CONDITIONS:**
Temperature: 10°C/40°C
Humidity: 30% - 75%
Pressure: 700hPa - 1060hPa
- **STORAGE CONDITIONS**
Temperature -40°C/55°C
Humidity 10% - 90%
Pressure 500hPa - 1060hPa

OPHTHALMOSCOPE

- **LIGHTING BULB** 2,5V 0,7° diameter=3,8 mm
- **WORKING CONDITIONS**
Temperature 10°C - 40°C
Humidity 30% - 75%
Pressure 700hPa - 1060hPa
- **STORAGE CONDITIONS**
Temperature: -40°C/55°C
Humidity: 10% - 90%
Pressure: 500hPa - 1060hPa

SPARE PARTS/ACCESSORIES:

Check the Moretti S.p.A. catalogue to see the spare parts and the accessories.

WARRANTY

The product has 2-year warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

**MORETTI****DIMED****WARRANTY CERTIFICATE**

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Sold to _____

Street _____ Place _____

Moretti S.p.a.INTERNET: www.morettispa.com E-MAIL: info@morettispa.com

MADE in P.R.C.

Moretti S.p.A.

Via Bruxelles 3 - Meleto 52022 Cavriglia
(Arezzo)

Tel. +39 055 96 21 11

Fax. +39 055 96 21 200

www.morettispa.com

info@morettispa.com