GARANZIA

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WARRANTY

The product has 2-YEAR warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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EASYRED

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Acquistato in data / Purchasing date _____

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Code EM250

Thank you for purchasing the **EASYRED** by Moretti spinal board. Made of High density PE, it is reinforced with 2 carbon fiber bars, in order to support heavy weight.

Intended use

Device for conventional or for water rescue.

Technical features

Size: 183x42x4,4cm

Weight: 6,1kg

Loading capacity: 275kg

Instructions

- 1) Place the spinal board near to the patient
- 2) transfer carefully the patient on the spinal board
- 3) immobilize the patient using the standard belts or the spider straps
- 4) it is possible to move the spinal board from different angles, thanks to lots of handles

Warning

- 1) Only qualified personnel can use this product and can assist the injured person; pay attention, above all, in case of spinal lesions.
- 2) Do not use this product for purposes not indicated in this manual.
- 3) Moretti SpA is not responsible for purposes not indicated in this manual.

Cleaning

Use water and non aggressive disinfectant; dry far from sun rays.