**SKEMA®**

MA MI432-MI433-MI435-MI436-MI437 01 A.IT\_06-2016

**PIANTANA PER IPODERMOCLISI**

---

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## INDICE

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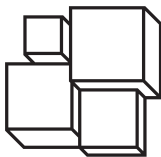
CE - I Class medical device,  
according Directive 93/42/EEC and subsequent amendments

## 1. CODES

<b>MI432</b>	Aluminium I.V. Poles with 2 Plastic Hooks and casters
<b>MI433</b>	Aluminium I.V. Poles with 4 Plastic Hooks and casters
<b>MI435</b>	Stainless steel I.V. Poles with 2 Plastic Hooks and casters
<b>MI436</b>	Stainless steel I.V. Poles with 2 Plastic Hooks and casters
<b>MI437</b>	Stainless steel I.V. Poles with 4 stainless steel Hooks and casters

## 2. INTRODUCTION

Thank you for choosing a I.V. Poles from the SKEMAFORM by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. I.V. Poles from the SKEMAFORM by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the item you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the I.V. Poles. Should you have any queries, please contact your retailer for advice and assistance.

## 3. INTENDED USE

The I.V. Poles for hypodermoclysis SKEMAFORM is a support device medical and health activities during administration to the patient of intravenous fluids.

### WARNING!

- Do not use the product for a purpose not indicated in this manual;
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product;
- The manufacturer reserve the right to change the information contained in this document without previous notice.

## 4. DECLARATION OF EC CONFORMITY

MORETTI S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI S.p.A. and belonging to the I.V. Poles category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC.

To this aim MORETTI S.p.A. guarantees and declares as follows, under its own responsibility:

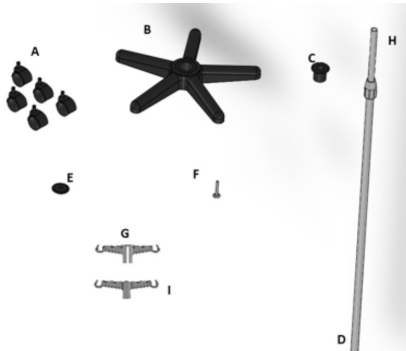
1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the mentioned directive. The complete list of the devices in question is available in annex.
2. The devices in question are not measurement tools.
3. The devices in question are not intended for clinical investigation.
4. The devices in question are sold in non sterile packing.
5. The devices in question should be considered as belonging to class I.
6. Do not use/install the devices in question for any other purpose than that one expressly indicated by Moretti S.P.A.
7. Moretti S.P.A. maintains and submits the technical documentation certifying the conformity of the products in question with the directive 93/42/eeC to the proper authorities.

## 5. GENERAL WARNINGS

- For proper use of the device refer to this manual;
- Keep the packaged product away from any source of heat because the packaging is made of cardboard;
- The useful life of the accessory is determined by wear of parts cannot be repaired and/or replaced;
- Be careful when children are nearby.

## 6. GENERAL DESCRIPTION


### 6.1 Parts view and description Aluminium I.V. Poles MI432, MI433

#### Parts list:

- A. Base casters (5pcs)
- B. Base (1pc)
- C. Pole connection (1pc)
- D. Extendible pole, lower part (1pc)
- E. Plastic washer (1pc)
- F. Screw with washer for pole (1pc)
- G. Plastic hook female (1pc) \*(Only MI433)
- H. Extendible pole, upper part (1pc)
- I. Plastic hook male (1pc)

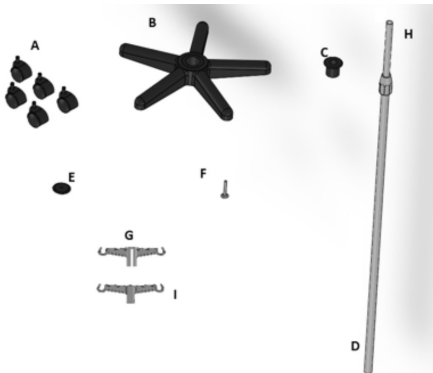
### 6.2 Parts view and description stainless steel I.V. poles MI435, MI436

#### Parts list:

- A. Base casters (5pcs)
- B. Base (1pc)
- C. Pole connection (1pc)
- D. Extendible pole, lower part (1pc)
- E. Plastic washer (1pc)
- F. Screw with washer for pole (1pc)
- G. Plastic hook female (1pc) \*(Only MI436)
- H. Extendible pole, upper part (1pc)
- I. Plastic hook male (1pc)
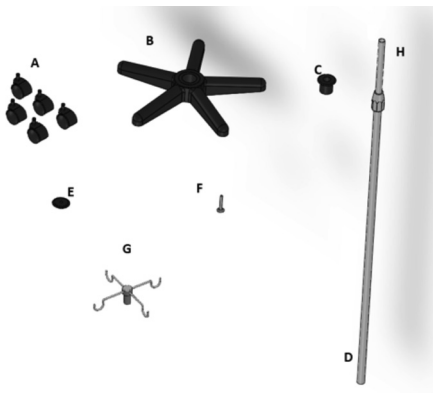
### 6.3 Parts view and description stainless steel I.V. MI437




#### Elenco parti:

- A. Base casters (5pcs)
- B. Base (1pc)
- C. Pole connection (1pc)
- D. Extendible pole, lower part (1pc)
- E. Plastic washer (1pc)
- F. Screw with washer for pole (1pc)
- G. Stainless steel hook (1pc)
- H. Extendible pole, upper part (1pc)



**7. ASSEMBLING**

**ATTENTION!**

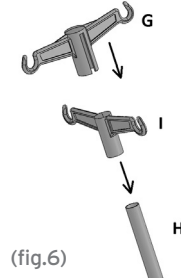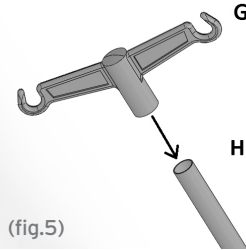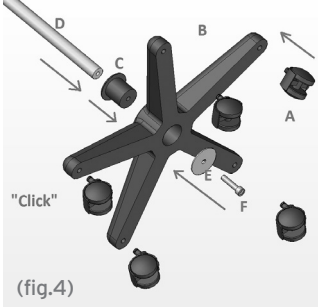During the assembling pay attention to avoid any injury to persons and/or damage to objects.

**7.1 Assembling instructions - MI432, MI433, MI435, MI436**

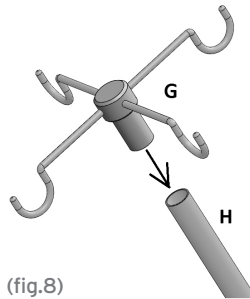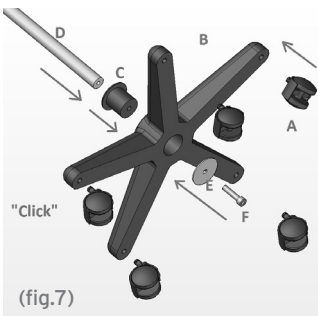1. Insert the casters (rif.A) in the lower part of the base (rif.B) until you hear a "click". (Fig.4)
2. Insert the pole connection (rif.C) on the upper hole of the base. (Fig.4)
3. Insert the pole (rif.D) in the connection (rif.C). (Fig.4)
4. Block the pole using the screw (rif.F) and its washer (rif.E). (Fig.4)
5. Tighten the screw. (Fig.4)
6. Insert the female hook (rif.G) (Only for MI433- MI436 (Fig.5) in the male hook (rif.I) and both in upper hole of the pole (rif.H) with a light pressure. (Fig.6)
7. Now the I.V. pole is ready to use.


**7.2 Assembling instructions MI437**

1. Insert the casters (A) in the lower part of the base (B) until you hear a "click"
2. Insert the pole connection (C) on the upper hole of the base
3. Insert the pole (D) in the connection (C)
4. Block the pole using the screw (F) and its washer (E)
5. Tighten the screw
6. Insert the hook (G) into upper hole of the pole (H) with a light pressure
7. Now the I.V. pole is ready to use

**8. HOW TO USE**

To adjust the height of the I.V. Pole turn the knob counterclockwise, fix the pole to the right height and turn the knob clockwise to block it.

Images only illustration purposes

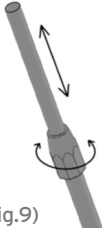(fig.9)

**WARNINGS!**

- Check the pole before every use. Make sure it hasn't any damage or signs of wear that could compromise its working. If it should be damaged, do not use the pole.
- Make sure the position of the pole is comfortable and safe for the patient, pay attention to the castors movement
- Do not overload the hooks, check the maximum weight on the technical features

**9. CLEANING AND DISINFECTION**

**9.1 Cleaning**

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

**WARNINGS!**

Do not use abrasive substances, acids, alkaline, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

**9.2 Disinfection**

If you need to disinfect the product use a common disinfectant neutral detergent.

**10. CONDITIONS OF DISPOSAL**

**10.1 General Condition of disposal**

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

**11. TECHNICAL FEATURES**

- Base made in Polypropylene molded
- 5 castors Ø50mm
- Pole adjustment by knob
- ABS Hooks molded (MI432-MI433-MI435-MI436)
- Max payload 5Kg for hook

**11.1 Dimensions e weight**

Codes	Height (min/max)	Base dimension (mm)	Maximun load capacity per hook	Maximun payload
MI432	1330mm/2300mm	500mm	5kg	10kg
MI433	1330mm/2300mm	500mm	5kg	20kg
MI435	1170mm/2030mm	500mm	5kg	10kg
MI436	1170mm/2030mm	500mm	5kg	20kg
MI437	1170mm/2030mm	500mm	5kg	20kg

**12. WARRANTY**

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations,

Images only illustration purposes

damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

### 13. REPAIRING

#### 13.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

#### 13.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

#### 13.3 Non defective device

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

### 14. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

### 15. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

#### WARRANTY CERTIFICATE

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

 **MADE IN ITALY**

**SKEMA®**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)