GIMA

GIMA S.p.A. - Via Monza, 102 - 20060 Gessate (MI) - Italia

ITALIA: Tel. 199 400 401 (8 linee r.a.) - Fax 199 400 403

E-mail: gima@gimaitaly.com - www.gimaitaly.com

EXPORT: Tel. ++39 02 953854209 - Fax ++39 02 95380056

E-mail: export@gimaitaly.com - www.gimaitaly.com

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HYSTOFREEZER

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GIMA

TECHNICAL SPECIFICATIONS HYSTOFREEZER

Hystofreezer® is a medical aid for cryotherapy of virosis and other benign skin lesions (warts, age spots, actinic keratosis).

1. COMPOSITION AND PHYSICAL FEATURES

Hystofreezer® box is composed of following items:

- Spray bottle, containing liquid gas made of diethyl ether, propane and isobutene, it does not harm the ozone layer.
- Disposable applicators, medium (5 mm) and small (2 mm)
- Instruction sheet.

2. STORAGE AND TRANSPORT

- Spray bottle is pressurized and its content is flammable
- Please avoid direct sun light exposure and temperatures above 50° C
- Do not perforate or burn bottle even after use
- Do not spray on flames or burning objects

3. OPERATIONAL PROCEDURE

Hystofreezer® exploits cryotherapy principles: the sponge tip of applicators is saturated with a mixture of liquid gas which through evaporation decreases tip temperature to - 55° C.

Tip application on skin for a variable time between 14 and 40" seconds brings about freezing and following skin necrosis, this will lead to lesion removal.

The number of treatments may vary from one to four, with two week breaks among applications

4. INDICATIONS

Skin lesions either benign or no benign, generically called warts.

Skin blemishes such as Age Spots, Actinic Keratosis, Seborrheic Keratosis, Molluscum Contagiosum Genital Warts and Skin tags.

5. CONTRAINDICATIONS

Cryotherapy is contraindicated for patients suffering from cryoglobulinemia

6. CAUTIONS

Do not use in case of uncertain diagnosis (possibility of skin cancers).

Dark skin individuals can experience depigmentation phenomenons.

7. CLASSIFICATION

Hystofreezer® pertains to the Medical aid category (**CE 0197 mark**)

8. PRODUCT CODES AND PACKAGES

- Cod. 30585: one 40 ml bottle containing a cryogenic mixture of diethyl ether, propane and isobutane + 5 applicators
- Cod. 30586: one 150 ml bottle containing a mixture of diethyl ether, propane and isobutane + 50 mix applicators
- Cod. 30587: one 150 ml bottle containing a mixture of diethyl ether, propane and isobutane + 50 2 mm applicators
- Cod. 30588: one 150 ml bottle containing a mixture of diethyl ether, propane and isobutane + 40 5 mm applicators

Moreover four boxes include an instruction leaflet.

9. PRODUCTION

Hystofreezer® is a registered trademark of OraSure Technologies, Inc.

Manufacturer: OraSure Technologies Inc., Bethlehem PA - U.S.A.