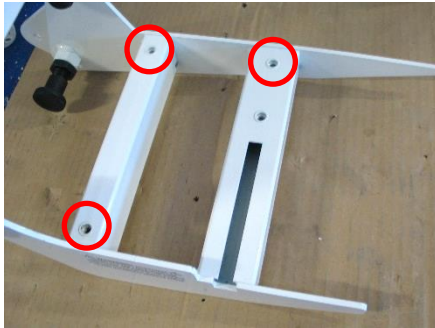
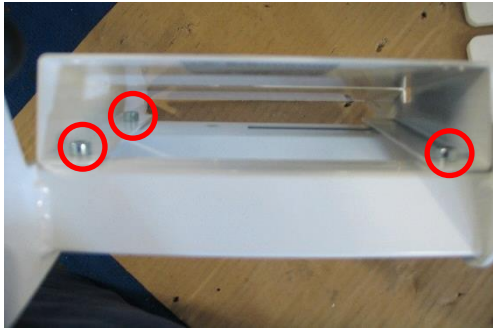
**AC0096 – WEDGE FOR CLUB FEET**  
**CUNEO PER PIEDE EQUINO**

<b>Manufacturer</b>	<p><b>CHINESPORT S.P.A</b>, Via Croazia, 2-33100 Udine, Italia.          Phone. +39 0432 621 621 – Fax +39 0432 621 620 – website: www.chinesport.com          The company produces according to UNI EN ISO 13485:2012</p>
<b>Intended use and equipment description</b>	<p>Wedges to be mounted on the footplates. They allow to increase the extension by 30 ° and the flexion of 10 ° if necessary for a patient with club foot.</p>
<b>Composition</b>	<p>Painted steel.</p>
<b>Storage</b>	<p>In case of storage the following ambient conditions must be respected:</p> <ul style="list-style-type: none"> <li>- relative humidity 10% / 90 %;</li> <li>- temperature -10°C / +50°C.</li> </ul>
<b>Special warning and use restrictions</b>	<p><b>These devices must not be used:</b></p> <ul style="list-style-type: none"> <li>• without making all the settings required and recommended setups.</li> <li>• when use is not advisable or causes the patient serious discomfort.</li> <li>• if the original product has been tampered with and/or modified</li> <li>• when set incorrectly or when there is any suspicion that it may harm the patient</li> </ul>
<b>Collateral effects and contraindications</b>	<p>There is no collateral effects or contraindications during proper use. Use the device always in the proper way. The knowledge of this instruction manual is recommended.</p>
<b>Unpacking and mounting</b>	<p>Simply remove the device from the package. Never use cutters to open box to prevent damages. Properly dispose of the packaging. The carrier and the supplier must be immediately notified of any damage that may have occurred during transport. Damaged equipment cannot be used.</p>
<b>Installation and use</b>	<p>Remove the footrest from the platform and screw the wedge into the three holes shown in Figs. 1 and 2. Use the cylinder head screws. Make sure that the wedge groove corresponds to that of the platform (ie outward). Apply the footrest over the wedge with the countersunk head screws as shown in Figs. 3 and 4 by applying the nuts on the underside (Fig. 5).</p> <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;">  <p>Fig. 1</p> </div> <div style="text-align: center;">  <p>Fig. 2</p> </div> </div>

**AC0096 – WEDGE FOR CLUB FEET**  
**CUNEO PER PIEDE EQUINO**

Fig. 1

Fig. 2

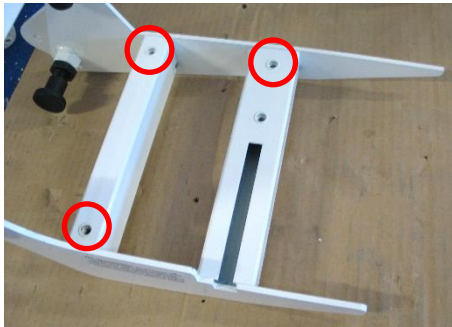
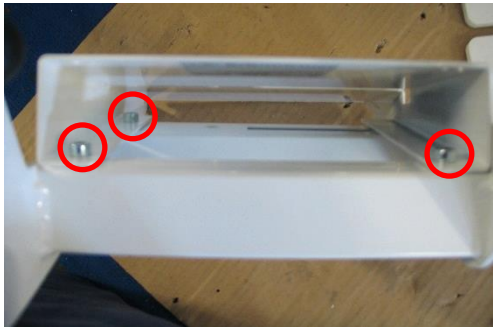
Fig. 3

	  				
<p><b>Maintenance</b></p>	<p>Check the condition of the safety of the device after each situation of "rough treatment" (ex.: fall, heavy impacts, liquid spills, etc.) and any time you are unsure about the state security apparatus. Routine maintenance tasks are to be carried out at least once every 6 months.</p> <table border="1" data-bbox="395 1294 1452 1422"> <thead> <tr> <th data-bbox="395 1294 630 1323">FREQUENCY</th> <th data-bbox="630 1294 1452 1323">TASK</th> </tr> </thead> <tbody> <tr> <td data-bbox="395 1357 630 1386">Every six months</td> <td data-bbox="630 1323 1452 1422">                     General cleaning                      Visual and functional inspection of the equipment                      Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation                 </td> </tr> </tbody> </table>	FREQUENCY	TASK	Every six months	General cleaning Visual and functional inspection of the equipment Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation
FREQUENCY	TASK				
Every six months	General cleaning Visual and functional inspection of the equipment Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation				
<p><b>Cleaning</b></p>	<p>Dust gently and in case of light stains pass the surface with a damp cloth or a neutral detergent, rinse with water. Dry avoiding direct exposure to the sunlight.                      The following instructions are recommended for small located and more obstinate stains (oil, fat, cosmetics, ink, coffee, liquor, chewing-gum etc.): dissolve immediately with clear ethylic alcohol diluted in water to 20%, dab and then treat with a neutral detergent solution. Rinse abundantly.</p>				
<p><b>Disinfection</b></p>	<p>For the disinfection of surfaces such as chassis, sky, hand control, etc. we recommend the use of disinfectants such as AMUCHINA® 10% or equivalent disinfectant with sodium hypochlorite 0.1%</p>				
<p><b>Additional informations</b></p>	<p>Please visit <a href="http://www.chinesport.com">www.chinesport.com</a></p>				

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