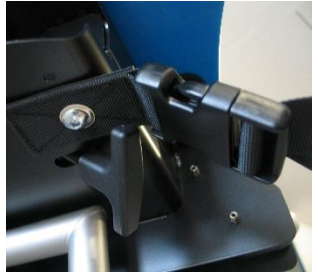
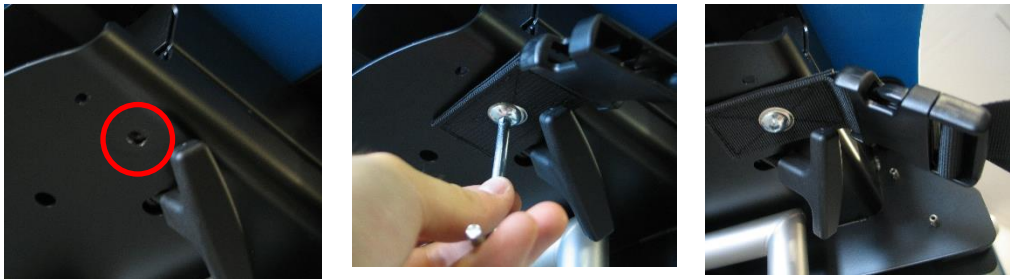
**AC0783 – BACK STRAP**  
**CINGHIA SUPPORTO DORSALE**

<b>Manufacturer</b>	<b>CHINESPORT S.P.A</b> , Via Croazia, 2-33100 Udine, Italia. Phone. +39 0432 621 621 – Fax +39 0432 621 620 – website: <a href="http://www.chinesport.com">www.chinesport.com</a> The company produces according to UNI EN ISO 13485:2012					
<b>Intended use and equipment description</b>	When necessary, it is possible to apply easily a strap for a greater restraint at back level once the person has reached the upright position. This strap is applied by an assistant, and its function is mainly to prevent risk movements as a result of an improper control of the trunk by the user. It is available in one size only.					
<b>Composition</b>	Synthetic fabric.					
<b>Storage</b>	In case of storage the following ambient conditions must be respected: <ul style="list-style-type: none"> <li>- relative humidity 10% / 90 %;</li> <li>- temperature -10°C / +50°C.</li> </ul>					
<b>Special warning and use restrictions</b>	<b>These devices must not be used:</b> <ul style="list-style-type: none"> <li>• without making all the settings required and recommended setups.</li> <li>• when use is not advisable or causes the patient serious discomfort.</li> <li>• if the original product has been tampered with and/or modified</li> <li>• when set incorrectly or when there is any suspicion that it may harm the patient</li> </ul>					
<b>Collateral effects and contraindications</b>	There is no collateral effects or contraindications during proper use. Use the device always in the proper way. The knowledge of this instruction manual is recommended.					
<b>Unpacking and mounting</b>	Simply remove the device from the package. Never use cutters to open box to prevent damages. Properly dispose of the packaging. The carrier and the supplier must be immediately notified of any damage that may have occurred during transport. Damaged equipment cannot be used.					
<b>Installation and use</b>	Screw the band ring to the frame at the threaded holes below the table. Use by firmly connecting the straps. <div style="display: flex; justify-content: space-around; margin-top: 10px;">    </div>					
<b>Maintenance</b>	Check the condition of the safety of the device after each situation of "rough treatment" (ex.: fall, heavy impacts, liquid spills, etc.) and any time you are unsure about the state security apparatus. Routine maintenance tasks are to be carried out at least once every twelve months. <table border="1" style="width: 100%; margin-top: 5px;"> <thead> <tr> <th style="width: 30%;">FREQUENCY</th> <th>TASK</th> </tr> </thead> <tbody> <tr> <td>Every twelve months</td> <td>                     General cleaning                      Visual and functional inspection of the equipment                      Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation                 </td> </tr> </tbody> </table>		FREQUENCY	TASK	Every twelve months	General cleaning Visual and functional inspection of the equipment Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation
FREQUENCY	TASK					
Every twelve months	General cleaning Visual and functional inspection of the equipment Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation					
<b>Cleaning</b>	Dust gently and in case of light stains pass the surface with a damp cloth or a neutral detergent, rinse with water. Dry avoiding direct exposure to the sunlight. The following instructions are recommended for small located and more obstinate stains (oil, fat, cosmetics, ink, coffee, liquor, chewing-gum etc.): dissolve immediately with clear ethylic alcohol diluted in water to 20%, dab and then treat with a neutral detergent solution. Rinse abundantly.					
<b>Disinfection</b>	For the disinfection of surfaces such as chassis, sky, hand control, etc. we recommend the use of disinfectants such as AMUCHINA® 10% or equivalent disinfectant with sodium hypochlorite 0.1%					
<b>Additional informations</b>	Please visit <a href="http://www.chinesport.com">www.chinesport.com</a>					

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