



*Cleaning Instructions for Covers  
PREVENTIX range mattresses*



CARPENTER





The covers for the PREVENTIX range mattresses are composed of a fabric coated with polyurethane. The material used is waterproof to liquids but permeable to vapour. It is also resistant to bacteria and mould.

Our covers are removable and zippers are protected from the aggression of liquids by a flap.

In order to preserve our products quality and to extend their life, we advise you to follow the instructions below.

#### **GENERAL INSTRUCTIONS**

- ✿ Regularly check the general state of the cover (holes, abrasion...) and do not use the mattress anymore if there is a defect on the cover.
- ✿ Do not use a mattress with a cover still damp or that would not be completely dry.
- ✿ In order not to damage the cover, it is not advised to use solvents or chloric cleaning solutions. In case chloric products are required, it is advised to dilute chlorine and to rinse several times the cover with clear water.
- ✿ During cleaning of the original cover, please use a spare cover.
- ✿ Sterilization is not advised as this could damage the product.

#### **DAILY MAINTENANCE OF THE COVER**

- ✿ If the patient is not infectious, it is only necessary to clean the cover surface with a non chloric decontamination lotion.

Caution : Use single-use gloves.



CARPENTER



## PERIODIC MAINTENANCE OF THE COVER

- If the patient is infectious, washing in machine is advised.
- Before washing machine, either spray the cover with a non chloride decontamination lotion and let the lotion acts, or damp the cover into a decontamination lotion.
- Machine washable up to 90°C with neutral detergents, without any softener.
- Air dry or in machine at a moderate temperature.

Caution : Use single-use gloves.



CARPENTER

