MOPEDIA®

MA RP515_RP538 01 A_09-2018

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CE I Class Medical Device according to directive 93/42/EEC
and subsequent amendments

1. CODES

RP515 4-wheel aluminum rollator with seat ATLANTE 1.0
RP538 4-wheel aluminum rollator with seat and footrest GAYA 1.0

2. INTRODUCTION

Thank you for choosing a MOPEDIA range rollators by Moretti S.p.A. MOPEDIA range rollators manufactured by Moretti S.p.A. are designed to satisfy all your requirements for practical, correct and safe use. This manual provides some useful suggestions for correctly using the device you have selected and offers valuable advice for your safety. We recommend reading the entire manual carefully before using the rollator you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

3. INTENDED USE

This rollator fit for people with movement problems. Can be used as assistance to facilitate ambulation or even servo-assisted ambulatory. Anyway, it can't be used by people with evident malfunction of the hand or the arm, even by people with balance problems. The rollator can be used inward and outward. We recommend using on flat surface where the wheels can reach the land. It can't be used on rock ground or on slanted area as the rollator can be unstable. The device is not intended to carry people or things.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. DECLARATION OF CONFORMITY EC

Moretti S.p.A. declare under sole responsibility that the product made and traded by Moretti S.p.A. and belonging to rollator range are in accordance with the European Directive on Medical devices (93/42/EEC). Moretti S.p.A. guarantees and declared under sole responsibility what follow:

1. The devices under discussion satisfy the essential requisites requested by the I° enclosure directive 93/42/CEE as laid down by the VII° enclosure of the above mentioned Directive.
2. The devices under discussion ARE NOT MEASURING INSTRUMENTS.
3. The devices under discussion ARE NOT MADE FOR CLINICAL PROBES.
4. The devices under discussion are traded with NON STERILE BOX.
5. The devices under discussion belong to class I°.
6. IT IS NOT ALLOWED to use and install devices except for the use provided by Moretti S.p.A. Moretti S.p.A. places technical documentation at relevant authorities in order to prove the conformity of 92/42/EEC device.
7. Moretti S.p.A maintains and provides for authorities, the technical documentation proving the accordance to European Directive on Medical devices (93/42/EEC).

5. WARNINGS

5.1 General warnings

- DO NOT install or use the device before reading and understanding this manual; If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional dealer or technical personnel before attempting to install this equipment to avoid injure or damage;
- The rollator should only be used by patients with a suitable motor capacity which ensures their use without risks;
- Advise with physician or therapist to determinate the correct adjustment and the correct use;
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly;
- The rollators are not intended to be used as self-propelled devices.

5.2 Warnings for use

1. Insure the brake cable to be correctly set;
2. Verify the correct operation of the brake after every height adjustment to avoid a free flexion of the cable;
3. Breaking this simple rules can cause personal damages;
4. The rollator can be used only on a hard, flat surface. Do not use the rollator to climb the stairs
5. Leaning always on both dumbbells to maintain balance;
6. Pay particular attention to wet, icy or snowy;
7. DO NOT overload the rollator;
8. DO NOT use the rollator seat (where present) to carry persons or property, the seat is for the user's session with the parking brake engaged;
9. All wheels must be in contact with the floor at any time during use to ensure the proper balancing of the rollator and maximum safety;
10. The brake must be engaged before using the seat.

Note When the rollator is used in steady position, the handbrake must be locked

11. DO NOT use the rollator to walk backwards, down gradients or to climb stairs, curbs, or to go over obstacles. Serious risk of fall or injury may occur.
12. The backrest is intended to provide back support while seated. The backrest is not intended to support full body weight. Avoid leaning while sitting as this may result in fall.
13. DO NOT sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.
14. DO NOT use the seat to transport people or objects.
15. When rising from a lower position, DO NOT use the rollator to provide assistance to the user. If the rollator is used to pull the user up to a standing position, the rollator may move even if the brakes are in the locked position and this could result in a fall.
16. DO NOT push the rollator while brakes are enabled. Brakes are to be used to keep the wheels from rolling. However, it may not prevent the rollator from moving if it is pushed.

5.3 Stability warnings

- The rollator can provide ambulatory assistance to an individual weighing up to 113 kg (Mod. RP515) and 130 kg (Mod. RP538), INCLUDING the loading bag/basket;
- The rollator bag/basket has a weight limitation of 4,5 kg.

5.4 Assembling warnings

- Verify that both handle support tubes are correctly inserted in the frame and secured with locking knobs locked into the holder with its main wing nuts and knobs;
- Both handle tubes are securely locked in the main support part with wing nuts and knobs;
- Do not hanging anything on the rollator frame to avoid distortions of the tubes when the rollator is open and damage the stability. Injury may also be caused by distortions;
- Take care to entrapment of fingers and / or clothing during the opening and closing.

6. BEFORE USE

Before using the device, always check:

1. That all parts of the rollator are locked in a secure manner.
2. Check that all screws are tight.
3. Make sure that the seat is properly locked
(if that 'not press further with the power seat bottom frame).
4. Check that the locking of the seat is correctly inserted.
5. Check the correct operation of the brake and the regular brake.

7. GENERAL DESCRIPTION

7.1 Parts description (Mod. RP515) Atlante 1.0

1. Rollator Frame (1 pc)
2. Backrest with Push-buttons (1 pc)
3. Handlebars (2pcs)
4. Bag/basket (1 pc)
5. Front Wheels (2pcs)
6. Rear Wheels (2pcs)
7. Star Knobs (2pcs)
8. Screws and Curved Washers (2pcs)

FIG. 1 (Picture with illustrative purpose only)

7.2 Parts description (Mod. RP538) Gaya 1.0

1. Rollator Frame (1 pc)
2. Backrest with Push-buttons (1 pc)
3. Handlebars (2pcs)
4. Bag/basket (1 pc)
5. Front Wheels (2pcs)
6. Rear Wheels (2pcs)
7. Star Knobs (2pcs)
8. Screws and Curved Washers (2pcs)

FIG. 2 (Picture with illustrative purpose only)

8. ASSEMBLING INSTRUCTION


8.1 Before assembling the product

Unpack the product and check that no damage has occurred during shipping. In the event when the rollator is damaged, please contact your nearest dealer.

8.2 Assembling

RP515

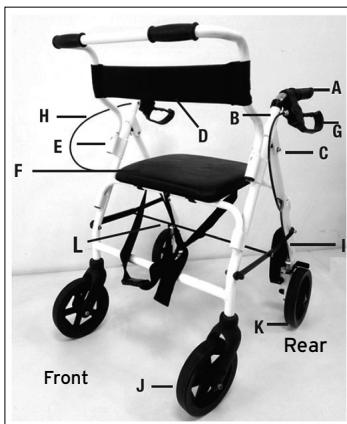RP538

FIG. 3 (Pictures with illustrative purpose only)

Assembled Rollator

- | | |
|---------------------------|------------------|
| A - Handgrips | G - Brake Lock |
| B - Adjustable Handlebars | H - Brake Cables |
| C - Star Knob | J - Front Wheels |
| D - Padded Backrest | K - Back Wheels |
| E - Backrest Push Buttons | L - Bag/basket |
| F - Padded Seat | |

1. Unfold the rollator by pushing down on the Folding Bracket (I) until the Frame is in full extension.
2. Insert the padded backrest (D) so that the padded portion faces the front of the unit. Push down on the backrest while depressing the the round plastic push button until it snaps into place.
3. Insert the handlebars (B) so that the handgrips (A) are aligned with the rear wheels. The brake cables should be on the outside of the handlebars and frame. Ensure excess brake cable loops towards the front of the rollator.

FIG. 4 (Picture with illustrative purpose only)

4. Adjust the height of the handlebars (B) for proper fit. To determine proper height, stand upright behind the rollator with shoulders relaxed, arms hanging at your side and eyes looking straight ahead. Standing in this position, the handgrips (A) of the rollator should be adjusted to a height even with the wrists. This will position the arms at an approximate 20° to 30° bend when using the rollator. Secure the handlebars at the closest hole to the desired height. Insert the screw from the inside of the rollator through the frame. Attach the washer so it fits snugly against the frame. Secure the star knob to the screw. See Figure 3. (Important: Hand Tighten Only!)

5. Place the bag/basket under the seat.

WARNING! Make sure that the knobs is located into the holes

WARNING!

Adjust the height of the rollator so that a user who is standing with his arms at his sides have the pulse height handles. This will ensure the positioning of the arms bent at about 20° -30° when using the rollator

WARNING! Use the bag/basket for small items only. Do not overload the bag/ basket to prevent imbalances and the risk of overturning (Max weight 4,5 kg)

8.2.1 Seat

Before sitting , place the rollator on a flat and solid surface and then lock the brakes. Make sure the latch of the lock is secured properly and that the rollator is in a stable position.

WARNING! Questions to sit down with your weight on the seat assembly. Do not sit by pushing only on one side.

WARNING! Make sure that the knobs is located into the holes.

Before use, always make sure that the rollator does not tend to close up under the influence a small lateral force.

Be careful not to twist the brake cables when connecting the handlebar and wheels.

9. HOW TO USE

- Start pushing when you are in an upright position with both hands on the handle;
- Make sure your hands are dry before using the rollator to avoid the risk of slipping;
- Walking maintaining an upright position with his eyes facing forward and not down;
- Do not push the rollator too far away from the body, this may cause falls and loads inadequate;
- Do not use the rollator to overcome steps higher than 20mm;
- When using outdoors, prefer the sidewalks properly prepared in accordance regulations for the removal of architectural barriers;

10. BRAKES

10.1 To brake

To brake the rollator pull the brake lever with your fingers upwards until the rollator slows down or stops. To continue to move the rollator, release the brake handles.

10.2 Lock the brake (handbrake)

Press down on the brake lever with the palm of your hand, when you hear a “click” wheels are locked. Lift the handle of the brake releasing it from the stop position.

WARNING! When the user is stationary, the brakes must be locked

WARNING! After making any adjustments to the rollator brakes, you must test the brakes before the rollator is used. See Maintenance Instructions.

11. CARE AND MAINTANANCE

The devices **MOPEDIA** by Moretti SpA on the market are carefully checked and carry the CE mark. For the safety of the patient and the doctor is recommended to be checked by the manufacturer or by an approved laboratory, at least every 2 years, the suitability for use of your rollator. If repairs are to be used only original spare parts and accessories.

The useful life of the product depends on the use, maintenance, storage and cleaning.

12. CLEANING

To clean parts, use only a damp cloth with a neutral detergent, not use cleaners containing solvents and metal brushes or abrasive or sharp objects, may damage the product.

13. WHEEL CARE

1. Clean the wheels with warm water and mild detergent. Wipe dry with a clean cloth.
2. Periodically lubricate the wheels.

14. TECHNICAL FEATURES

Measures in cm*

Series	ATLANTE 1.0					GAYA 1.0			
Product codes	RP515					RP538			
Frame	Aluminum								
Weight and dimensions									
									
RP515	59,5	78-89,5	54	61	47,5	6,5 Kg	15	15	113 Kg
RP538	59	91	56,5	62	48	8,5 Kg	20	20	130 Kg

*The dimension could be change ± 1 cm

15. SPARE PARTS / ACCESSORIES

For spare parts and accessories please refer to the Moretti S.p.A. master catalogue

16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use.

Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover from warranty: A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

Non-defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

16.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months as from the receiving date.

16.3 Exempt clauses

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be fixed by according to the law is limited by the warranty period, in the limits of law. Some states or countries don't allow to limit the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user.

The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Sold to _____

Street _____ Place _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com