MOPEDIA®

MA RS846_RS847 01 A_01-2018

SEDIE DA DOCCIA

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ITALIANO

INDICE

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CE Class I Medical Equipment according Directive 93/42/EEC and subsequent amendment

1. CODES

RS846 - Shower chair with four castors

RS847 - Self propelled shower chair

2. INTRODUCTION

Thank you for choosing a MOPEDIA Shower chair by Moretti S.p.A. MOPEDIA Shower chair manufactured by Moretti S.p.A. are designed to satisfy all your requirements for a practical, correct and safe use. This manual provides some useful suggestions to correctly use the device you have selected and offers valuable advices for your safety. We recommend carefully reading the entire manual before using the hospital bed you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

NOTE: Check all parts of the product have not been damaged during shipment. In case of damage do not use the product and contact your dealer for further instructions.

CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.

3. INTENDED USE

The MOPEDIA shower and toilet chairs are created for people with movement difficulty and can be used indoors as substitution of toilet or as aid under the shower.

4. EC DECLARATION OF CONFORMITY

Moretti S.P.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to Hospital beds range are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/ EEC and further modifications. For this purpose Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive. The full list of the devices at issue are specified in Annex A in declaration.
2. The devices in question ARE NOT MEASUREMENT DEVICES.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I.
6. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC and further modifications to the proper authorities.

5. GENERAL WARNINGS

WARNINGS!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.

- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts cannot be repaired and / or replaced.
- Be careful when children are nearby.

6. SYMBOLS

	Product code
	CE Mark
	Manufacturer
	Month and year of manufacture
	Read the manual for instructions
	Conditions of disposal

7. GENERAL DESCRIPTION

7.1 View and list of Components

Fig.1

Fig.2

RS846 Shower chair with four small chair (Fig.1)

1. Frame - 1 pc
2. Backrest - 1pc
3. Knob - 2pcs
4. Footrest - 2pcs
5. Castors - 4pcs
6. Bucket toilet WC- 1pc

RS847 Self propelled shower chair (Fig.2)

1. Frame - 1 pc
2. Backrest - 2pcs
3. Knob - 2pcs
4. Footrest - 2pcs
5. Self propelled wheels - 2pcs
6. Castors - 2pcs
7. Bucket toilet WC- 1pc

8. ASSEMBLING/DISASSEMBLING

8.1 Assembling RS846 - RS847

You will receive the MOPEDIA wheelchair with disassembled backrest and footrest.

A. Remove the product from its packaging and check that contents comply with the illustrated component description.

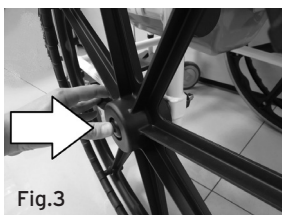
Fig.3

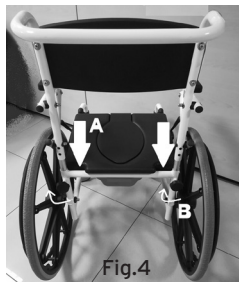
Fig.4

B. Insert the Wheel by using the quick release pins pushing the button (Fig 3). **ONLY RS847**

C. To assemble the backrest, insert the ends of the tubes in the appropriate holes "A".

Screw the knob "B" to fix the backrest as in (Fig 4).

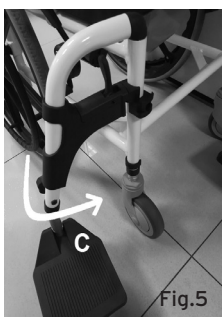
Fig.5

D. To assemble the footrest, insert it in the hole, push it downwards and then inwards until the safety block. To remove the footrest, unlock the safety block, turn the footrest outwards and pull it out. Footrests have to be turned inward. (Fig 5)

E. Place the toilet bucket WC under the seat thanks the grooves.

F. Now your device is ready to use.

WARNINGS!

- Correctly identifying which platform to use. The footrests should always be turned inward.
- Insert the WC bowl from the back of the chair, making sure to insert it correctly in the grooves provided.

8.2 Disassembling

Follow the procedure described above in reverse.

9. BEFORE TO USE

- Always check the status of the mechanical parts in order to guarantee they are used in total safety for persons and environment.
- Check the proper assembly of the device with attention to the correct insertion and locking screws and nuts of the backrest and footrest.

10. PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity for any reason.
- Assist always the patient during the handling operations.
- Don't skew the chair more than 10% without assistance.
- Block the wheels during patient transfer.
- Wheelchair has to be in a stable surface and wheels locked during user transfer.
- Before leaning or get up, be sure casters are in straight position and rear castors are blocked in order to avoid a turnover.

- Do not lean out of the backrest to avoid a turnover.
- Pay attention to your fingers during assembling, regulations or simply during the use.
- When using the self-propelled wheel, pay attention to your fingers.
- Keep the chair far from fire.
- Use always original accessories and spare parts.

11. HOW TO USE

Use only the back handle to move the chair, never use other supports. Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. Assistance is recommended when in slope higher than 10%. If you need to stop in a sloping surface, avoid abrupt movements. During the downhill, do not change suddenly the direction. Lift the chair up if necessary, using the main frame and not armrests or footrests.

WARNING! The patient must not put pressure on footrests when go up and down from the chair.

11.1 Brakes (RS847) You can brake the chair pushing the lever on each wheel. If you need to adjust the brake lever, you can unscrew the nut, move the clamp at the required position and then tighten again.

11.2 WC

Use the toilet when the wheelchair is completely blocked. To use the toilet take out the removable part of the seat using the handle on the front part. Take out the toilet using the handle.

11.3 Armrest

To facilitate the transport of the chair and the transfer of the patient, all the armrests are completely foldable.

WARNING! Never use the armrest to lift or move the bath chair.

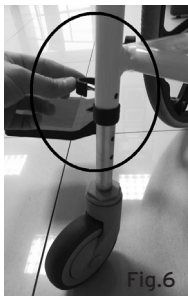
Fig.6

11.4 Footrest adjustment

The footrests of shower chair can be adjusted according to your needs: unfasten the screw, adjust the footrest height and fasten the screw again. Check that both footrests are adjusted at the same height.

11.5 Seat height adjustment

The seat height can be adjusted from 45cm to 52.5cm, with 3 different levels. Remove the safety clip, change the height and insert the clip again.

N.B.: Check the stability of the chair before the use: all the safety clip have to be in the correct position.

12. MAINTENANCE

The MOPEDIA by MORETTI S.p.A. chairs need minimal maintenance and general care, but we suggest you to control regularly:

- Braking system: monthly
- Cleaning parts (seat, backrest, armpad, wheels and so on) if necessary
- Cleaning the upholstery of the seat as required.
- General check by qualified personnel every two years.

It is extremely important that the wheelchair is checked at the first use. In particular, please

check that all screws and nuts are tight.

13. CLEANING AND SANITISATION

13.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

WARNING! Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

WARNING! Sanitising agents are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective glasses and safety shoes. In case of accidental contact, rinse the affected part thoroughly using warm water and consult a doctor immediately.

14. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

15. TECHNICAL FEATURES

15.1 Dimension and weight

Dimension* and weight												
Measures in CM												
RS846	85	40	92,5÷100	70÷77,5	48,5÷56	46÷52,5	44	42	55	12,5	12,5	12,5
RS847	102		92,5						63	17,5	60	12,5

*The dimension could be change ± 0.5 cm

15.2 Technical specifications

- Powder coated aluminum frame
- PU seat, armrest and back
- Folding armrests
- Height adjustable seat in 3 positions.
- Removable WC bucket with cover
- Wheels 60cm with quick release pin * RS846
- Max User weight 113Kg

16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the

possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17. REPAIRING

-Warranty repair-

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

-Repair not covered by warranty-

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

-Non defective device-

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

18. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

19. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com