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Rollator Bellator.

Destinazione d'uso

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Rollator Bellator.

Use

The walkers and rollators are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance. This aid is designed for a single user, in indoor or outdoor environments according on models.

Code	Length	Inside Width	Width	Height	Seat Dimensions	Floor to Seat Height	Weight Capacity	Front Wheel	Rear Wheel	Net Weight	Carton Size
01.06.410	72.1	45.7	64	82/92	45.7x25	48	120	20 (8")	20 (8")	7	75x27x85

Features	
Folding aluminum frame with seat and back	
Four wheels with a diameter of 20 cm (swivel front wheels, two fixed rear wheels), there is a brake device on the rear wheels	
Nylon storage basket included	
Ergonomic and height-adjustable knobs	
Height of the seat from the ground 48 cm	
Materials	
Frame	Aluminum
Wheels	PVC
Seat	Nylon
Frame Maintenance and Cleaning	
Use a damp cloth with mild soap	
Do not use water and / or steam jet washing machines, abrasive objects and metal brushes	
Must not use acid, alkaline or solvent products such as acetone	
Wheels Maintenance and Cleaning	
Clean the wheels with warm water and neutral detergent	
Lubricate the wheels periodically	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	12.06.09.003
CND Code	Y120609
RDM Code	1734371/R