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Compressore Prestige.

Destinazione d'uso

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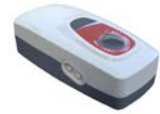
Prestige Pump.

Use

The Prestige pump has to be combined with: bubble air mattress BP2 (code 05.03.BP2), bubble air mattress BP4 (code 05.03.BP4) and Supercare mattress T01 (code 05.03.T01). The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Features	
Code	05.03.6600A4
Dimensions	24 x 11.5 x 9 cm
Case	Fire Retardant ABS
Air Mode	Alternate (A-b-A-b)
Air Output	7 LPM
Pressure Range	50 to 120 mm Hg
Noise Level	≤20 Dba
Cycle Time	12 min
Power	220/240 V-50 Hz
Power Consumption	7 Watt
Electric Cable Length	3 m
Weight	0.8 Kg
Fuse	1.5 A
Other Features	Two steel hooks for anchoring to the bed
Warnings	
The pump can be used only with mattresses recommended by the manufacturer	
Check the correct assembly of the device with particular attention to the connection of the tubes between the pump and the mattress	
Maintenance and Cleaning	
Clean the pump with a damp cloth and neutral detergent	
WARNING: never immerse the compressor in any type of liquid	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	04.33.06.018
CND Code	Y033306
RDM code	1522221/R