DEMARTA
VIRGINIO

MANUALE D'USO
BARELLA A CORSETTO
COD. 01.01.1101

CE

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USER MANUAL
CORSET STRETCHER
SKU 01.01.1101

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

The sling are associated with a lift of the SCANMEDICALEUROPE Line for patient handling.


DESCRIPTION

The CORSET STRETCHER has a mesh sling with perineal hole and 8 adjustable bindings. The sling bar is made of Inox steel.

TECHNICAL FEATURES

Code	Dimensions (cm)	Capacity (Kg)
01.01.1101	170 x 55	160

USE

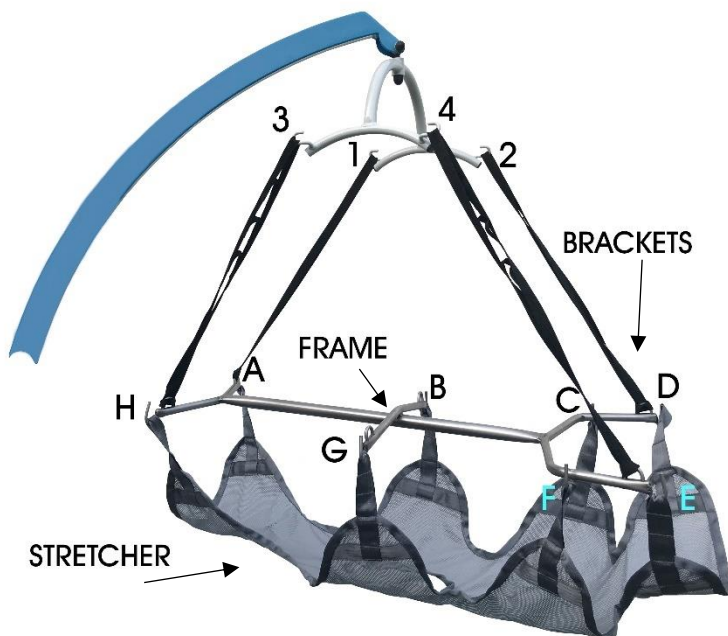Hook the stretcher to the frame by inserting the hooks into the points A-B-C-D-E-F-G-H.

A-H in the picture are the feet, the perineal notch on the stretcher must be positioned on this side.


Points C-D-E-F are the housing for the patient's head.


Important! Check that all points A-B-C-D-E-F-G-H are hooked as shown in the pictures.

At the ends of the frame, in points A-D-E-H, there are "u" hooks. To these must be anchored the black brackets with a cow hitch.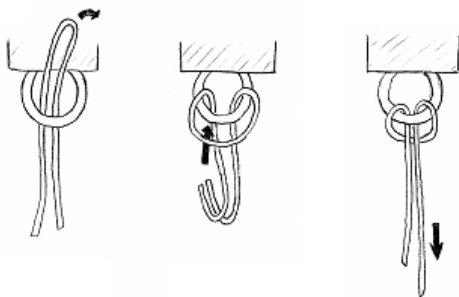

The ends of the brackets must be hooked into the sling bar of the lift used.

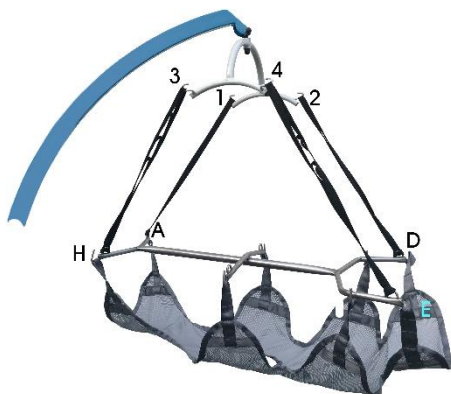- 1 with hook A
- 2 with hook D
- 3 with hook H
- 4 with hook E

CLEANING

Sling Maintenance and Cleaning	
	Water Temperature not Above 40°C or 105°F
	Do not Iron
	Do not Bleach
	Do not Tumble Dry

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.

Worn components are not covered by this warranty if the damage is due to normal use of the product.