COD. 01.06.650S

COD. 01.06.650A

COD. 01.06.650T

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SKU 01.06.650A

SKU 01.06.650T

USER MANUAL FOLDABLE FOREARM WALKER

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

The walker are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance. This aid is designed for a single user, in indoor environments.

SPECIFIC WARNINGS

- The walker must be used only by patients with a suitable motor ability that guarantees their use without risks.
- Consult your doctor or therapist to determine the suitability of the product, the right adjustment and the correct use of the walker.
- Your doctor or therapist should control the height-adjustment of the walker, the correct activation of the brake, the correct assembly in general.
- Failure to follow these simple warnings may result in personal injury.
- The walker can only be used on a solid, flat surface.
- Do not use the walker to climb stairs.
- Do not use the walker seat to transport people or things. The seat is intended for the user's seat with the brake on.
- All the wheels must be in contact with the floor at any time during use to guarantee the correct balance.
- Do not hang anything on the walker frame to avoid distortion of it.
- Before using the device, always check:
 1. That all parts of the walker are securely locked.
 2. Check that all screws are tight.
 3. Make sure the seat is properly placed.
 4. Check for proper brake operation.

DESCRIPTION

- Foldable height-adjustable steel structure. The two knobs are adjustable in width and inclination.
- Four swivel wheels, two rear with stop brake.
- Axillary support (code 01.06.650S).
- Forearm support (code 01.06.650 A).
- Forearm support with table (code 01.06.650T). The two knobs are adjustable in width and inclination.

TECHNICAL FEATURES

Code	Length	Inside Width	Width	Height	Seat Dimension	Floor to Seat Height	Floor to Support Bar Height	Weight Capacity	Wheel Dimension	Net Weight	Carton Size
Measures in Cm											
01.06.650S	71	46	50.5	113/160	52x21	46/58.5	97.5/110	130	10 (4")	13	78x15x100
01.06.650A	71	46	50.5	100/144	52x21	46/58.5	97.5/110	130	10 (4")	13	78x15x100
01.06.650T	71	46	50.5	101/147	52x21	46/58.5	97.5/110	130	10 (4")	13	78x15x100

ASSEMBLY

The frame of the rollator is divided into 4 parts:

SKU 01.06.650S

SKU 01.06.650A

Base frame assembly:

Open the frame (1) of the walker by pushing downord the X-shaped support placed on the front. Insert the U-shaped support (2) in the frame (1), everything will be fixed by two spring-loaded buttons.

Fix the forearm support:

Insert part nr. 3-4 to the newly assembled frame (1 + 2).

WARNING!

Tightly screw the two side handles, this will also allow the height adjustment of the forearm support.

The frame of the rollator is divided into 3 parts:

SKU 01.06.650T

Base frame assembly:

Open the frame (1) of the walker by pushing downord the X-shaped support placed on the front. Insert the U-shaped support (2) in the frame (1), everything will be fixed by two spring-loaded buttons.


Fix the forearm support:

Insert part nr. 3 to the newly assembled frame (1+2).

WARNING!

Tightly screw the two side handles, this will also allow the height adjustment of the forearm support.

Closing the frame:

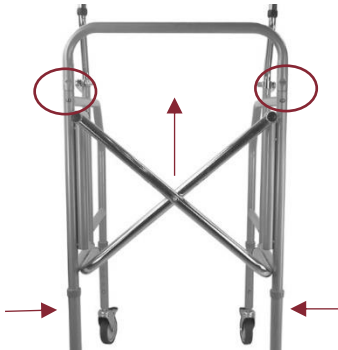Crush the two spring-loaded buttons and push up the U-shaped support, Act on the X placed on the frame anteriorly appro aching the two side parts of the frame.

USE

- Start pushing when you are in an upright position and both hands on the handle (see figure below).

- Make sure you have dry hands before using the walker to avoid the risk of slipping.
- Walk keeping your position upright with your eyes facing forward and not down.
- Do not use the walker to go over steps.

CLEANING

Use a damp cloth with mild soap, the water temperature not above 30°C. Dry well before use. If necessary, disinfect the product using a common disinfectant detergent. Do not use water and/or steam jet washing machines. Must not use acid, alkaline or solvent products such as acetone.

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems. It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.