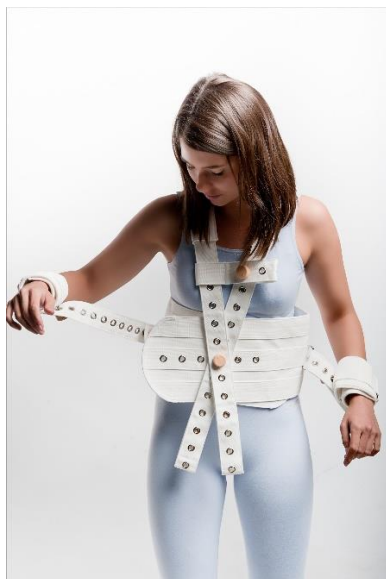
# DEMARTA VIRGINIO

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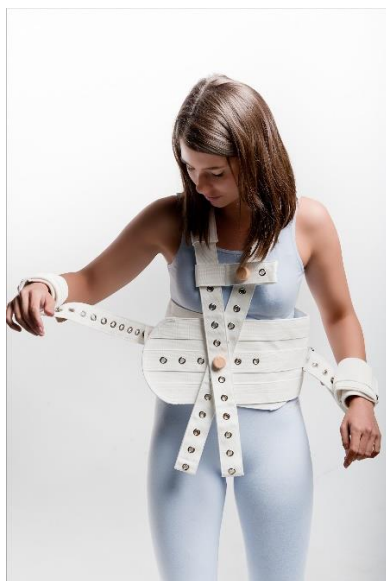
## USER MANUAL PSYCHIATRIC FIXING BELT

I Class Medical Device according  
to directive UE 2017/745

## **INTRODUCTION**

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

## **GENERAL WARNINGS**

1. Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
2. Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
3. Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
4. DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
5. Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

## **DECLARATION OF CONFORMITY**

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

## **INTENDED USE**

The ADIGGY Line includes all the aids used to immobilize the patient in a wheelchair, bed, etc. They are used to ensure maximum safety of those who use them to prevent falls or traumas of various kinds.

## **WARNINGS**

- Restraint systems must be used by qualified healthcare personnel.
- If used at home, you should ask your doctor or therapist for advices.
- Patients with restraint products require frequent monitoring.

## DESCRIPTION

The FIXING BELT is made from an abdominal belt with straps and cuffs and two padded shoulder straps. It is worn to avoid inappropriate movements by the user. This product must be used under the supervision of qualified personnel.

### Magnetic closure system:

The belt is locked with a magnetic closure:

#### 01.12.MAD55 – Closure for Magnetic Belt

It consists of two parts:

- the base with the pin that is passed through the special openings on the buckles.
- the button cap that closes everything by fixing in a magnetic way.

#### 01.12.LAD55 – Key for Magnetic Belt

The MAGNETIC KEY is used to open the button cap of the MAGNETIC BLOCK

**IMPORTANT:** Do not immerse the magnetic closure or magnetic key in liquid, humidity can cause serious damage to the locking system.

IMPORTANT: Patients with pacemakers should keep the lock and magnetic key at a distance of no less than 10 cm from the pacemaker to avoid the appearance of tachycardia.

## TECHNICAL FEATURES

Code	Size
01.12.AD55	One

Materials	
Belt	Polyester
Maintenance and Cleaning	
	Water temperature not above 60° C
	Do not iron
	Do not bleach
	Do not dry in dryer

## GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems. It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

## WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.