MANUALE D'USO
POLTRONA COCOON

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USER MANUAL
COCOON LIFT CHAIRS

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

Devicare lift chairs are indicated to facilitate daily difficulties in elderly and disabled people. The main help function is the lift-mode, which gives the possibility to reach the upright position in safety. The frame allows the backrest to be tilted and the footrests to be raised, the movements can take place independently or synchronously

GENERAL WARNINGS

- For correct use of the device, refer carefully to the following manual.
- For correct use of the device always consult your doctor or therapist.
- Always pay close attention to the presence of moving parts that could cause limb entrapment and personal injury.
- Keep the packed product away from any source of heat as the packaging is made of cardboard.
- Always pay close attention to the presence of children.
- Immediately disconnect the electric cable from the mains if operating irregularities are found and check for any electrical leaks.
- Do not unplug or insert the socket with wet hands.
- The lift chair must be mounted on a horizontal surface. Avoid loose bumpy plans.

DESCRIPTION

Features
Electric model remote control
Lift-mode armchair
Backrest, seat padded with silicone hollow fiber, removable covers
Removable outer cover
Retractable armrests
Painted steel structure with
Model with two engines: independent inclination of the backrest and the foot-raisers. The frame also rises to reach the upright position
Standar Model Maximum Load: 160 Kg, XL Model Maximum Load: 220 Kg

Brown

Anthracite

Raspberry

Grey

Code	Color	Material
01.11.Z2-VME	Brown	Polyester Velvet
01.11.Z2-PAE	Anthracite	Leather
01.11.Z2-PLE	Raspberry	Leather
01.11.Z2-TGE	Grey	Polyester Fabric

TECHNICAL FEATURES

Dimensions	
Seat height from the ground	52 cm
Seat width	44 cm, 50 cm
Seat depth	45 cm, 55 cm

ASSEMBLY

1. Remove the chair from the packaging;
2. Fit the seatback to the base by sliding it down along the 2 metallic rails until a perfect engagement
3. of both parts, until the 'click sound';
4. Insert each left and right armrest rod into the opening intended for this purpose of the left and right
5. armrest boards;
6. Plug the transformer to the electric actuator's cable;
7. Plug the electric cable of the jack to the power transformer on the other end. Please check that all
8. electrical connexion are in the correct place;
9. Plug in the power supply cable to an electricity network in compliance with the current standards and matching the 220V–power-supply-unit voltage.


IMPORTANT: Check that the voltage of the power outlet is identical to that shown on the engine label.

IMPORTANT: Even minor electrical interventions require the work of professionally qualified personnel.

Place the lift chair in large places, away from heat sources (fireplaces, stoves etc.) and on flat surfaces. Make sure that the movement of the footboard and the backrest is not hindered in any way. The lift chair must have a distance of about 50 cm from any obstacle (wall, table, etc.). The lift chair is supplied already assembled and ready for use; the user only has to connect it to the electricity grid.

Assembly the magazine holder:

- Open the armrest,
- Put the magazine holder in the armrest board by taking notice to not trouble the closing of the armrest (see photo below).

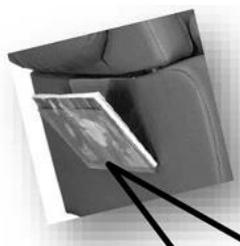Put your magazine holder in the middle of the armrest board in order to not close the notch in front of the armrest board

TELECONDI

1


2

1-STANDARD MODEL

2-XL MODEL

TOP Button : Reset in seated position with lowering of the leg rest, then after move to the rising up position.
BOTTOM Button : Backward tilt with rising of the leg rest.

COCCON 2 MODEL

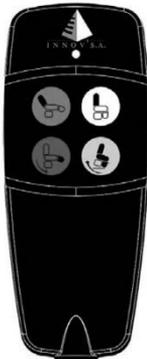1

1 - GREEN button : Back reclining

2

2 - WHITE button : back rising in straight position

3

3 - BLUE button : Rising of the leg rest.
Return in sitting position

4


4 - YELLOW button : Downgrading of the leg rest.
Switching in lifting position.

NOTES: When operating the controls, you must be able to view the surrounding area in order to prevent people, animals or things from remaining under the moving parts.

NOTES: Do not lean or apply force downwards when the backrest is tilted. Risk of tipping over and injury.

NOTES: The lift chairs is not designed to move the user.

Move the lift chairs only when nobody is on it.

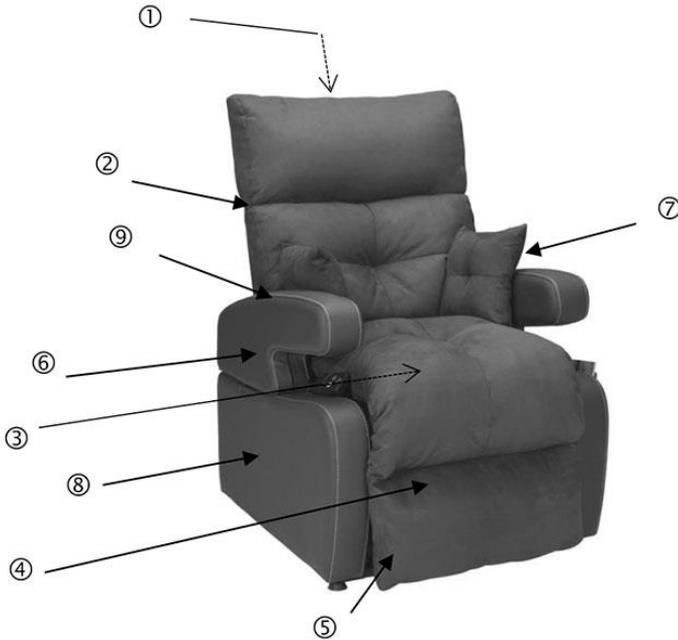- 1 – Seatback
- 2 – Seatback quilt washable/replaceable
- 3 – Seat base
- 4 – Seat base/leg rest quilt washable/replaceable
- 5 – Leg rest
- 6 – Two armrests retractable/removable
- 7 – Comfort cushion reducer seat
- 8 – Two armrests boards
- 9 – 2-buttons remote-control reversible
left/right – Stop Start function

WARNINGS FOR USE

- Do not exceed the maximum permitted load (130 kg).
- Always assist the user during handling operations.
- Connect the electric cable directly to the socket. Do not use an extension cord.
- Before sitting on the lift chair, make sure that the remote control and its wire are not in the way.
- Move your hands and feet away from the attachment points such as the chair and footboard lifting mechanisms.
- When not in use, leave the lift chair in an upright position with the footboard closed.
- If the device does not work or malfunctions, immediately stop use, switch off the device, unplug it and contact the authorized dealer or the Service Center for repair.

- When the device is not used, make sure that the remote control is correctly placed in the appropriate pocket.
- When removing the plug from the socket, never pull on the cable, but take it directly to the plug near the socket.
- Do not use when wearing wet clothing.
- Do not use when the lift chair when is wet.
- Do not use with voltage other than that indicated on the label, also do not enter the plug into multiple sockets.
- Do not use in an environment with a high percentage of humidity such as in bathrooms or outdoors.
- Do not use in case of damage to the electric cable, plug and socket.
- Do not damage, break, modify, bend, pull, twist, wrap the electric cable during use.
- Do not place the power supply cable near heat sources or hot surfaces.

MAINTENANCE

Checks periodically:

That all screws and bolts are tight.

The mechanical parts (hinges and pins) must be checked monthly and lubricated with a drop of oil in order to promote smoothness and wear resistance.

CLEANING

Seat back quilt:

- Remove the Velcro strap from the flap of the cover behind the seat back
- Zip down the left and the right side on the quilt

Seat base/leg rest quilt:

Remove the Velcro straps at the bottom, on the right and on the left of the seat base.

All modules can be machine-washed at 30° C
--

We advise you to put the modules in a washing net or in a pillowcase, to preserve their original aspect.

Changing the seat back:

It is possible to modify the seat base depth by replacing the back rest with another one.

- Drawn on the 'CLIP' and remove the seat back by sliding it away from its base along the 2 metallic rails;
- Assembly the new seat back to the base by sliding it down along the 2 metallic rails until a perfect engagement of both parts, until the 'CLIP'.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, because these substances cause the abrasion of the components of the lift chair.

GENERAL DISPOSAL CONDITIONS

In case of disposal of the device, never use the normal municipal solid waste disposal systems. It is recommended to dispose of the device through the common municipal ecological islands for the envisaged recycling operations of the materials used.

TREATMENT OF EXHAUSTED BATTERIES (DIRECTIVE 2006/66 / EC) BATTERY REGISTRATION NUMBER IT21040P00006953

The device contains a non-removable battery and must not be disposed of as urban waste, as it is subject to separate collection to avoid damage to the environment. In compliance with the directives 2002/96, 2006/66 and the national transposition laws for the disposal of the product at the end of its life.

WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU.

AEE REGISTRATION NUMBER IT21050000012993

At the end of its useful life, the product must not be disposed of with municipal waste. It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.