MANUALE D'USO
SOLLEVATORE EVOLUTION
REF: 01.01.EVO

ASSEMBLY INSTRUCTIONS VIDEO

WATCH ME!

TECHNICAL FEATURES VIDEO

WATCH ME!

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**MANUAL
EVOLUTION LIFT
REF: 01.01.EVO**

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO S.r.l..
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO S.r.l. declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO S.r.l. in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO S.r.l. declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO S.r.l. comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO S.r.l. guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

SAFETY

Passive lifts allow the lifting from bed / wheelchair / stretcher of patients with important functional limitations. Below are safety tips.

It is advisable to check the lift in all its parts before use.

- Check the connection between the remote control and the battery.
- Check that the battery is charged.
- Check the harness for any signs of wear or damage.
- Check the brake seal.

Maximum load: 180 kg.

Warning! Do not exceed the maximum range.

Warning! Always keep the lift on a level surface.


Warning! Do not attempt to cross passages, steps or other obstacles.

Warning! The battery should only be assembled and repaired by qualified personnel.

ELECTROMAGNETISM

Radio Wave has influence on the hoist. Your hoist will be affected by your mobile phone, radio, radar, etc. Please kindly pay attention.

TECHNICAL FEATURES

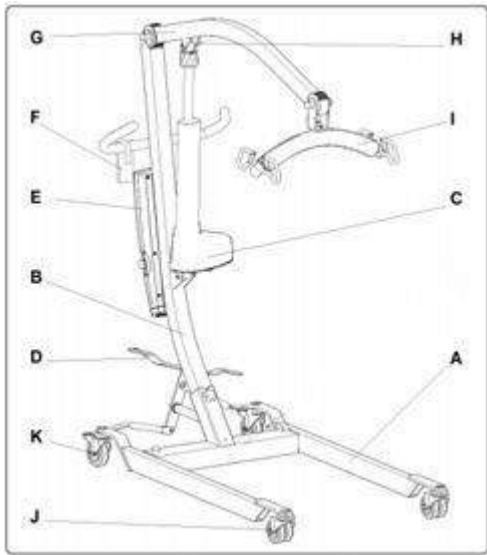- A- Basic lift
- B- Supporting column
- C- Actuator
- D- Base opening lever
- E- Control unit
- F- Remote control
- G- Lifting arm
- H- Emergency manual descent
- I- Lifting beam
- J- Front wheels
- K- Rear wheels with brake

Code	Length	Width	Internal Base Width	External Base Width	Minimum Height at Sling Bar	Maximum Height at Sling Bar	Base Height	Front Wheels	Rear Wheels	Weight Capacity	Weight	Carton Size
Measures in Cm												
01.01.EVO	120	60	46/90	57/102	60	175	10	7,5 (3")	10 (4")	180	38	115x58x33

ASSEMBLY

1) Loading column insertion.

Take the lift base (A) and lift mast (B) out of the box. Align and insert the lift mast into the lift base.

Insert the pin and screw the locking handle.

Attention! Check that everything is secure.

2) Assembling the lifting arm.

Extract the lifting arm from the box (G), align it with the appropriate holes in the supporting column (B), insert and tighten the fixing screw with the Allen key.


3) Actuator assembly.

Insert the actuator (C) between the lift arm (I) and the lift mast (B).

Then, after having taken the two snap closures, insert the pin of one of them into the special hole located in the upper part of the actuator and block it with the special snap closure. Repeat the same operation to block the lower part of the actuator.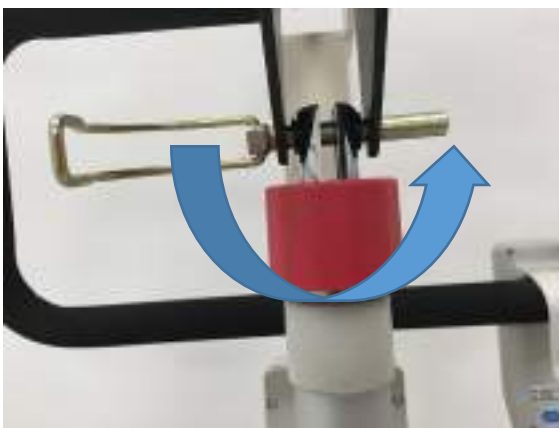

4) Assembly of handles.

Take the padded handles out of the box. Then take the supplied screws and insert the washers. Align the holes in the handles with those in the supporting column (B) and insert the screws, tightening them with the Allen key provided.

Attention! The handles must face towards the inside of the lift.

5) Insertion of the base adjustment lever.

Take the base adjustment lever out of the box (D). Next, take the screw provided and align the hole in the lever with the hole in the lifter base (A). Insert the screw by tightening it with the Allen key provided.

6) Assembling the control unit.

Fix the control unit (E) to the supporting column (B), making it slide, from top to bottom, on the metal hooking bracket present on the supporting column. The movement is fluid and does not require blocks / hooks.

Attention! Get to the end of the race.

7) Inserting the battery.

Insert the battery in the appropriate housing provided with the control unit (E).

8) Connection of actuator and remote control cables to the control unit.

- A) Connect the 4-pole plug to socket n°2
- B) Connect the 8-pole plug to socket n°1

A)

B)

Attention! Be sure to insert the plugs all the way in.

9) Harness hooking.

1. Hook the loops of the head section into the outer hooks.
2. Hook the loops of the central area into the internal hooks.
3. Hook the loops of the leg section into the internal hooks.

1

2


3

WARNINGS AND SAFETY

1) Emergency manual descent:

When the boom is raised, turn the red knob (H) clockwise to activate the manual emergency descent. Proceed with the reverse operation to raise the lifting arm manually, i.e. turn the red knob anticlockwise.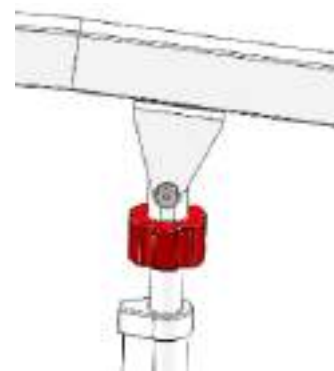

2) Red mushroom key

Press the red mushroom button when the lift is not in use, in order to deactivate any electric operation.

To deactivate the red mushroom button and thus restore electrical operation, turn it clockwise.

Attention! Make sure the red mushroom button is deactivated before use.

3) Charging the battery.

Take the battery charger supplied and insert the plug into the appropriate socket located on the side of the control unit. Once inserted, connect the black transformer to the socket.

Battery status

Red led on: battery charging.

Green led on: battery charged.

Attention! Before each use, make sure the battery is charged.

Do not charge the battery in a humid environment.

Do not use the lifter while charging.

4) Rear wheels with locked brake.

The rear wheels of the lift can be locked. The rear brake is pressed down to lock the wheels. To release the wheels, push the brake back up to the horizontal position.

Brake Released

Locked Brake

Attention! Check that the brake is at the end of its travel to be sure of the locking of the wheels.

5) Basic width adjustment with lever.

The width of the base is adjusted by lever.

Lever to the right, base closed

Lever in the centre, base open 50%

Lever to the left, base fully open

6) Patient transport.

Once the patient is in the harness, raise the arm with the remote control (F):

- Up arrow to raise the lift arm.
- Arrow down to lower the lift arm.

Attention! On the remote control there are two illuminated buttons for Checking the weight (1) and the battery charge level (2).

1) Red led: the range has been exceeded

2) Red led: battery almost flat

- Use the push handle to move the lift.
- Transport the patient with the base open to ensure greater stability.
- Always face the patient forward during transport.
- Block the rear wheels at each stop during transport.
- Always check that the straps are stretched out and inserted into the hook of the sling bar before lifting a patient.
- Never leave the patient alone when he is positioned on the sling.
- Avoid sudden movements.

Optional: Cordless Charger 2.

Insert the battery into the cordless charger to check the battery percentage. Connect the cordless charger to the mains using the supplied cable and turn it on using the black button on the side. This operation will start the battery charging phase.

MAINTENANCE

This product is a mature hoist that requires regular maintenance. Maintenance is very important, and if you follow the instructions in this section for maintenance checking, you can use this product for years with ease. You can do some routine checking by yourself, while others are assisted by dealers. If any problems with the maintenance or operation of this product, please contact the dealer of this product. Warning! This product, like most electrical or mechanical equipment, is susceptible to the natural environment. In any case, it should be avoided the humid environment. Directly contacting with water or humid environment may cause malfunction of the electrical equipment or mechanical equipment of the hoist. Water may cause corrosion of the electronic components and the frame may rust!

If the product is exposed to water, please perform the following maintenance:

1. Dry the water as much as possible with a rag and place it in a dry place;
2. Perform all safety and operational tests prior to use;
3. If there is any abnormality, please contact the dealer of this product.

GENERAL GUIDANCE

1. Avoid knocking or hitting the electric control box, especially the controller handle;
2. To avoid prolonged exposure to harsh conditions, such as overheating, too cold or humid environment and keep the electric control box clean;
3. Check all electrical connections to ensure they are securely connected and secure;
4. When the lamp of the battery status table is fully illuminated, the battery is fully charged, and the controller and electrical system are working normally;
5. If there is only one space on battery status table, it indicates that the battery is low and needs to be charged, but the electrical system works normally;
6. Check all cable connections to make sure they are securely connected and free of corrosion;
7. All wheel bearings are lubricated and sealed. They do not require the addition of lubricating oil.

PRODUCT INSPECTION

Daily Inspection :

1. Visually inspect the lift for external damage or wear;
2. Check if the screws and nuts on the boom, electric push rod and base are tight;
3. Check the hooks and slings for damage;
4. Check if the manual control, electric push rod and width adjustment function are normal.
5. Check the emergency descent function. If you have any questions, please contact your dealer;

Weekly Inspection :

Disconnect the manual connector and the electric push rod connector from the electric control box, check the connection and check for corrosion. If necessary, please contact the dealer;

Ensure that all screws are securely in place and that there are no breaks or bends;

Check if the brake function is normal.

Monthly inspection:

1. Check the wear of the universal wheel. If maintenance is required, please contact your dealer.
2. Check the four wheels for wear and looseness, which may indicate that adjustments are needed or the bearings need to be replaced. Please contact the dealer for repair or replacement;
3. Keep the product clean and free of debris, such as hair, food, beverages, residues, etc.

Other related circumstances

Temperature (some parts of the lift are susceptible to temperature)

Charger: At extremely low temperatures, the battery may freeze. There are many factors that may cause freezing at special temperatures, such as charger type, usage, and battery composition (such as sealed lead-acid batteries or gel batteries);

High temperatures may slow the lift. The safety of the electrical box protects the motor and other electronic components from damage.

STORAGE

This product should be stored in a dry environment with a storage temperature of $>0^{\circ}$ and a relative humidity of $\leq 60\%$. Do not place it at the extreme temperature. Remove the battery during storage and understand the structure. If it cannot be stored under the above conditions, it may cause rust of the lift and damage to the electrical system.

CLEANING

Never rinse the product with water or contact it directly with water.

This product can be easily wiped clean with a damp cloth. Never use chemicals to clean the sling. This may cause corrosion and tearing of the sling. Clean it with a wet towel and a mild soap solution until it is completely dry.

FAULTS AND REMEDIES

The following table shows common faults and corresponding solutions. If you still can't solve it, please contact an authorized company.

Faults	Remedies
Do not work	Check if the emergency stop button is pressed Check if the wires of the manual or electric push rod are connected correctly. Check if the battery is seriously low. Check the battery contact panel for problems or damage. Find an authorized supplier
Can not Charge	Check if the charger is plugged into the socket Check the battery contact panel for problems or damage. Find an authorized supplier of DEMARTA
Suddenly stop during lifting	Check if the emergency stop button is pressed Use emergency descent device Find an authorized supplier

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

TREATMENT OF EXHAUSTED BATTERIES (DIRECTIVE 2006/66 / CE)

BATTERY REGISTRATION NUMBER IT21040P00006953

The battery contained in the appliance must not be disposed of as urban waste (Directive 2006/66 / CE), as it is subject to separate collection to avoid damage to the environment.

The user must therefore give the exhausted battery-accumulator free of charge to the appropriate municipal centers for the separate collection of electrical and electronic waste, or deliver it to the special containers provided by the points of sale.

To remove the battery, open the door on the control unit, remove the battery by sliding it out of its housing.

WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU.

AEE REGISTRATION NUMBER IT21050000012993

At the end of its useful life, the product must not be disposed of with municipal waste.

It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of

separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.

I am not covered by this usurabili component warranty if the damage is imputed to the normal use of the product.

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