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USER MANUAL
COMODINA-FOLDABLE FRAME
REF: 01.04.101.43.

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly. The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO S.r.l..
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO S.r.l. declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO S.r.l. in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO S.r.l. declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO S.r.l. comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO S.r.l. guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

Compared to a traditional manual wheelchair, the commode also acts as a toilet seat. It is therefore useful during daily hygiene functions of disabled people or elderly.

The commode wheelchair, thanks to the four 125 mm wheels, can be easily moved by the healthcare staff or the caregiver at home.

SPECIFIC WARNINGS

- Keep the product packaging away from any heat source because is made of cardboard, it is inflammable.
- Do not exceed the maximum capacity (120 Kg).
- Use with caution on wet and slippery surfaces.
- Always assist the patient during handling operations.
- Before transferring the patient from / to the comfort, make sure that the device is placed on a flat and stable surface.
- Avoid leaving the device outside.

Features	
Foldable Frame	Steel
Seat	Padded
Closing Seat	Removable
Wc Bowl	PVC
Front Wheels	12,50 cm
Rear Wheels	12,50 cm
Armrest	Flip Back Armrest with PVC Armpad
Footrest	Swing Away with Leg Support
Footplate	Plastic Black
Coating	Washable and fireproof

TECHNICAL FEATURES

Code	Size	Length	Width	Height	Width Folded	Seat Width	Seat Depth	Back Height	Floor to Seat Height	Weight Capacity	Front Wheel	Rear Wheel	Weight with Footrest	Carton Size
Measures in Cm														
01.04.101.43	43	60	55	92	26	43	42.5	42	51	100	12.5 (5")	12.5 (5")	12	55x53x52

BEFORE EVERY USE

- Always check the state of wear of the parts involved so as to ensure that the device is used in total safety for people and things.
- Check the correct assembly of the device.

ASSEMBLY

1) Backrest assembly.

Remove the frame and backrest from the packaging.

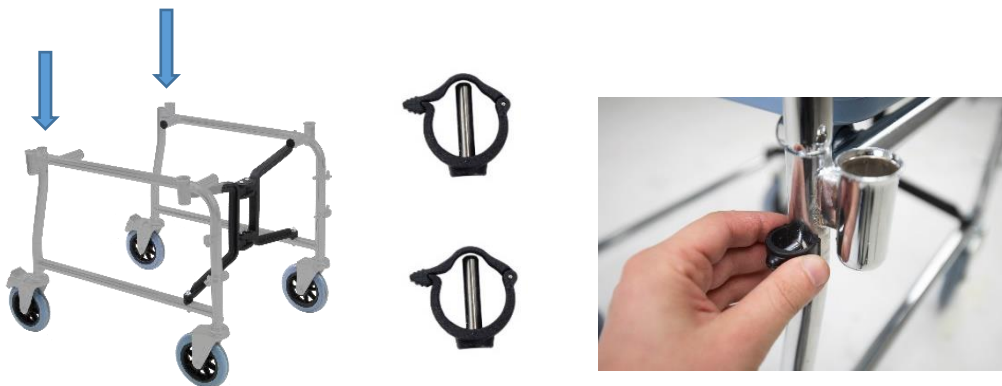
Attention!

During assembly it is recommended to lock the 2 rear wheels by pressing the grey lever downwards.

Open the frame and insert the side tubes of the backrest inside the two ends of the frame, securing the two tubes with the two plastic clips supplied, as in the image below.

Attention!

Make sure the backrest is properly secured before proceeding with assembly.

2) Seat assembly.

Remove the PVC seat from the packaging.

Place it on top of the frame and apply light pressure on it until the 2 side clips are properly locked.

Attention!

Make sure the seat is properly secured before proceeding with assembly.

3) Assembling the basin.

Remove the basin from the packaging and place it under the seat, insert it by passing from the back side of the chair by sliding it, taking care to insert it correctly into the appropriate guides.

Attention!

Make sure the basin is properly secured before proceeding with assembly.

4) Leatherette seat positioning.

Remove the leatherette seat from the box and place it correctly on top of the previously fixed PVC seat.

5) Mounting side armrests.

Remove the two side armrests from the packaging.

Insert the two side armrests into the appropriate slots located on the outside of the seat.

Attention!

Make sure to insert the two undrilled tubes near the backrest first, then attach the other two to the front of the nightstand.

It is possible to vary the height of the two armrests by choosing between two holes.

6) Footboard assembly.

Remove the two footboards from the packaging.

The removable footboards must be inserted into the two ends of the bedside table frame by inserting the pin into the hole in the frame tube.

Once inserted correctly, push the footrest down and rotate it towards the inside of the chair until it locks into place.

It is possible to vary the height of the footboards by choosing between two holes.

To remove the footboard, use the release lever, then rotate the footboard outwards and pull it upwards.

Brakes:

The brake is operated by pressing down on the grey lever.

MAINTENANCE

Check list to be carried out periodically.

- Check the brakes: monthly.
- Cleaning of the parts (seat, backrest, armrests, wheels, etc.) as needed.
- Pin lubrication: monthly.

CLEANING AND DISINFECTION

Cleaning

Use a cloth warm water and mild soap and dry with a clean cloth.

Disinfection

If necessary, carry out the disinfection of the product using a delicate disinfectant detergent of normal domestic use.

The toilet bucket must be regularly cleaned and disinfected using a normal bathroom disinfectant.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, because these substances cause the abrasion of the components of the shower chair.

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.

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