MANUALE D'USO

CINTURA PELVICO ADDOMINALE

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USER MANUAL ABDOMINAL PELVIC BELT

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains small tips for the correct use of the device of your choice and valuable advice for your safety. We recommend that you read this manual carefully before using the item you have purchased. If you have any doubts, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and the following manual without prior notice in order to improve its features.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO S.r.l.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the treating physician or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO S.r.l declines any responsibility for damage to property or people resulting from improper use, not agreed with a professional figure, which does not correspond to what is indicated in the user manual.
- Any accidents, anomalies, breakages, must be reported to DEMARTA VIRGINIO S.r.l in order to monitor the quality and criticalities of the product.
- Keep the packaged product away from any heat source as the packaging is made of cardboard.
- Always pay attention to the presence of children.
- The product has been designed for its intended use only. Do not abuse it in any way.

CE DECLARATION OF CONFORMITY

DEMARTA VIRGINIO S.r.l declares under its sole responsibility that the product manufactured and placed on the market by DEMARTA VIRGINIO S.r.l complies with the applicable provisions of EU Directive 2017/745 on medical devices.

To this end, DEMARTA VIRGINIO S.r.l guarantees and declares under its sole responsibility the following:

- The devices meet the general safety and performance requirements as required by Annex I to Regulation 2017/745 as prescribed by Annex IV to that Regulation.
- The devices are NOT MEASURING INSTRUMENTS.
- The devices are NOT INTENDED FOR CLINICAL INVESTIGATION.
- The devices are marketed in NON-STERILE packaging.
- The devices are to be considered as belonging to class I in accordance with the provisions of Annex VIII of the aforementioned regulation.

INTENDED USE

ADIGGY Abdominal Pelvic Belt offers a comfortable attachment of the patient to the wheelchair providing adequate stability of the trunk. Prevents slippage and sideways shifting.

SPECIFIC WARNINGS

- Restraints should be used by qualified healthcare professionals. If used at home, you should seek advice from your doctor or therapist.
- Patients with restraints require frequent monitoring.

DESCRIPTION


- Breathable and padded polyester fabric that offers great comfort.
- Consisting of an abdominal belt.
- Ergonomic design that adapts to any morphology.
- Adjustment and closure system by buckles on the front that allows quick positioning.

ASSEMBLY

WATCH ME!

CLOSURE SYSTEM WITH PLASTIC CLIPS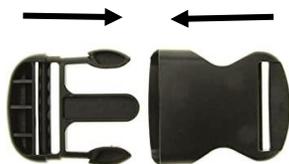

To open the plastic clip latch system, press inward on each side to release the latch. To engage, insert the two parts together and apply light pressure.

TECHNICAL DATA

Code	Size
01.12.AD10	Universal

Materials	
Waist	Polyester
Buckles	Plastic
Manutenzione e Pulizia	
	Max wash 60° C
	Do not iron
	Do not bleach
	Do not use the dryer

GENERAL DISPOSAL CONDITIONS

In case of disposal of the device, never use the normal systems for the disposal of solid urban waste. It is recommended to dispose of the device through the common municipal ecological islands for the foreseen recycling operations of the materials used.

WARRANTY

All Demarta products are guaranteed against defects in materials or workmanship for a period of 2 (two) years from the date of sale of the product, except for any exclusions and limitations specified below. This warranty is not valid in the event of misuse, abuse or modification of the product and for failure to adhere to the instructions for use. Demarta does not guarantee the products against damage or defects in the following conditions: natural disasters, failure to adhere to the guidelines and instructions for use, or failure to perform maintenance as indicated in the manual.

Wearable components are not covered by this warranty if the damage is attributable to normal use of the product.