DEMARTA
VIRGINIO

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USER MANUAL ABDOMINAL PELVIC FIXING BELT FOR BED

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains small tips for the correct use of the device of your choice and valuable advice for your safety. We recommend that you read this manual carefully before using the item you have purchased. If you have any doubts, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and the following manual without prior notice in order to improve its features.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO S.r.l.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the treating physician or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO S.r.l declines any responsibility for damage to property or people resulting from improper use, not agreed with a professional figure, which does not correspond to what is indicated in the user manual.
- Any accidents, anomalies, breakages, must be reported to DEMARTA VIRGINIO S.r.l in order to monitor the quality and criticalities of the product.
- Keep the packaged product away from any heat source as the packaging is made of cardboard.
- Always pay attention to the presence of children.
- The product has been designed for its intended use only. Do not abuse it in any way.

CE DECLARATION OF CONFORMITY

DEMARTA VIRGINIO S.r.l declares under its sole responsibility that the product manufactured and placed on the market by DEMARTA VIRGINIO S.r.l complies with the applicable provisions of EU Directive 2017/745 on medical devices.

To this end, DEMARTA VIRGINIO S.r.l guarantees and declares under its sole responsibility the following:

- The devices meet the general safety and performance requirements as required by Annex I to Regulation 2017/745 as prescribed by Annex IV to that Regulation.
- The devices are NOT MEASURING INSTRUMENTS.
- The devices are NOT INTENDED FOR CLINICAL INVESTIGATION.
- The devices are marketed in NON-STERILE packaging.
- The devices are to be considered as belonging to class I in accordance with the provisions of Annex VIII of the aforementioned regulation.

INTENDED USE


The ADIGGY Pelvice Bed in Polyester with Magnetic Closure is made for patients who require immobilization. It offers limited mobility and makes postural changes possible.

WARNINGS

- Retainers should be used by qualified healthcare personnel.
If used at home, you should seek advice from your doctor or therapist.
- Patients with restraints require frequent monitoring.

DESCRIPTION

- Polyester fabric.
- Made up of a perineal band with padded fabric.
- Magnetic locking system with key.
- Ergonomic design adapted to any type of morphology.
- Adjustment and fixing system that allows quick assembly.
- Polyester fabric.

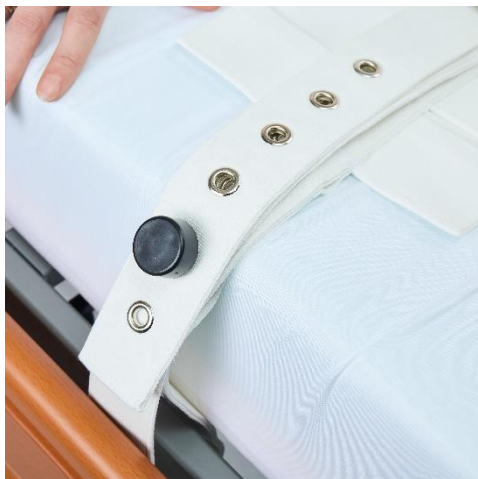OPTIONAL

- ADIGGY Bed Belt Extension in Polyester with Magnetic Closure (01.12.CAD56).

ASSEMBLY

WATCH ME!

MAGNETIC CLOSING SYSTEM

01.12.MAD55 – MAGNETIC BLOCK

1. Base with pin, it is passed through the appropriate openings on the buckles.
2. Button cap, it blocks everything by fixing magnetically to the pin.

01.12.LAD55 – MAGNETIC KEY

The magnetic key is used to open the button cap of the magnetic lock described above.

To open the magnetic lock, you need to use the magnetic key: place the key on top of the lock, this will unlock the lock and allow the pin to be extracted.

To attach, place the key on top of the lock and insert the pin into the hole.

IMPORTANT: Do not immerse the magnetic lock or the magnetic key in liquids, humidity can cause serious damage to the locking system.

IMPORTANT: Patients with pacemakers should keep the lock and magnetic key at a distance of no less than 10 cm from the pacemaker itself to avoid the onset of tachycardia..

TECHNICAL FEATURES

Code	Size	Ø Minimum	Ø Maximum
01.12.AD56PM-S	S	40 cm	60 cm
01.12.AD56PM-M	M	60 cm	100 cm
01.12.AD56PM-L	L	100 cm	140 cm

Materials	
Belt	Polyester
Magnetic Closure	Metal and Plastic
Maintenance and Cleaning	
	Water temperature not above 40° C
	Do not iron
	Do not bleach
	Do not dry in dryer

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.