MANUALE D'USO
SEDIA DOCCIA
AQUA MARINE AD AUTOSPINTA

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SHOWER CHAIR

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO S.r.l..
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO S.r.l. declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO S.r.l. in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO S.r.l. declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO S.r.l. comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017. For this purpose, DEMARTA VIRGINIO S.r.l. guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

Evacuation and mobility aid intended for people with walking difficulties in the bathroom.

TECHNICAL FEATURES

Length (cm)	Width (cm)	Height (cm)	Seat Width (cm)	Seat Depth (cm)	Backrest Height (cm)	Seat Height from Ground (cm)	Front Wheel Dimensions (cm)	Rear Wheel Dimensions (cm)	Maximum Load (Kg)	Net Weight (Kg)	Carton Dimensions (cm)
											
98	63	97,5 / 97	46	44,5	47,5 / 50	52	12.5 (5")	60 (24")	120	17.2	82 x 39,5 x 65

ASSEMBLY

1) Assembly of front wheels.

Remove the frame from the packaging and insert the two front wheels into the appropriate holes, securing them with the two plastic clips supplied.

2) Rear wheel assembly.

Remove the rear wheels from the packaging and screw them to the frame with the screws and washers shown in the photo.

The frame has several holes, where you can adjust the height of the wheels.

After placing the screws and washers between the frame and the wheels of the shower chair, secure the wheels using the 2 bolts shown in the photo.

Subsequently, equip yourself with the black caps shown in the photo and apply them in the appropriate holes on the two wheels.

Attention!

After having assembled the 4 wheels, make sure that they are correctly stable before proceeding with the assembly of the frame.

3) Frame assembly.

Remove the two white tubes from the packaging and insert them into the appropriate holes in the two ends of the frame.

After inserting the 2 tubes, secure the frame by inserting the 4 plastic clips supplied into the holes, as in the image below.

Attention!

After having fixed all the clips, ensure the correct stability of the frame before proceeding with assembly.

4) Backrest assembly.

Remove the backrest depicted in the image below from the box.

The side tubes of the backrest have several holes where the height can be adjusted.

After choosing the desired height, insert the two side tubes of the backrest inside the two ends of the frame.

Then secure the two side tubes with the two star locking knobs.

5) Seat assembly.

Remove the seat from the packaging.

Position it over the frame and apply light pressure on it until the 4 clips are correctly locked.

Attention!

Make sure it is correctly secured before proceeding with assembly.

6) Tray assembly.

Remove the basin from the packaging and place it under the seat, insert it from the rear side of the chair, sliding it, taking care to insert it correctly into the appropriate guides.

7) Insertion of the seat closing cap.

Remove the seat closing cap from the packaging and insert it into the appropriate hole provided between the basin and the seat.

8) Footrest assembly.

Remove the two shower chair platforms from the packaging.

The extractable footrests must be inserted into the two ends of the chair frame by inserting the black pin into the hole in the frame tube.

After inserting them correctly, push the footrest down and rotate it towards the inside of the chair until it locks onto the stop point. To remove the footboard, use the release lever, then rotate the footboard outwards and extract it upwards.

BEFORE EACH USE

- Always check the state of wear of the mechanical parts involved so as to guarantee use of the device in total safety for people and things.
- Check the correct assembly of the device.

WARNINGS FOR USE

- Do not exceed the maximum permitted capacity (120 kg) for any reason.
- Always assist the patient during handling operations.
- Completely block the chair wheels whenever the patient is transferred.
- Before transferring the patient from/to the chair, make sure that the latter is placed on a flat and stable surface and with the wheels blocked.
- Before leaning forward or getting up from the chair, make sure that the front wheels are straight and the rear wheels are blocked, to avoid a possible tip-over.
- To avoid possible tipping, do not lean beyond the back of the chair.
- Keep the chair away from direct sources of heat.

USE

Get on the shower chair

- Bring the wheelchair as close as possible so that it is positioned next to the patient.
- Lock the rear wheel brakes.
- Lean on the frame structure using your arms, then slowly lower yourself towards the seat.

Get off the shower chair

- Get as close as possible to the side of a new session.
- Lock the rear wheel brakes;
- Place your arms on the frame of the seat on which you are to move and slowly slide towards the seat as you stand up.

IMPORTANT! The assistance or supervision of experienced and qualified personnel is required to avoid slips and falls.

TOILET BASIN

Use the toilet bowl with the shower chair fully braked.

ARMRESTS

To facilitate the transport of the chair and the transfer of the patient, all the armrests are completely foldable.

ATTENTION!

Never use the armrests to lift or move the chair.

MAINTENANCE

The shower chair requires a few basic periodic maintenances. Below we list the checks to be carried out periodically.

- Check the brakes: monthly.
- Cleaning of parts (seat, backrest, armrests, wheels, etc.) as needed.
- Pin lubrication: monthly.

It is of fundamental importance that the shower chair is completely checked in all its parts when used for the first time, in particular that the screws and nuts are all well tightened.

CLEANING AND DISINFECTION

Cleaning

Use a cloth soaked in warm water and neutral soap and dry with a clean, dry cloth.

Disinfection

If necessary, disinfect the product using a delicate disinfectant detergent for normal household use.

The toilet bowl should be regularly cleaned and disinfected using a standard bathroom disinfectant.

ATTENTION!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, because these substances cause abrasion of the shower chair components.

The manufacturer will not be responsible for damage caused by the use of materials that may deteriorate the surface of the product or by chemical corrosives used during cleaning.

GENERAL DISPOSAL CONDITIONS

If disposing of the device, never use normal municipal solid waste disposal systems. It is recommended to dispose of the device through the common municipal recycling centers for the planned recycling operations of the materials used

WARRANTY

All Demarta products are guaranteed against defects in material or workmanship for a period of 2 (two) years from the date of sale of the product, subject to any exclusions and limitations specified below. This warranty is not valid in the event of improper use, abuse or modification of the product and failure to follow the instructions for use. Demarta does not warrant products against damage or defects under the following conditions: natural disasters, failure to adhere to guidelines and instructions for use, or failure to perform maintenance as indicated in the manual.

Wearable components are not covered by this warranty if the damage is attributable to normal use of the product.