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## Warranty

We guarantee this device (except bulbs) for five years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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ENGLISH

## HEINE tongue-depressor for wooden tongue-blades

Read these instructions carefully before using the HEINE tongue-depressor and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

### Safety information

**Range of applications:** The HEINE tongue-depressor is used exclusively to illuminate the mouth and throat when fitted with a wooden tongue-blade.

**Safety in use:** Use only handles which are specifically designed for use in medical applications.

### Instructions

Insert the wooden tongue-blade into the opening (3) as far as it will go. If it does not locate firmly, reverse the blade and insert the other end.

### Changing the bulb

Unscrew the light-guide (1) from the head and pull out the used bulb (2). Insert a new bulb and screw in the cap (1).

**Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.**

### Cleaning and sterilisation

The tongue-depressor can be gas sterilised, although cleaning with alcohol is the method of choice. Autoclaving is not permitted.

### Replacement parts and accessories

Replacement light-guide for bulb	<b>B-00.12.308</b>
HEINE XHL® Xenon halogen spare bulb 2,5 V	<b>X-01.88.037</b>
HEINE XHL® Xenon halogen spare bulb 3,5 V	<b>X-02.88.049</b>

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