GARANZIA

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WARRANTY

The product has **2-year** warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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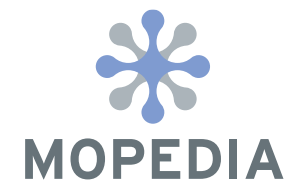
WOOD BATH CHAIRS INSTRUCTION MANUAL

Moretti S.p.A.
Via Bruxelles 3 - Meieto
52022 Cavriglia (Arezzo)
Telefono +39 055 96 21 11

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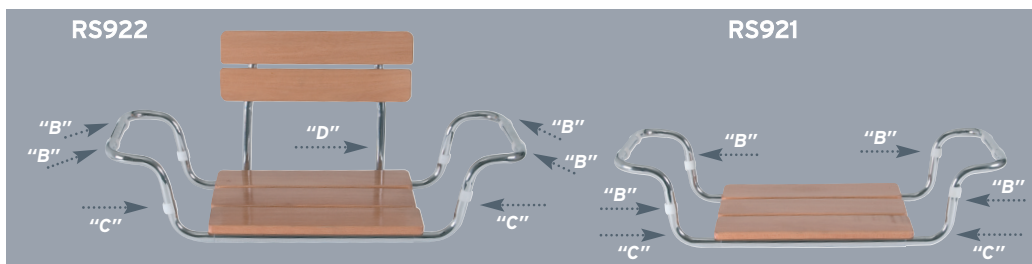
Directive 93/42 EEC

Codes RS921-RS922**Technical features**

Thank you for purchasing our bath chairs **MOPEDIA** by Moretti, which have been manufactured according to the highest quality and safety standards.

Assembly instructions

- Slide armrests "C" in the frame.
- Use the plastic spacers to use the chair on baths 67 cm width.
- 8 plastic clips "B" are provided to protect the sides of the bath.
- Insert the backrest "D" in the frame as illustrated (only for RS922).

Cleaning

Use a damp cloth and a normal detergent. We recommend not to submerge the seat in the bath water, drain the seat tubes after use and dry the seat thoroughly.

Warning

This product is recommended for people weighting 100 Kg. maximum.

The manufacturer and distributor decline all responsibility for any consequences resulting from an incorrect use of this product.

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the use, adjustment or maintenance of the product, please contact your dealer.
- Worn, damaged or missing components must be replaced.
- Do not store the product outdoors.
- Periodically check the stability of the product.

To fit baths from 53 to 67 inside bath measurement.