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HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0 · Fax +49(0)81 52/38-202
E-mail: info@heine.com · www.heine.com
med 5489/5 1 / 2.09

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Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories.

(Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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Instructions for use

HEINE Slit instrument head

Read these instructions carefully before using the HEINE slit instrument head and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

The safety, reliability and performance of the unit can only be guaranteed when it is used according to the instructions for use.

Please note: The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

Tips and Specula

Reusable tips and specula should be attached to the head in such a way that the locating pins engage in the bayonet fitting. They can be firmly located by twisting the milled ring to the right. Disposable specula can be attached by means of an adaptor (G-000.21.302).

Magnification lens

The head is fitted with a swivel lens of approx. 2.5 x magnification. The lens can be used with all tips and specula except the 150 mm tubular speculum (G-000.21.350).

Changing the bulb

Please note: The performance of this unit can only be guaranteed if genuine HEINE bulb are used.

- Remove head from handle and remove the bulb from the male connector by grasping the lower edge and pulling downwards.
- Before inserting a new bulb, check that the voltage printed on the body of the bulb corresponds to that of the handle or power supply.
- Push in the bulb as far as possible.

Cleaning and sterilization

The lens and bulb must be removed before cleaning / disinfection.

Manual cleaning and disinfection

Use only medical products which are approved for metal and plastic when cleaning and disinfecting the instrument head and specula or tips.

Cleaning and disinfection in a machine

The above products can be cleaned and disinfected in a machine at up to 93 °C. Use only cleaning and disinfectant products which are approved for use on metal and plastic.

Sterilisation

Instrument head, tips and specula can be sterilised and autoclaved at up to 134 °C for 5 minutes.

Accessories and Replacement Parts

HEINE XHL® Xenon halogen spare bulb 2,5 V	X-001.88.077
HEINE XHL® Xenon halogen spare bulb 3,5 V	X-002.88.078
Swivel lens	G-000.21.209
Adaptor for disposable specula	G-000.21.302
Pack of 1000 UniSpec disposable specula	
dia. 2,5 mm	B-000.11.242
dia. 4 mm	B-000.11.241
Specula, closed in SANALON S, length 40 mm	
dia. 2,2 mm	G-000.21.310
dia. 2,8 mm	G-000.21.311
dia. 3,5 mm	G-000.21.312
dia. 4,5 mm	G-000.21.313
dia. 5,5 mm	G-000.21.314
1 set = 5 pcs.	G-000.21.316
Specula, closed in metal	
Length 57 mm, dia. 4 mm	G-000.21.330
Length 65 mm, dia. 6 mm	G-000.21.331
Length 90 mm, dia. 7 mm	G-000.21.332
Specula, slit in metal	
Length 65 mm, dia. 6 mm	G-000.21.341
Length 90 mm, dia. 7 mm	G-000.21.342
Metal tubus, length 150 mm, dia. 10 mm	G-000.21.350
Metal spreadable speculum	G-000.21.360

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