MOPEDIA

CE

RS947

RS946

RS944

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Moretti S.p.a.INTERNET: www.morettispa.com E-MAIL: info@morettispa.com

MADE in TAIWAN

Medical Device Class I
Directive 93/42/EEC and subsequent amendments

Codes:

RS944 reclining and tilting shower chair

RS946 shower chair with four small wheels

RS947 self propelled shower chair

Introduction: Thank you for purchasing a shower and toilet chair **MOPEDIA** by Moretti S.p.A. Its design and its quality guarantees to the user comfort, safety and reliability. The commode shower and toilet chairs **MOPEDIA** by Moretti S.p.A. are designed and realized to meet your requirements for a correct use in inner spaces. The production has been made according to the highest quality and safety standards in accordance with the European Directive on Medical devices (93/42/EEC). This instruction manual gives you suggestion for a correct use of the aid and good safety advice. We recommend you to read carefully this instruction manual before the use. In case of doubts, please contact your dealer.

Intended use: The **MOPEDIA** shower and toilet chairs are created for people with movement difficulty and can be used indoors as substitution of toilet or as aid under the shower.

EC Declaration of conformity: Moretti S.p.A. declare under sole responsibility that the product made and traded by Moretti S.p.A. and belonging to Shower chair range are in accordance with the European Directive on Medical devices (93/42/EEC).

Moretti S.p.A. guarantees and declared under sole responsibility what follow:

1. The devices under discussion satisfy the essential requisites requested by the **I°** enclosure directive 93/42/EEC as laid down by the **VII°** enclosure of the above mentioned Directive.
2. The devices under discussion ARE NOT MEASURING INSTRUMENTS.
3. The devices under discussion ARE NOT MADE FOR CLINICAL PROBES.
4. The devices under discussion are traded with NON STERILE BOX.
5. The devices under discussion belong to class **I°**.
6. IT IS NOT ALLOWED to use and install devices except for the use provided by Moretti S.p.A. Moretti S.p.A. places technical documentation at relevant authorities in order to prove the conformity of 92/42/EEC device.

Assembling instructions: You will receive Mopedia wheelchair with disassembled backrest and footrests. To assemble the backrest, insert the ends of the tubes in the appropriate holes. Screw the handles to fix the backrest as in (pic.1). To assemble the footrest, insert it in the hole, push it downwards and then inwards until the safety block. To remove the footrest, unlock the safety block, turn the footrest outwards and pull it out (Pic. 2). Footrests have to be turned inward. Place the toilet bowl from the back part of the chair.

Footrest adjustment: The footrests of shower chair can be adjusted according to your needs: unfasten the screw, adjust the footrest height and fasten the screw again. Check that both footrests are adjusted at the same height.

Seat height adjustment (only for RS946): The seat height can be adjusted from 49cm to 54cm, with 3 different levels. Remove the safety clip, change the height and insert the clip again. Check the stability of the chair before the use: all the safety clip have to be in the correct position.

(Pic. 1)

WARNING! Read carefully this instruction manual for a correct use.
Pay attention to the presence of children.

Use: Use only the back handle to move the chair, never use other supports. Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. Assistance is recommended when in slope higher than 10%. If you need to stop in a sloping surface, avoid abrupt movements. During the downhill, do not change suddenly the direction. Lift the chair up if necessary, using the main frame and not armrests or footrests.

(Pic. 2)

WARNING! The patient must not put pressure on footrests when go up and down from the chair.

Tilting shower chair (only item rs944): The tilt range here can be variable tilted 25° backwards and 5° forwards. Control of tilting action is simply performed by a lever under the backrest bar-style push handle control. Push back handle down to tip commode chair back or lift it up from a tilted position. Stop pushing the tilting lever to stop tilting adjustment

WARNING! Patient safety measures must be attended while tilting adjustments.

WARNING! Apply the wheels brake while tilting adjustments.

WARNING! Pay attention to tilting action while patient is sitting on the chair. Tilt action must be performed only by trained personnel. Rude tilting movements may put in danger stability and safety chair.

WARNING! Two spring pistons make tilting movement easier. Attend closely to backrest while tilting action. Particular attention is recommended from backward to forward position in order to avoid dangerous status for patient.

Headrest adjustment (only item rs944): Headrest may be depth and height adjusted thanks to two knobs on the backrest bar. A swivel ball joint allows lateral backrest tilt to meet all patient needs.

Brakes: In model RS946 you can brake the chair pushing the lever on each wheel. In model RS947, if you need to adjust the brake lever, you can unscrew the nut, move the clamp at the required position and then tighten again. (Pic. 3)

(Fig. 3)

Casters: Casters should be inspected periodically and lubricated when necessary.

If the wheel wobbles, tighten the nut of the pivot slightly. To test the wheel, lean the chair to one side and spin the wheel.

Armrest: To facilitate the transport of the wheelchair, you can remove or overturn the armrests. Never use them to move or to lift the chair.

WC: Use the toilet when the wheelchair is still. To use the toilet take out the removable part of the seat using the handle on the front part. Take out the toilet using the handle.

Maintenance: The MOPEDIA by MORETTI S.p.A. chairs need minimal maintenance and general

care, but we suggest you to control regularly:

- Braking system: monthly (In model RS946 you can brake the chair pushing the lever on each wheel. In model RS947, if you need to adjust the brake lever, you can unscrew the nut, move the clamp at the required position and then tighten again).
- Cleaning of the seat: when necessary.
- For the cleaning of chromium plated or metallic parts of the seat, use water and soap used for cars. Seat and backrest can be cleaned with a damp cloth and a normal detergent. The toilet must be cleaned and disinfected regularly using a normal bath detergent.

WARNING! Do not use the chair for a purpose not indicated in this manual.

Moretti S.p.A. declines all responsibility for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer can modify the device or the instruction manual without prior notification.

- Don't skew the chair more than 10% without assistance.
- Block the wheels during patient transfer.
- Wheelchair has to be in a stable surface and wheels locked during user transfer.
- Before leaning or get up, be sure casters are in straight position and rear castors are blocked in order to avoid a turnover.
- Do not lean out of the backrest to avoid a turnover.
- Pay attention to your fingers during assembling, regulations or simply during the use.
- When using the self-propelled wheel, pay attention to your fingers.
- Keep the chair far from fire.
- Use always original accessories and spare parts.

Disposal: The product must not be disposed of as urban waste. Contact the local authority for separate waste collection.

Technical features:

FRAME: painted-aluminium

PADDED SEAT/BACKREST/LEG REST/HEADREST: PU

MAXIMUM WEIGHT: 125 kg.

MODELLO														
RS944	135 100	82 88	42	105 132	75	54	40 50	45	46	56	14,5	12	12	
RS946		78 85	42	99 101,5	69 71,5	49 51,5	40 55	45	46	57	12	12	12	
RS947		105 112	42	99	73	54	39 54	45	46	64	17	60	20	

Accessories and spare parts:

You can check Moretti catalogue for accessories and spare parts.

WARRANTY

The product has 2-year warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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