MA CPA5xx 03 A_07-2018

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CPA517 Shaped Wheelchair Tray 68x57cm

2. INTRODUCTION

Thank you for purchasing a **ARDEA ONE product by Moretti** product range. Its design and quality guarantee comfort, safety and reliability to the user. The products have been planned in Italy from the Research and Development department of Moretti S.p.A. Italia in order to meet your requirements. The material used, the accessories and the quality test are continuously checked by Moretti staff. Only the production is made in PRC according to the highest quality safety standards in accordance with the European Directive on Medical devices. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: check all parts for any shipping damage, and test before using. In case of damage, do not use the device. Contact your dealer for further instruction.

3. INTENDED USE

This product is an accessory for medical devices (wheelchairs) for people with mobility problems. Adaptable for wheelchairs of measures from 38 to 53cm (**CPA501, CPA502 and CPA506, CPA507**) and > of 53cm (**CPA516, CPA517**).

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without notice

4. DECLARATION OF CONFORMITY CE

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the wheelchair category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the said directive

2. A complete list of the concerned devices is mentioned in the enclosure A of the declaration

3. The devices in question **ARE NOT MEASUREMENT DEVICES**

4. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**

5. The devices in question are sold in **NON STERILE** packaging

6. The devices in question should be considered as belonging to class I

7. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.

8. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/CEE and further modifications to the proper authorities

5. GENERAL WARNING

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Do not smoke and / or contact with heat sources.
- Keep the packaged product away from any heat source.
- The useful life of the device is determined by wear of non-repairable parts and / or replaceable.
- Always pay attention to the presence of children.
- Do not put on the tray weights exceeding the payload of the table.

WARNING!

Danger of tipping and/or deformation of the supports

6. GENERAL DESCRIPTION

6.1 Parts view

CPA501 - CPA502 Tray 55x32cm

- Tray - 1 pc
- Support for fixing - 2 pcs
- Screws M5 - 4 pcs
- Screws Withworth - 4 pcs
- Knobs - 2 pcs

CPA506 - CPA507 Shaped Tray 58x38cm

- Tray - 1 pc
- Support for fixing - 2 pcs
- Screws M5 - 4 pcs
- Screws Withworth - 4 pcs
- Knobs - 2 pcs

CPA516 - CPA517 Shaped Tray 68x57cm

- Tray - 1 pc
- Support for fixing - 2 pcs
- Screws M5 - 4 pcs
- Screws Withworth - 4 pcs
- Knobs - 2 pcs

7. ASSEMBLING AND USE

The product can be easily installed, just following a few manual operations.

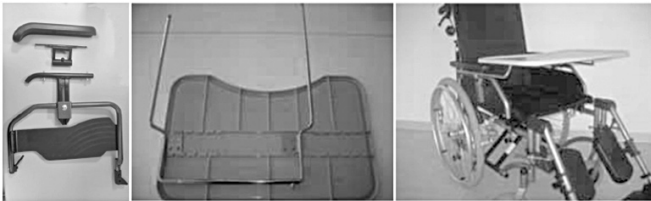
Note: make reference to safety warnings in this manual.

CPA501 Wheelchair Tray 55x32cm

CPA506 Shaped Wheelchair Tray 58x38cm

CPA516 Shaped Wheelchair Tray 68x57cm

1. Unscrew the screws and put the support between armrest and the handrail tube, fix the whole set with the screws provided or reuse the existing ones.
2. Repeat for the other armrest.
3. Insert the tube into the supports of the tray and fix with the knobs.
4. The wheelchair tray is now ready for use.


Only illustrative purpose images

NOTE If you need, cut the table's tubes to use the accessory

For the wheelchairs **CP200-xx e CP205-xx** use the spacers and the screws **M5x25**

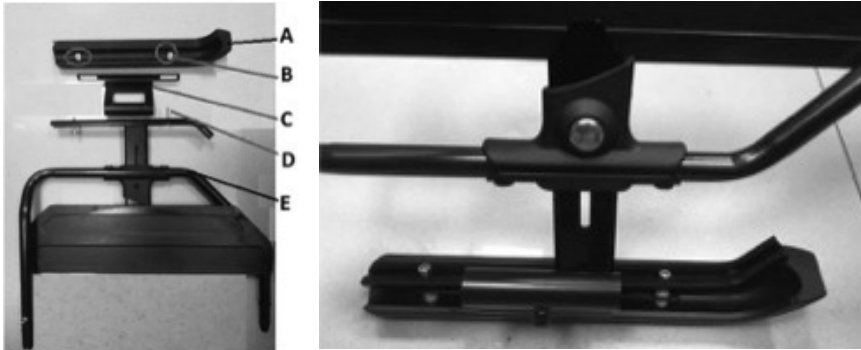- A.** PU padding
- B.** Spacers (2 pcs)
- C.** Tray support
- D.** Screws M5x25
- E.** Armrest frame

Turn over the armrest

1. As showed in the picture, remove the PU padding (A), from the armrest frame (E)
2. Place the PU padding on a flat surface and insert the spacers (B)
3. Then set the tray support (C), the armrest frame (E) and insert the screws (D).
4. Tighten the screws (D).

8. BEFORE USE

- Always check the wear of the parts where the accessories have to be fixed to ensure the safe use of the device for persons and things.
- Check the correct assembly of the product. Ensure that all parts are properly assembled.
- After completing the assembly, verify the correct operation of the product.
- You can use the self-propelling wheelchairs, being very careful to entrapment due to the reduction of visibility.
- Adapt products to the user and the wheelchair in use.

9. CLEANING AND DISINFECTION

9.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

WARNING!

Do not use abrasive substances, acids, alkaline, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

9.2 Disinfection

If you need to disinfect the product use a common disinfectant cleaner.

Note: absolutely avoid the use of acids, alkaline or solvents such as acetone or diluent.

10. GENERAL CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycle.

11. TECHNICAL FEATURES

11.1 Dimensions and weights

CPA501 - CPA502 Wheelchair Tray 55x32cm

- Height **3cm (folded)**
- Width **55cm**
- Depth **32cm**
- Pay load **5Kg**

CPA506 - CPA507 Shaped Wheelchair Tray 58x38cm

- Height **7cm (folded)**
- Width **58cm**
- Depth **38cm**
- Pay load **5Kg**

CPA516 - CPA517 Shaped Wheelchair Tray 68x57cm

- Height **7cm (folded)**
- Width **68cm**
- Depth **57cm**
- Pay load **5Kg**

11.2 Technical specifications

CPA501 - CPA502 Wheelchair Tray 55x32cm

- Adaptable for wheelchair <53cm
- Steel frame
- Tray by 100% PP
- rectangular shape with rounded corners
- Supports adaptable for any type of armrest, made of black painted steel
- Recommended storage: Store in a cool and dry place and keep away from direct heat sources such as sunlight. (Temp. Recommended between +10 ° C / + 40 ° C)

CPA506 - CPA507 Shaped Wheelchair Tray 58x38cm

- Adaptable for wheelchair <53cm
- Steel frame
- Tray by 100% PP
- rectangular shape with cut, rounded corners
- Supports adaptable for any type of armrest, made of black painted steel
- Recommended storage: Store in a cool and dry place and keep away from direct heat sources such as sunlight. (Temp. Recommended between +10 ° C / + 40 ° C)

CPA516 - CPA517 Shaped Wheelchair Tray 68x57cm

- Adaptable for wheelchair >53cm
- Steel frame
- Tray by 100% PP
- rectangular shape with cut, rounded corners
- Supports adaptable for any type of armrest, made of black painted steel
- Recommended storage: Store in a cool and dry place and keep away from direct heat sources such as sunlight. (Temp. Recommended between +10 ° C / + 40 ° C)

12. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

13. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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Prodotto/Product _____

Acquistato in data/Purchased on (date) _____

Rivenditore/Retailer _____

Via/Address _____ Località/Town/city _____

Venduto a/Sold to _____

Via/Address _____ Località/Town/city _____

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN ITALY

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com