**HEINE BETA 200®/K180®**

CE 93/42/EWG/CEE/EEC

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 - 82211 Herrsching - Germany  
Tel. +49(0)8152/38-0  
Fax +49(0)8152/38-202  
E-Mail: info@heine.com · www.heine.com

med 9396 1/11.10

**HEINE AllSpec® Tip-Spender**

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**Instructions****HEINE BETA 200® / K 180® Otoscope**

Before using the Otoscope read these instructions carefully and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

**Safety information**

**Range of applications:** The HEINE BETA 200® and HEINE K 180® Otoscope is used for the examination of the external ear and for general non-invasive examination.

**Safety in use:** Use only handles or power sources which are specifically designed for medical use. The Otoscope should only be inserted into the ear when a tip or speculum is fitted. Pressure should be applied with great care when doing pneumatic testing.

**Please note:** The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

**Using the instrument**

The otoscope should only be introduced into the ear when a SANALON S or AllSpec tip is fitted.

- Push the tip (1) onto the instrument head (2) with a rotary movement so that the projection inside the tip engages in the slit (3) and rotate clockwise until it fits firmly.
- With the lens window (4) closed, a magnification of ca. 3x is available for routine examinations.
- An insufflation bulb can be attached to the insufflation nipple (5) for pneumatic testing.
- The lens window (4) can be swivelled to facilitate instrumentation under magnification.

**General illumination:** The HEINE BETA 200® and K 180® otoscope can be used with or without speculum or tip for general illumination of e.g. the oral cavity or skin. Avoid contact with sensitive or damaged skin.

**Changing the bulb**

**Please note:** The performance of this unit can only be guaranteed if genuine HEINE XHL® Xenon Halogen spare bulbs are used.

- Simply detach the otoscope from the handle and pull out the bulb (6).
- Push in the new bulb as far as it will go.

**Cleaning and Sterilization**

**Otoscope head:** The outside can be cleaned with a soft cloth, the inside with a cotton-wool bud moistened with alcohol. Gas sterilization is permitted. The instrument should not be soaked.

**Reusable SANALON S tips** can be cleaned with commercially – available solutions, disinfected or boiled. Sterilization at up to 170°C is permitted.

**AllSpec tips and disposable specula** are for single use only. Do not attempt to clean them as the tip/specula may be damaged and could injure the patient.

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