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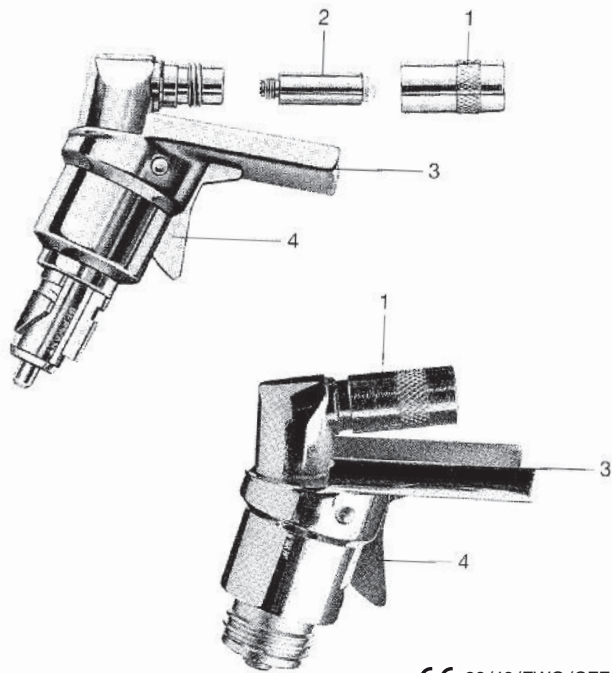
# Instructions

## HEINE tongue-depressor with blade ejector

Read these instructions carefully before using the HEINE tongue-depressor with blade ejector attachment and keep them in a safe place for future reference.

### Safety information

**Range of applications:** The HEINE tongue-depressor is used to illuminate the mouth and throat with or without a tongue-blade fitted to it. Only HEINE disposable tongue blades may be used.

**Safety in use:** Use only handles which are specifically designed for use in medical applications. Avoid contact with mucosa or damaged skin with the tongue-depressor.

### Inserting and ejecting blades

Push a HEINE disposable tongue-blade into the guide aperture (3) until the catch latches into the rectangular notch in the blade.

The blade can be ejected after use by squeezing the trigger (4).

### Changing the bulb

Unscrew the bulb cover (1) from the head and remove the bulb (2). Insert a new bulb and retighten the cover (1).

**Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.**

### Cleaning and sterilisation

The tongue-blade holder can be gas sterilised, although cleaning with alcohol is normally satisfactory. Autoclaving is not allowed.

### Replacement parts and accessories

Bulb cover	B-000.12.308
HEINE XHL® Xenon halogen bulb 2,5 V (also für alpha)	X-001.88.037
HEINE XHL® Xenon halogen bulb 3,5 V	X-002.88.049
Pack of 100 disposable tongue-blades	B-000.12.304

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## Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories.

(Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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