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INSTRUCTIONS FOR USE AND MAINTENANCE

Mod. HAPPYNEB II

HAPPYNEB II is a unit for the treatment and relieve of respiratory ailments. It is designed principally for domestic use.

HAPPYNEB II is equipped with a powerful piston compressor that does not need lubrication. The unit includes an air filter to be periodically replaced (every 70 applications). It is built in compliance with the current European Manufacturing and Safety Standards for electromedical equipments (EN 60 601-1).

HAPPYNEB II is supplied with the following 3A accessories: nebulizing bulb with mouthpiece, nasal prong, pediatric mask, adult mask, connection tubing, filter (3).

SAFEGUARDS

- Never use an extension cord or electrical adapter. Keep the cord away from heated surfaces.
- Do not handle the plug with wet hands, and not use the product while bathing. Never submerge the unit in water; should this occur, unplug immediately. Do not reach for a product that has fallen into water; unplug it first. After removing the product from the water, do not use it again. (Return it immediately to an authorized service station).

Always unplug the power cord immediately after use.

Service should be performed only by authorized repair voids the warranty.

INSTRUCTIONS FOR USE AND CLEANING

Plug the power cord into the electrical outlet corresponding to the voltage of the unit. To use the ampul and its accessories, consult the additional sheet.

When the application has finished, turn the unit off and carefully clean the accessories. Do not wash the unit under running water or do not submerge it in water; wipe it only with a damp cloth using mild detergents. Do not use abrasive cleaners.

Replacing the spare filter: lift the filter support lid (3) by levering with a flat-ended screwdriver as illustrated in figure 1a; detach the filter to be replaced and insert the spare filter as illustrated in figure 1b.

Finally, insert the filter support lid (3) into its seat, making it adhere well to the surface.

Does not cause of interference with TV or radio.

Unit case not water - proof.

The unit should not be used in the presence of anesthetic mixture inflammable with air, oxygen or nitrous oxide.

IMPORTANT SAFEGUARDS

This is a medical device and is to be used only upon medical prescription and to be operated in accordance with this manual. It is important that the patient reads and understands the information provided for use of the unit. Please contact your retailer with any question. Children and invalids should always use the unit under strict supervision of an adult who had read this manual.

MICROBIC CONTAMINATION

In the presence of pathologies with the risk of infection and microbic contamination, a personal use of the accessories and the nebulizing bulb is suggested (always consult your physician).

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