



Item code: **EL0072**


PROFESSIONAL LAMP

Ultraviolet (UV) and Infraray (IR) lamp

User manual

Table of Contents

1. INSTRUCTION FOR ASSEMBLING.....	3
2. SPECIAL WARNINGS AND USE RESTRICTIONS.....	3
3. INSTRUCTION FOR USE OF THE LAMP.....	4
4. TIMING FOR UV-IR RAYS EXPOSURE.....	4

Follow us on 

Lamp is equipped with:

- Column and relative securing rings, nut and top cover.
- The lamp is mounted on a stand
- Circuits and commands
- Cap provided with generators for UV rays and for IR rays and with adjustable arm and securing sleeve.

INSTRUCTION FOR ASSEMBLING

1. After removing the top cover, insert the sleeve of the cap into the column and secure it to desired height using the ring supplied. After this operation replace the top cover.
2. Attach the tripolar plug to the connecting cable.

SPECIAL WARNINGS AND USE RESTRICTIONS



WARNING

- Wait 2/3 minutes before initiating exposure so the generators have enough time to reach the necessary intensity;
- Do not use from a distance of less than 50 cm;
- The exposure to UV rays must be progressive, starting with one minute and not exceeding 12 minutes., but after at least 10 or 12 applications;
- During the exposure to rays, keep eyes well protected with the appropriate glasses to avoid inflammations.



REMEMBER

- The lamp has a double circuit and so it is possible to use UV and IR rays separately, also with possibilities for combined treatments (UV and IR)

INSTRUCTION FOR USE OF THE LAMP

- a. Adjust the lamp to the most suitable position for exposure to rays.
- b. To initiate the UV ray system, switch on the interrupter located to the right of the timer clock.
- c. To initiate IR system, switch on the interrupter located to the left of the timer clock.
- d. For the combined use of UV and IR system, switch on both interrupters.
- e. After the machine has turned on, wait 2/3 minutes, before initiating exposure so the generators have enough time to reach the necessary intensity.
- f. After the lamp is turned off it is necessary to wait until the quartz generator has cooled (5 min. ca.), before turning on again.
- g. The distance for exposure of the UV rays varies from 50 to 100 cm, according to the individual skin sensibility.
- h. Slightly decrease the distance for subjects with resistant skin. During the exposure to rays, keep eyes well protected with the appropriate glasses to avoid inflammations.
- i. If subject experience a reaction to UV rays (redness, burning, etc.) follow the minimum times (1/2minutes) for the first 3 or 4 treatments.

TIMING FOR UV-IR RAYS EXPOSURE

In the following chart times for UV exposure after every single treatment considering different skin types.

As described, the exposure to UV rays must be progressive, starting with one minute and not exceeding 12 minutes, but after at least 10 or 12 applications.

Keeping in mind that reactions vary from skin to skin, follow the times indicated in the chart. The exposure to IR rays must be progressive too, starting with a minimum of 3 minutes and not exceeding 12 minutes.

For exposure to combined rays, follow times for UV treatment.

TIMES FOR UV EXPOSURE			
(expressed in minutes)			
DISTANCE 50 cm	Very sensitive skin	Normal skin	Resistant skin
1 st treatment	1	1 ½	2
2 nd treatment	1 ½	2	2
3 rd treatment	2	2 ½	3
4 th treatment	2	3	4
5 th treatment	3	3	5
6 th treatment	3	3	6
7 th treatment	3	4	7
8 th treatment	4	5	8
9 th treatment	4	5	9
10 th treatment	5	6	10
11 th treatment	5	7	11
12 th treatment	6	8	12
13 th treatment	6	8	12
14 th treatment	6	9	12
15 th treatment	6	9	12

CHINESPORT spa
Via Croazia, 2 - 33100 Udine - Italy
Tel. +39 0432 621 621
Fax +39 0432 621 620
chinesport@chinesport.it
www.chinesport.it



C.F e P.IVA IT-00435080304
Cap.Soc. € 1.377.750,00 i.v.
Reg. Impr. Udine 00435080304
M/UD 007048



CHINESPORT S.P.A
Via Croazia,2 - 33100 Udine, Italia
www.chinesport.it

DIRECT E-MAIL FOR TECHNICAL ASSISTANCE

assistenza@chinesport.it

Alternatively you can contact us by
fax at n° (+39) 0432-621698 or telephone (+39) 0432 621699

Please note:

Before contacting the Service Center, please identify the "SERIAL NUMBER" in the product label attached to the equipment (see facsimile above), or the transport document/invoice number. This will speed things up and improve service quality.

A piece of equipment that requires repairs at Chinesport's facilities must have an "AUTHORIZATION NUMBER" on the shipment packaging. This number must be requested from Technical Support and will speed things up and improve service quality.