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USER MANUAL
SWIVEL BATH SEAT
SKU 02.02.357U

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

Bath safety are all those devices that can be used by a disabled or elderly person to prevent, compensate or eliminate the difficulty in performing all the functions in the bathroom.

SPECIFIC WARNINGS

- Always help the patient with walking difficulties to position on the seat.
- Avoid sudden movements during positioning.
- Watch out for wet, damp or slippery floors, which could make positioning of the patient difficult.
- Do not store the product outside. Periodically check the stability of the product and the correct fixing of the screws and the condition of the rubber protections.
- If in doubts about the use, adjustment or maintenance of the product, contact the retailer.

DESCRIPTION

The SWIVEL BATH SEAT has a steel frame with four anchor points at the edge of the bathtub and is equipped with armrests to ensure maximum comfort and safety. This device has an adjustable fixing system.

The supporting structure, thanks to the presence of holes on the seat, allows the drainage of water. A knob under the seat allows to rotate the seat to get in and out of the tub with minimal effort. A rotation blocking system is present in material resistant to wear and corrosion.

MATERIAL

Seat and Back: Polypropylene, rigid and water repellent material

Frame: 18/8 Stainless Steel

Anchor Points: Non-Slip Material

Dimensions	
Seat dimensions	40 x 38 cm
Base dimensions	74 x 50 cm
Distance between the armrests	55 cm
Backrest height	40 cm
Tub Support Edge Width	From 55 cm to 69 cm

PRECAUTIONS

Before use, always check the correct positioning of the device on the edges of the bathtub and the tightening of the side screws. Make sure the chair is stable by checking that the locking lever under the seat is properly fixed in the hole of the desired position. Always assist users with limited ability to walk.

USE

- Rest the seat on the bathtub and tighten the clips to make it stable.
- Rotate the seat to position it in front of the patient.
- If necessary, help the patient position himself on the seat.
- Rotate the seat again, positioning the patient inside the bathtub.
- Use the armrests as an aid to move and position yourself correctly on the seat.

Cleaning
Washing can take place with warm water and neutral soap
Disinfection can take place with simple products of normal domestic use

MAINTENANCE

Periodically check the functioning of the locking lever under the seat. Check the tightness of the welds paying attention to cracks or lifts of paint.

Check the correct tightening of screws and bolts and, in particular:

- the seat fixing screws, paying attention to the condition of the supports on which they are screwed (there must be no cracks or breaks);
- the nut fixing the central pin under the seat.

STORAGE

Store away from light, heat and moisture. Avoid direct exposure to sunlight.

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.

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