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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result. Consult your doctor or health care professional before using a mobility aid.

Distributor: This manual must be given to users of this product.

PRODUCT DESTINATION

The swivel shower seat is a technical toilet aid for people with reduced mobility elderly, disabled, handicapped, injured or those who can easily lose their balance. It is simple to use and makes the shower safe to preserve and encourage autonomy and independence.

A very practical shower chair that allows you to swivel in your shower tray. In a shower with limited space, it is easier to sit facing the door and then swivel towards the fitting. Comfortable all-round seat. Non-slip tips ensure the stability of the chair. Pivoting thanks to a simple lever placed under the seat. Seat drilled to facilitate water drainage.

SAFETY PRECAUTIONS

 Failure to follow the safety precautions may result in the risk of serious injury.

- Make sure that the product is in good working order.
- Check Visually the product
- Make sure the button is engaged and that all 4 feet are level at the same height.
- Use on a flat surface
- Check the seat and backrest fixing wheels are securely fastened.
- Make sure when installing on fibreglass bathtubs, excessive tightening of the clamp may crack the bathtub.
- Make sure the anti-slip foot is in good condition.
- When not in use, dry the shower chair with a cleaning cloth, do not leave it soaking in water for a long time.
- For users with physical disabilities, please use the shower chair with the help of an assistant.
- Do not use the shower chair before tightening the screw.
- Avoid hitting or scratching the covering during use, avoid dust and clean term.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING

a. Mount the seat kit onto the base
b. Tighten up the limit screw

Tighten up the outer hexagon screw to make sure the seat kit couldn't fall out.

CARE AND MAINTENANCE

 Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

 Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing and storing your product :
 - In a dry and temperate place
 - Protect your shower chair by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
 - All components must be stored without any load (do not place any objects that are too heavy on the shower chair elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

TECHNICAL FEATURES

Overall dimensions	Width 56 x Depth 53 x Height 85 cm
Seat dimensions	width 44 x depth 40 x height 51.5 cm
Wheelbase	56 x 53 cm
Anti-corrosion aluminium structure painted white	
ABS seat - Color White	
Weight	5 kg
Maximum Weight supported	114 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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