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USER MANUAL  
FOLDABLE SHOWER CHAIR  
SKU 02.02.538500

## INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

## GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

## INTENDED USE

Bath safety are all those devices that can be used by a disabled or elderly person to prevent, compensate or eliminate the difficulty in performing all the functions in the bathroom.

## TECHNICAL DATA

| Code         | Capacity (Kg) |
|--------------|---------------|
| 02.02.538500 | 120           |

## DESCRIPTION

The FOLDABLE SHOWER CHAIR is fixed to the wall through four anchor points. The seat is rectangular with holes for water drainage and side handles. Moreover, it has two legs height adjustable, the structure is foldable thus reducing the overall dimensions.

| Dimensions          |                    |
|---------------------|--------------------|
| Seat Width          | 51 cm              |
| Seat Depth          | 31.5 cm            |
| Height              | From 43 to 57.5 cm |
| Dimensions (Folded) | 15 x 51 cm         |

## INSTALLATION

The fixing plate has holes to be fixed to the wall. Installation by competent personnel is recommended. The seat must be fixed to a wall capable of withstanding the weight of the user.

**Important!** Remove the clip and slide the internal barrel until the desired height is reached; insert the clip to stop the leg at the chosen height.

Decide the height of the legs BEFORE fixing the plate to the wall.

### Before each use:

- Always check the state of wear of the parts to guarantee a safe use.
- Check the correct assembly of the device.

## PRECAUTIONS

- Always help the patient with difficulties to position on the seat.
- Do not exceed the maximum capacity.
- Caution in case of wet or slippery floor.
- Please keep the device in a dry place. Do not store the product outside.

## CLEANING

|                                                                         |
|-------------------------------------------------------------------------|
| Washing can take place with warm water and neutral soap                 |
| Disinfection can take place with simple products of normal domestic use |

## GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

## WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.